# Plug & Pure - APST 000

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## **General instructions**

Terms of warranty .....	13
Handling .....	13
Disclaimer .....	13
Specific safety instructions .....	13
Operating and fitting instructions .....	14

<b>System overview .....</b>	<b>15</b>
------------------------------	-----------

## **Technical details**

Technical data .....	16
Filter capacity of the filter cartridge .....	16

## **Installation instructions**

Installation instructions .....	17
Fitting the filter system .....	17
Connection to the Miele small steriliser.....	18
Installation of the water filter system.....	19

## **Handling**

Time for filter cartridge change .....	20
Filter cartridge exchange.....	20
Disposal of the filter cartridge .....	20
Maintenance .....	21
Troubleshooting .....	21

Miele Plug & Pure is a water filter system for demineralising water using ion exchange resins. In the flow process, the total salt content of the water to be filtered is reduced to a minimum, whereby deposits in downstream terminals are avoided.

The water filter system is supplied with a non-return valve tested by the DVGW, which fulfils the requirements of DIN EN 1717 as a safety valve.

Typical application areas are steam sterilisers in accordance with EN 13060.

### **Handling**

All surrounding conditions for storage, transport and operation in accordance with the technical data must be observed. Installation may only be carried out by qualified personnel. Proper use of this product requires the installation, use and maintenance described in these instructions for use. Only use filter cartridges intended for the present filter system.

### **Disclaimer**

Installation and exchange of the cartridge must be carried out precisely in accordance with these instructions for use. Miele and the filter system manufacturer are not liable for any damage, including consequential damage, resulting from incorrect installation or use of the product.

### **Specific safety instructions**

As feed water for the water filter system, only non-microbiological contaminated water in drinking water quality may be used.

Pressure and temperature ranges must be adhered to in accordance with the technical data.

The treated water is classified in category 2 in accordance with EN 1717.

After commissioning, the filter system should not be taken out of service for a longer period of time. After a break in use of more than a week, the filter must be flushed with a volume of 6 litres. Replace the filter cartridge after a break in use of more than four weeks. The maximum period of use is 12 months.

The filter system must not be opened or removed during operation. The filter cartridge may not be opened.

## General instructions

---

### **Operating and fitting instructions**

A shut-off valve must be installed in front of the input hose of the water filter system.

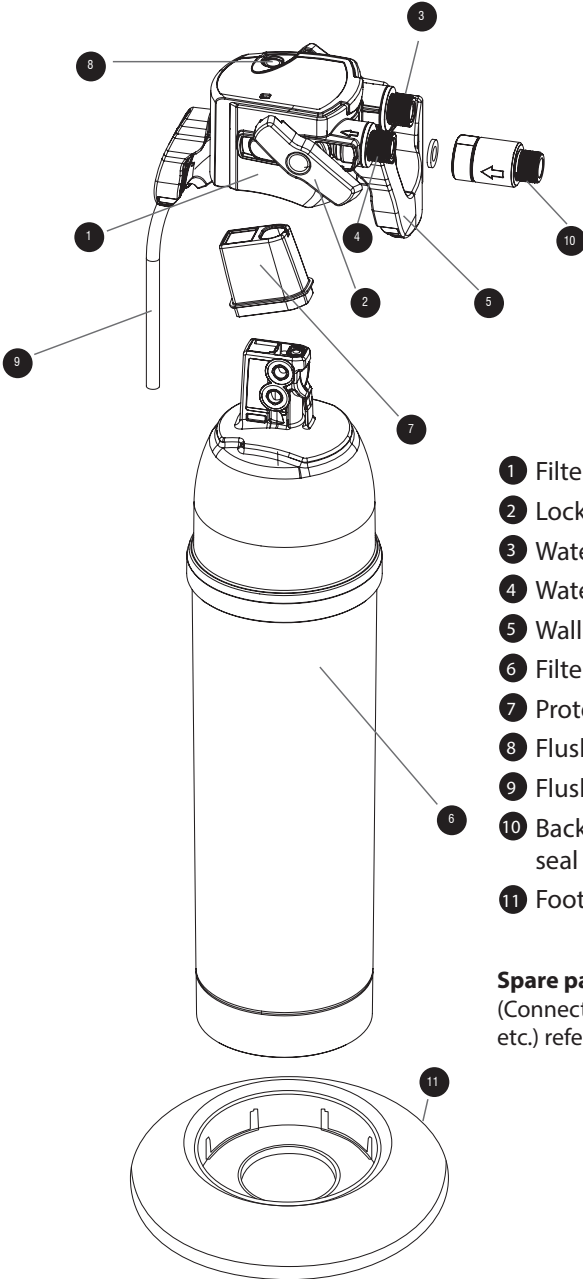
At a water input pressure  $>860$  kPa (rel), a pressure reducer must be installed in front of the filter system.

Do not install near to sources of heat or naked flames.

Protect the filter system against sunlight and mechanical damages.

The terminal device as well as the materials of all parts in contact with water after the Miele water filter system must be suitable for operation with demineralised water.

All parts must be installed in accordance with country-specific guidelines for drinking water facilities.

- 1 Filter head (11479320)
- 2 Locking handle
- 3 Water outlet (G3/8")
- 4 Water inlet (G3/8")
- 5 Wall holder
- 6 Filter cartridge (11479360)
- 7 Protective cap
- 8 Flush valve
- 9 Flush tube
- 10 Backflow inhibitor 3/8", incl. flat seal (11479350)
- 11 Foot (11479340)

**Spare parts for on-site connection**  
(Connection hoses, connection adapters etc.) refer to "installation advice" chapter

## Technical details

### Technical data

Operating pressure .....200- 860 kPa (rel.)

Water start temperature .....4°C to 30°C

Ambient temperature

- in operation.....4°C to 40°C

- in storage ..... -20°C to 50°C

Nominal flow.....60 l/h

Loss of pressure at nominal flow .....200 kPa

Bed volume..... 2.7 l

Weight (dry / wet) ..... 2.7 / 3.9 kg

Dimensions

- Filter system (WxDxH) ..... 124 x 123 x 476 mm

- Filter cartridge (WxDxH) ..... 123 x 123 x 469 mm

Operating position.....vertical

Water connection..... 3/8"

### Filter capacity of the filter cartridge

Conductivity in input water in $\mu\text{S}/\text{cm}$	Miele Plug & Pure Filter capacity in litres
50	2250
100	1130
200	565
300	375
400	280
500	225

The specified filter capacity refers to a shutdown time of the output water of 10  $\mu\text{S}/\text{cm}$ .

Due to external influencing factors (fluctuating raw water quality, water temperature, etc.), this information may deviate by +/- 10%.

**Note:** The filter capacity can be monitored with a commercially available conductivity indicator. When connected to Miele CUBE and CUBE X models of small steriliser, the conductivity is monitored via an internal sensor of the unit.

## Installation instructions

Non-commissioned systems (new filters) that have been transported and stored below 0°C must be stored in their original packaging at least 24 hours before start-up at the ambient temperature of the installation site.

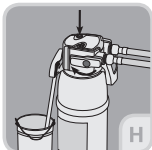
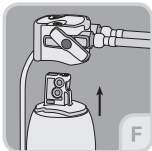
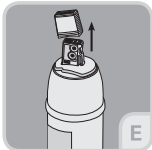
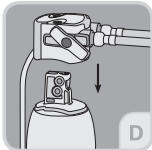
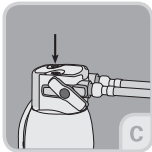
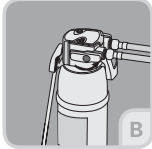
Please observe the technical data and the operating and safety instructions in this operating manual before commissioning.

EN

## Fitting the filter system

- A Close the water supply and switch off any existing power supply of the terminal.
- B Mount the wall holder in the intended position or **alternatively** set the foot to the intended position.
- C Mount the backflow inhibitor on the filter head inlet. Attention: Use the supplied flat seal. The max. tightening torque must not exceed 14 Nm. Attach the tube or pipe connection. At the inlet, the installation of a shut-off valve is recommended.
- D Click the filter head into the wall holder. (not required if you are using the foot).
- E Pull tab off protective cap of filter cartridge upwards, remove protective cap. Note the next replacement date.
- F Open the filter head lock. Insert the filter cartridge vertically in the filter head and pay attention to the correct position. Then turn the locking handle until it clicks into place.
- G If using the foot, insert the cartridge into the foot. Open the water supply and, if necessary, switch on any existing power supply of the terminal.
- H Operate the 'flush valve' and flush the filter system until bubble-free and the filtered water runs clear. At least 5.4 litres must be flushed.

Note: When venting/flushing, collect the water from the flush tube in a suitable container and check the system for any leaks.


# Installation instructions

EN


## Connection to the Miele small steriliser

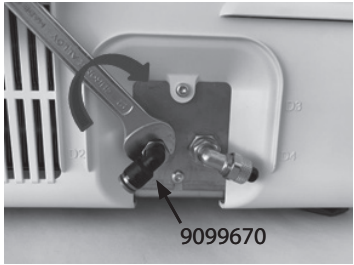1) De-installation of the standard connection with a size 9 wrench.

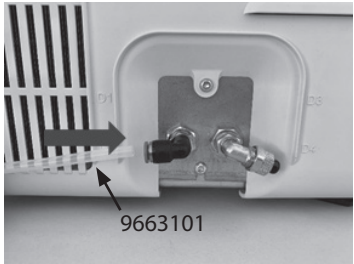2) Installation of the plug-in connection with a size 9 wrench.

3) Connection of the PTFE tube.


Installation advice:  
When cutting the tube, maintain a 90° cutting angle.


### Note

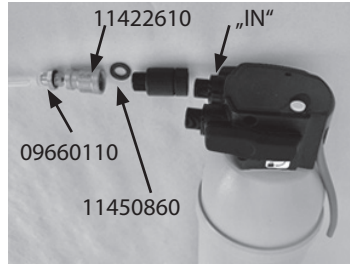To connect another Miele small steriliser, the Miele spare part 'T-piece MN: 1056794x' can be used.


## Installation of the water filter system

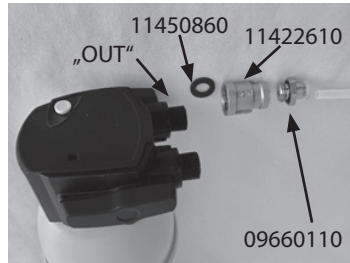4) Installation of filter head: On-site water supply.

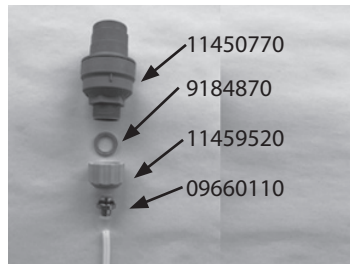5) Installation of filter head: Connection to the unit.

6) Connection of a flow reducer to the on-site water supply.

Installation is finished.

### Filter system set up advice

Do not place behind the unit (source of heat). When installed close to the unit, maintain a minimum distance of 10 cm (heat source).

## Time for filter cartridge change

For small sterilisers according to EN 13060, we recommend changing the filter cartridge from a conductivity of the demineralised water of 15 - 20  $\mu\text{S}/\text{cm}$ .

## Filter cartridge exchange

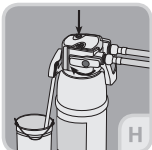
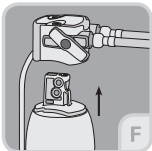
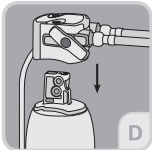
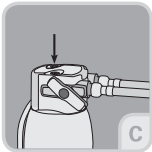
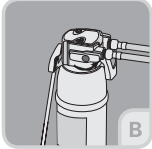
- A Close the water supply and switch off any existing power supply of the terminal.
- B Open the locking handle
- C Actuate the 'flush valve' and depressurise the system
- D Remove the used filter cartridge from the filter head.
- E Pull tab off protective cap of filter cartridge upwards, remove protective cap. Note the next replacement date.
- F Open the filter head lock. Insert the filter cartridge vertically in the filter head and pay attention to the correct position. Then turn the locking handle until it clicks into place.
- G If using the foot, insert the cartridge into the foot. Open the water supply and, if necessary, switch on any existing power supply of the terminal.
- H Operate the 'flush valve' and flush the filter system until bubble-free and the filtered water runs clear. At least 5.4 litres must be flushed.

Note: When venting/flushing, collect the water from the flush tube in a suitable container and check the system for any leaks.

## Disposal of the filter cartridge

Used filter cartridges are classified as non-hazardous special waste. They must be disposed of in accordance with the local, regional and state regulations.

The CER number for wetted or used ion exchange resins intended for the production of demineralised water is 19 09 05.

## Maintenance

Check the filter system regularly for leaks and the tubes regularly for kinks. Bent tubes must be replaced.

The complete filter system should be replaced after 10 years, the backflow inhibitor every 5 years.

Clean the filter system regularly from the outside with a soft, damp cloth. Do not use corrosive chemicals, cleaning solutions or harsh detergents.

## Troubleshooting

### No flow of water

Cause ..... Water supply shut off  
 Solution ..... Open water supply to the upstream shut-off valve ..... Check and close the locking handle on the filter head.

Cause ..... Backflow inhibitor has been mounted on the output side.

Solution ..... Mount the backflow inhibitor on the input side.

### No or limited flow of water despite opened water supply

Cause ..... Pipe pressure too low  
 Solution ..... Check pipe pressure. If the error persists, check the filter system and filter cartridge, and replace if necessary.

Cause ..... Filter head not mounted in the direction of flow.

Solution ..... Remove the filter head and mount in the direction of flow.

### Leak

Cause ..... Screw connections not properly fitted.

Solution ..... Check pipe pressure. Check all screw connections and check that seals are present. Replace the filter system if the error still occurs.

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elector GmbH