Adhesive

Gebrauchsinformation

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Instruction for use

EN

Carefully read the instructions prior to use.

Product description

COLTENE Adhesive is a universal adhesive for silicone-based impression materials for use in dentistry. It can be used for all conventional impression trays (e.g. metal).

Intended Use

A thin layer of COLTENE Adhesive serves as an adhesive between the impression material and the impression tray. It prevents the impression material from detaching from the impression tray during removal.

Composition

Main component: Polysiloxane

Instructions for Use

Used in combination with medical devices.

Contraindication

In case of known allergies to the main ingredient of COLTENE Adhesive, do not use.

Safety instructions

DANGER

- H225 Highly flammable liquid and vapour.
- H400 Very toxic to aquatic life.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P233 Keep container tightly closed.
- P273 Avoid release to the environment.
- P280 Wear protective gloves/protective clothing/ eye protection/ face protection
- P391 Collect spillage.

GHS02 GHS09

WARNING

Contains Hexamethylsiloxane.

Use only in a well-ventilated areas!

Keep away from sources of ignition!

Always wear gloves.

Close bottle immediately after use!

The use of COLTENE Adhesive may only be undertaken by

Safety instructions

DANGER

- H225 Highly flammable liquid and vapour.
- H400 Very toxic to aquatic life.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P233 Keep container tightly closed.
- P273 Avoid release

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