1105A

2

3

1

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
INSTRUCTIONS FOR USE 1105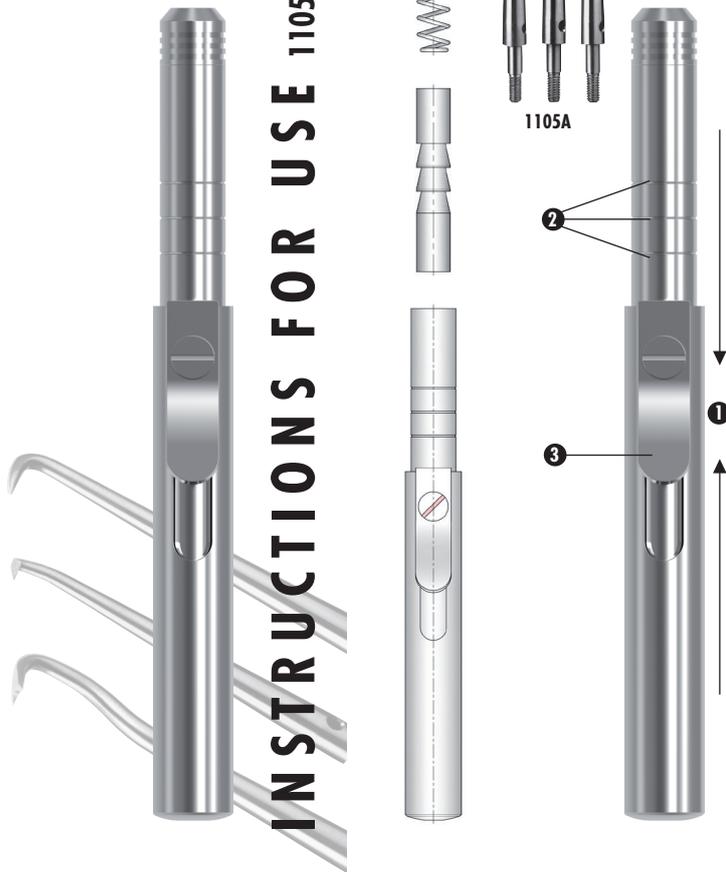

INSTRUCTIONS FOR USE

For automatic crown and bridge remover
AUTOREM Ref. 1105 including 3 tips +
1 removal key.

Tips are only available in a set of 3 pieces Ref. 1105A
(designs of the keys: see illustrations)

In use

The crown and bridge remover AUTOREM allows atraumatic removal of temporary or fixed tooth replacements.

The specially designed tips have to be screwed into the receptor thread using the removal key included.

The impulse force is set by pushing together the hand piece ①. The impulse force can be adjusted in three levels. These are marked on the instrument (1-3) ②.

Place the removal key onto the crown margin and push against the key towards the crown. This ensures, that during impulses the key is not displaced.

The impulse can be released via the lever ③.

We recommend starting with the lowest level (stage 1). Should the impulse force not be sufficient at the first stage further levels (stage 2 and 3) shall be used.

Sterilisation, Disinfection

The automatic crown and bridge remover is completely autoclavable. The hand piece is not applicable for thermo-disinfection. The removal keys can be used also in the thermo-disinfection. You can find detailed information in our instruction manual QSA313.