Gebruiksaanwijzing
Instrukcija obslugi
Instruções de uso

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DIRECTIONS FOR USE

- Pretreatment of the material to be bonded Perform appropriate pretreatment for each material to be bonded (composite resin, porcelain, metal or natural tooth).
 - Follow the instructions of each pretreatment material.
- Setting of Needle Tip Follow the instructions below to attach the BEAUTIFIL Needle Tip provided.
 - Pull off the shield cap while turning it slowly. (Fig.A)
 - Attach the needle tip by turning it clockwise until it is securely attached. (Fig.B)
 - Wear clean medical (dental gloves) when replacing the needle tip.

- Application Apply necessary amount of BEAUTIFIL OPAQUER directly to the material to be bonded. BEAUTIFIL OPAQUER may also be extruded onto a paper pad to be applied with a suitable instrument.
 - When storing these materials in a refrigerator, allow them to reach room temperature before use.
 - The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.
 - BEAUTIFIL OPAQUER should be applied as thinly as possible (0.5 mm or less)
 - Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.

- Light-curing Light-cure using a dental light-curing unit.
 - When applying thickly, incremental placement and light-curing are required.
 - When the curing surface or the area to be repaired is large, light-cure in segments.
 - Dental light-curing unit and light-curing time

Light-curing unit	Halogen	LED
Light-curing time	40 sec.	20 sec.
Wavelength	400-500 nm	440-490 nm
Light Intensity	≥ 500 mW/cm ²	≥ 1,000 mW/cm ²

- Use a dental light-curing unit having specifications equivalent to those described above. When using the one having performance other than these recommendations, follow the instructions of each unit to light-cure BEAUTIFIL OPAQUER properly.
 - Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light. Operators must also protect patients' eyes from direct exposure to curing light.
 - Disinfect the syringe and shield cap with alcohol after each use to avoid cross-contamination. Then, tightly close the cap. Use caution to avoid including alcohol into the paste.
 - Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.
- Fill direct composite resin and light-cure. Then, contour, finish and polish. Follow the direct restorative manufacturer's instructions for use.

NOTE ON USE

- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example) >>>>MM-DD → Year-Month-Date of the expiration date)

STORAGE

- Store in a dark place at room temperature (1-30 °C /34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Discard the used Needle Tip and store this product in either way of the following.
 - Attach new one. Insert the needle into the small hole on the head of the shield cap. (Fig.C)
 - Replace the shield cap securely. (Fig.D)

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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