(FR)

**COMPOSITION**

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(EN)

**COMPOSITION**

Powder composed of tricalcium silicate, zirconium oxide and povidone.

Aqueous solution of calcium chloride and polycarboxylate.

**INTENDED USE**

Permanent root canal sealer

**CLINICAL BENEFIT**

Preservation of the tooth functionality

**INTENDED POPULATION**

Permanent and mature teeth

**INDICATIONS FOR USE**

Permanent root canal filling in combination with gutta-percha points in case of inflamed or necrotic pulp.

Permanent root canal filling in combination with gutta-percha points following a retreatment procedure.

BioRoot™ RCS is suitable for use in single cone technique or cold lateral condensation.

**CONTRA INDICATION**

Hypersensibility to one of components

Immature or temporary teeth

**PERFORMANCE CHARACTERISTICS**

**Technical performance**

Seal hermetically the root canal.

**Technical specifications**

Setting time from 45 minutes to 4 hours allowing a sufficient working time.

Radiopaque, non-staining, dimensionally stable, non-soluble in fluids, adhesion to mineralized tissues and gutta-percha.

**Properties**

- BioRoot™ RCS is highly biocompatible and reduces the risk of adverse tissue reaction.
- BioRoot™ RCS crystallization creates a tight seal within the dentin tubules.
- BioRoot™ RCS fosters mineralization of the dentin structure and facilitates the physiological bone process. BioRoot™ RCS therefore creates a favorable environment for periapical healing.
- BioRoot™ RCS limits bacterial propagation.
- BioRoot™ RCS can be easily removed from the root canal if a retreatment is required.

**CLINICAL PERFORMANCE**

Permanent root canal obturation of mature teeth.

**EXPECTED LIFETIME**

At least 5 years

**MIXING INSTRUCTIONS**

- Collect the powder with the spoon supplied in the box. Put a level spoonful of the powder on the mixing pad.
- Detach a mixing solution single dose container. Twist cap to open. Pour 5 drops from the single dose container on the mixing pad.
- Prepare the root canal sealer by progressively adding powder to the liquid. Mix until obtaining a smooth paste (about 60 seconds). The mix must be made extemporaneously.
- Immediately rinse and clean the instruments to remove any residual material.

**PLACEMENT INTO THE ROOT CANAL**

- Perform the root canal cleaning and shaping procedure using standard endodontic procedures.
- Select standardized gutta-percha master cone and check that it fits snugly at the working length.
- Enjuague con una solución salina estéril para eliminar cualquier residuo de solución irrigante y seque sin deshidratar totalmente el conducto radicular.
- Prepare BioRoot™ RCS according to the above mentioned mixing instructions.
- Apply a coating of BioRoot™ RCS onto the canal walls using either a paper point or the gutta-percha point.
- Complete the obturation by inserting the gutta-percha master cone previously coated with BioRoot™ RCS (single cone technique) or several coated gutta-percha points (lateral condensation technique).
- Assess the quality of the root canal filling with a radiography.
- Use a hot instrument to sever the protruding gutta-percha.
- Perform the temporary restoration using any temporary filling material.
- If a post placement is required, it should be performed after complete hardening of BioRoot™ RCS, i.e. between 1 week and 1 month.

**RETREATMENT PROCEDURE**

Remove the existing sealer (BioRoot™ RCS or another sealer) using conventional removal techniques. Then follow instructions given in the sections "Mixing instructions" and "Placement into the root canal".

**WARNINGS**

- Do not use the mixed paste on several patients to avoid cross contamination.
- The working length in the root canal should not exceed 2 mm from the radiological apex to avoid apical extrusion.
- The device may cause eye irritation and skin burn. Do not manipulate the device without appropriate personal protective equipment.
- Use a rubber dam to ensure that the cement is not in contact with the mucosa or swallowed.
- Use the spoon provided with BioRoot™ RCS. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Do not use BioRoot™ RCS powder with another liquid than BioRoot™ RCS liquid. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Use the recommended ratio of liquid/powder. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Do not use a warm technique obturation. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Using BioRoot™ RCS without gutta-percha points jeopardizes endodontic and retreatment procedures.

**DISPOSAL**

No specific recommendations

**STORAGE**

Do not store above 30 °C; protect from moisture.

Use the powder within 6 months after first opening.

Liquid in single-dose containers must be used immediately after opening. Any remaining material shall be discarded.

Use the 5 liquid single-dose containers within 3 months after first opening of the foil.

**PRESENTATION**

Assembly kit for 35 applications, containing:

- 35 liquid single-dose containers
- 1 bottle of 15 g powder
- 1 spoon

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