# **BioSonic<sup>®</sup> UC50D/UC50DB**

---

## **Ultrasonic Cleaning System**

- EN** Owner's Guide
- DE** Benutzerhandbuch
- FR** Guide d'utilisation
- ES** Manual de usuario
- IT** Guida per l'utente
- NL** Gebruiksaanwijzing
- SV** Bruksanvisning
- DA** Betjeningsvejledning
- FI** Käyttöohje
- NO** Bruksanvisning
- PT** Guia do Utilizador
- EL** Εγχειρίδιο χρήσης
- ZH** 用户指南

## An essential part of your infection control system

Congratulations on your purchase of a BioSonic® Ultrasonic Cleaner — the finest, most efficient cleaning system in dentistry. Developed specifically for the dental industry, BioSonic units feature unique product design and optimize time and safety in the office or laboratory. BioSonic is a complete ultrasonic cleaning system, with a full line of cleaning solutions and unit accessories, which provides you with the flexibility to efficiently clean instruments, prostheses and other appliances.

## What is Ultrasonic Cleaning?

Ultrasonic Cleaning is a process created by high frequency sound waves. The sound waves, enhanced by specially formulated cleaning solutions, create high energy cavitation. During cavitation, millions of tiny bubbles form and then collapse or “implode,” releasing enormous amounts of energy and shock waves which scour the surface of instruments, appliances and other devices placed in the cleaning solution. This powerful scouring action reaches into minute crevices, which manual brushing cannot reach. The combination of energy and specially formulated solutions make ultrasonic cleaning the most effective method for removing gross and microscopic debris.

**Intended Use:** The BioSonic® UC50 Ultrasonic Cleaner is intended for cleaning medical instruments by the emission of high frequency soundwaves.

## I. Unpacking

1. Remove your BioSonic unit from the shipping carton and inspect the unit for any possible damage which may have occurred during shipping.
2. Compare the serial number of the unit (found on either the underside or rear of the unit) with the number on the shipping carton. If they are not the same, contact your dealer.
3. Fill out the warranty card and mail to Coltène/Whaledent within 10 days of receipt.
4. Select your desired location for the cleaner. It should be in close proximity to a sink or waste lines to facilitate draining of soiled solution.

## II. Operating Instructions

### A. General Cleaning

1. Plug the power cord into the power cord receptacle on the back of the unit, then plug the cord into a three-wire grounded AC outlet (in accordance with local regulations).
2. Fill the unit with water to the “fill” line indicator (Fig 1).
3. Pour the required amount of ultrasonic cleaning solution (either BioSonic General Purpose, BioSonic Germicidal or BioSonic Enzymatic Ultrasonic Cleaner) into the tank.

**CAUTION!**

- (a) Never operate the unit without solution. Operating a dry tank may result in permanent damage to the unit.
- (b) Never place objects directly on the bottom of the tank. Use accessory BioSonic baskets, cassette racks or a beaker to avoid damage to the unit.

4. To activate the unit, depress the keypad for the selected cleaning time. Unit will automatically shut off at the end of the cycle. To cancel the cycle at any time, press “OFF.”
5. Change the solution in the tank daily, or sooner if it appears soiled or discolored. When beakers are used, change the solution in the beaker after each use.
6. Empty the unit by tilting it over a sink (Fig. 2). Make sure the solution pours out of the tank over the indicator arrow (Fig. 3).

### B. Degassing

Degassing the ultrasonic solution is an important step. Degassing is the removal of air bubbles found in all fresh solution. These bubbles act as a barrier to efficient cleaning.

Degassing is necessary every time new solution is added. Degas the solution by activating the unit for 15 minutes before you add any items to be cleaned.

### C. Cover

The BioSonic Ultrasonic Cleaner should always be covered when operating to trap aerosols in the unit. The BioSonic UC50DB is equipped with a “hinged” cover for easy one hand opening. The cover also conveniently lifts off for cleaning or for use in spaces with low cabinets above the unit.

### D. Instrument Draining

The design of the UC50DB directs condensation back into the tank, keeping the work area around the unit dry. The housing acts as a draining stand for the basket. Simply position the basket on the bezel of the tank (Fig. 4) and allow the solution to drain off instruments.

## III. Cleaning Methods

### A. In Tank

**Basket Cleaning:** Place parts to be cleaned into the basket (Fig 5). Then place the basket directly into the main tank containing BioSonic General Purpose, Germicidal or Enzymatic Ultrasonic Cleaner. Clean according to instructions on bottle label.

### B. In Beaker

For Specialized Item Cleaning (e.g., prostheses): Place item into a beaker, add desired cleaning solution (e.g., BioSonic Cement Remover) according to the product's directions. Suspend the beaker in the tank by the use of the beaker holder and place the beaker positioning ring around the beaker so that the bottom 2/3 of the beaker is suspended in the main bath.

## IV. Accessory Use Directions

### A. Instrument Basket


BioSonic UC58 Instrument Basket increases instrument processing efficiency and prevents instruments from damaging the tank. **Use Directions:** Place instruments across ridges in bottom of basket (Fig. 6). Place basket in tank and clean for desired time. After cleaning, rinse basket and instruments thoroughly under running water. After rinsing, the basket may be placed on the housing for draining (Fig. 4).

### B. Beaker Holder

Place two UC153 Beaker Holders on the housing. Place beaker positioning rings around the beaker so that at least 2/3 of the beaker is suspended in the main bath (Fig. 7). Clean for desired time.

## V. Environmental Conditions

- Intended for indoor use.
- Maximum altitude 2000 m.
- 

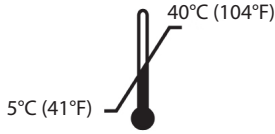- Main supply voltage fluctuations not to exceed  $\pm 10\%$  of the nominal voltage.
- Transient over voltages category II.
- Pollution degree 2.
- Class I equipment.

## VI. Cautions and Warnings

Note: Failure to comply with these Caution and Warning statements may result in personnel injury and/or equipment damage.

 CAUTION!	Do not operate the unit without sufficient solution in the tank. Do not use water alone as the carrier bath. Water is not a satisfactory coupling agent unless a wetting agent is used. For best results use BioSonic General Purpose, BioSonic Germicidal Cleaner or BioSonic Enzymatic Ultrasonic Cleaning solutions
	Do not place objects directly on bottom of the tank because they may block cavitation and cause buildup of heat in the transducer causing premature failure. Use only BioSonic UC50 accessory baskets, cassette racks or beakers to hold items.
	Do not restrict airflow. Provide adequate ventilation by maintaining at least 2 inches clearance at the back.
	Do not subject unit to shock or impact.
	Do not immerse the unit in water.
	This product is intended for use only by trained health-care professionals.
	Use this product in accordance with the instructions in the owner's guide
	Before cleaning items of considerable value ultrasonically, consult the item's manufacturer for cleaning precautions.
	The power off selection from the keypad does not turn off main power to the unit.
	This equipment may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures such as re-orienting or relocating the unit or shielding the location.
Unspecified or improper use of this product may impair safety protection..	

 WARNING!	Use only the supplied power cord or equivalent.
	Do not disassemble the unit. There are no user serviceable parts inside.

 WARNING!	Keep unit covered to reduce airborne contaminants.
	Do not place fingers or hands in the ultrasonic tank while in operation.
	Operation for long periods of time will cause the cleaning solution to get very hot. To avoid scalding, do not place hands in the solution or spill any onto you. Be careful when removing an item from the tank as it may also be hot.
	Do not use flammable liquids with Biosonic® units.

## VII. Maintenance

### A. Cleaning

All components should be cleaned each day by wiping with a soft cloth moistened with a commercially available mineral deposit remover followed by cleaning with isopropyl alcohol or a mild disinfectant.

 CAUTION!	Do not use petroleum based solvents, iodophors or phenolic based products (Iodophors and phenolics can stain the surface of the unit). Clean all liquid spills immediately.
---	---

### B. Stainless Steel Cleaning

#### Allowable:

- Wash with a clean cloth or rag soaked with warm water and mild soap or liquid dish detergent.
- Always clean in the direction of the original polish lines.
- Always rinse well with clear water (2 or 3 times) after cleaning. Wipe dry completely.
- Specialized household stainless steel cleaner may also be used.

#### Do Not Use:

- Steel or stainless steel wool or scrapers to remove stubborn debris.
- Harsh or abrasive cleaners.
- Do not let dirt accumulate.

#### Avoid:

- Cleaners that contain bleach.
- Products that contain chloride, fluoride, iodide, or bromide.

#### Chemicals harmful to use in the tank:

Acetophenone	Aluminum Chloride
Aluminum Floride	Aluminum Sulphate
Aluminum Bifluoride	Ammonium Chloride
Ammonium Hydroxide	Amyl Chloride
Antimony trichloride	Aqua Regia
Bromine	Calcium Bisulfate

Calcium Bisulfite	Calcium Hypochloride
Chloroacetic Acid	Chloric Acid
Chlorine Anhydrous	Copper Chloride
Copper Fluoborate	Ethyl Chloride
Ferric Chloride	Ferrous Chloride
Ferris Sulfate	Fluoboric Acid
Flourine	Hydrobromic Acid
Hydrochloric Acid	Hydrocyanic Acid
Hydrofluoric Acid	Hydrofluosilicic Acid
Iodoform	Mercuric Chloride
Muriatic Acid	Phosphoric (crude)
Sodium Hypochlorite	Potassium Chloride
Stannic Chloride	Stannous Chloride
Zinc Chloride	Sulfuric Acid

**C. Inspecting the Power Cord**

Periodically check the power cord for damage and see that the connectors at both ends of the cord are fully seated.

**D. Changing the Fuse**

1. Turn the main power switch to the "off" position.
2. Remove the power cord from the wall outlet first and then from the power cord receptacle on the rear of the unit.
3. The fuse is held by a fuse drawer which is inserted into the fuse holder. The fuse holder remains fixed to the UC50DB housing. These parts are located directly beside the power cord receptacle (Fig. 8). To access the fuse, depress the center tab on the fuse drawer and pull the drawer open (Fig. 9).
4. Remove the fuse from the fuse drawer.
5. Examine the fuse. If it is blown (Fig. 10), replace with a fuse which matches the fuse type listed under the unit specifications section of this manual. If fuse appears to be in good condition (Fig. 11) and power is not activated when the main power on/off switch is turned "on," return the BioSonic unit to your dealer or Coltène/Whaledent.
6. Slide the fuse back into the holder. To secure the fuse drawer, simply push the drawer closed.
7. Make sure the main power switch is "off." Plug the power cord into the power cord receptacle on the rear of the unit. Then plug the power cord into the wall receptacle.

**VIII. Unit Specifications**

UC50DB115	[115V ± 10%, 50/60 Hz, 70W max]
UC50DB230CE	[230V ± 10%, 50/60 Hz, 70W max]
UC50DB230UK	[230V ± 10%, 50/60 Hz, 70W max]
UC50DB240	[240V ± 10%, 50/60 Hz, 70W max]
Operating Frequency:	53.4 kHz
Tank Capacity:	.85 gallon or 3.2 L
Use Capacity:	.53 gallon or 2 L
Overall Dimensions:	11.5 x 9 x 7.75 inches or 29.2 x 22.9 x 19.7 cm
Tank Dimensions:	7.75 x 5.75 x 4.125 inches or 19.7 x 14.6 x 10.5 cm
Fuse Type:	100V Units: 5 x 20 mm, 1.25A/250V, Two each 115V Units: 5 x 20 mm, 1A/250V, Two each 230V Units: 5 x 20 mm, 500mA/250V, Two each 240V Units: 5 x 20 mm 500mA/250V, Two each

EMC Class A: Suitable for use in all establishments other than domestic and those directly connected to a low-voltage power supply network which supplies buildings used for domestic purposes

**IX. Symbols**

Caution/Warnings: See accompanying documents

Caution: Risk of electric shock

Fuse location

Temperature Limits

Humidity Limits

Proper Disposal Required

**X. Order Information**

Cat. No.	Description	Qty.
UC50DB115	Unit (115V ± 10%, 50/60 Hz)	1
UC50DB230CE	Unit (230V ± 10%, 50/60 Hz)	1
UC50DB230UK	Unit (230V ± 10%, 50/60 Hz)	1
UC50DB240	Unit (240V ± 10%, 50/60 Hz)	1

Accessories:		
UC58	Instrument Basket	1
UC153	One Beaker Holder	1
UC53	600 ml Beaker with Cover and Positioning Ring	1
UC55	One Beaker Cover	1

**Concentrated Cleaning Solutions:**

UC30 BioSonic®	General Purpose Cleaning Solution Concentrate in MeterDose® Dispensing Bottle (16 oz.)	473 ml
UC31 BioSonic®	General Purpose Cleaner + Super Rust Inhibitor	20 Single Dose Packets 15 ml
UC32 BioSonic®	Enzymatic Ultrasonic Cleaner Concentrate in MeterDose Dispensing Bottle (8 oz.)	236 ml
UC34 BioSonic®	Plaster & Stone Remover Powder Concentrate in Jar (1.85 lbs.)	840 g
UC38 BioSonic®	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle (8 oz.)	236 g

UC39 BioSonic®	Cement Remover Concentrate in MeterDose Dispensing Bottle (16 oz.)	473 ml
UC42 BioSonic®	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle (16 oz.)	473 ml

## XI. Disposal

The Biosonic® UC 50 is an electric appliance requiring proper disposal according to local laws and regulations.

## XII. Limited Warranty

The UC50 has been developed specifically for use in dentistry and intended to be operated only by dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer or distributor, whether written, verbal or by demonstration, shall not relieve the dental professional of his/her obligation to control the product and to make all professional judgments regarding its use.

This product has been carefully manufactured and has met stringent quality assurance requirements. Our products are manufactured from new parts or new and serviceable used parts. Regardless, our warranty terms apply. If, in normal use, it is found defective in material or workmanship within a period of one year from date of purchase, it will be repaired at Coltène/Whaledent's option, or replaced. Tampering with any of the components, misuse, negligence, alteration, water damage, accident or lack of reasonable or proper maintenance and care will void this warranty. Failure to use and/or maintain the product in accordance with the instructions contained in this manual will void this warranty.

**THE FOREGOING IS IN LIEU OF ANY EXPRESSED OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OR MERCHANTABILITY OR FITNESS FOR USE. COLTÈNE/WHALEDENT ASSUMES NO RESPONSIBILITY FOR ANY INCONVENIENCE, LOSS, INJURY OR DIRECT, INDIRECT OR CONSEQUENTIAL DAMAGE ARISING FROM THE POSSESSION OR USE OF THIS PRODUCT.**

Any claim for damage or breakage in transit should be made at once against the carrier. If factory service is required, be certain to properly pack your equipment and return prepaid and insured to the factory.

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Coltène/Whaledent Inc.
Coltène/Whaledent GmbH + Co. KG