## Gebrauchsanweisung Bio-Ink® Flow

# Bausch

### Hersteller:

Dr. Jean Bausch GmbH & Co. KG  
Oskar-Schindler-Str. 4  
50769 Köln Germany

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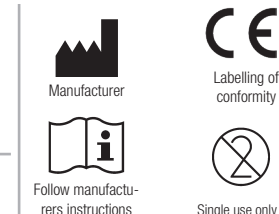
**REF**

### Contents:

- Bio-Ink® - Flow Paste (1ml)
- 5 single use applicator needles

**Bio-Ink® Flow**

### General description and purpose of the medical device:

Color Indicator Paste for checking sore spots on dentures and partial dentures

### Medical Product - For professional use only

### Indication

Bausch Intra Oral Ink = Bio-Ink® - Flow green is indicated for marking sore spots on the gum. Bio-Ink®-Flow is a highly viscous paste, containing green food dye.

It can be applied directly to the oral mucosa with the single use needles or the attached Magic-Brush™ applicator. The sore spot on the gum is gently marked. After replacing the denture, a green contact spot can be detected on the lower side of the denture showing the area to be adjusted.

Sore Spots also known as decubitus can occur frequently with new dentures or partial dentures. Also a degeneration of the alveolar ridge can cause sore spots as well as unbalanced occlusal forces on the denture. The adaption of the oral mucosa to the new denture often causes inflammatory lesions.

Because of its thixotropic properties the green paste can easily be applied with Magic-Brush™ applicator in spots, areas and lines. Also the Palatal Vibrating Line and the mucobuccal fold can be marked. If necessary the denture can be adjusted. Before application the areas to be marked on oral mucosa have to be dried.

### Contraindication

A direct application onto highly inflamed areas or purulent lesions of the oral mucosa areas should be avoided.

### Side Effects

Possible side effects could be allergic reactions.

### Notes

- The color paste can be applied directly with the disposable cannulas (single use applicator needles) and spread with the disposable applicator (e.g. Magic-Brush™, ) to avoid cross-contamination.
- Alternatively, the paste can be applied to a disposable applicator (e.g. Magic-Brush™, Easy Brush, ...) and then applied to the surface to be tested.
- The mucous membranes, palate and mandible must be dried with clean pulp or cotton roll previously.

### Application

Carefully dry the gum, the oral mucosa and the alveolar ridge with a clean cotton roll to avoid smearing of the color indicator paste on the gingiva.

There should be no saliva on the surface to be tested. The denture should be cleaned and dried as well. The color paste could also be applied with single use applicator needles onto the gingiva.

### Effect / Diagnosis

The color marking of the sore spot will be transferred to the denture when the denture is seated into position. When the denture is removed the spot is visible on the surface of the denture. If necessary the denture can be adjusted by the dentist.

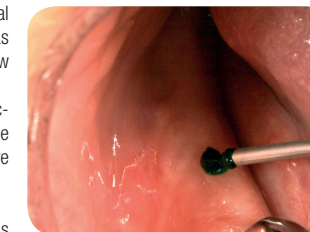
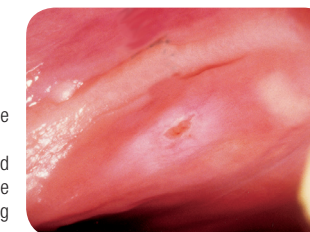
### Cleaning

After diagnosis, the color can simply be removed with water or any detergents.

### Disposal instructions

Bio-Ink® - Flow consists of a green food color and natural waxes, which are approved for food, cosmetics and pharmaceutical applications. All components can be disposed of with the usual practice waste.

**Note: Serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State.**

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Manufacturer

Labelling of conformity

Follow manufacturer's instructions

Single use only

## Bio-Ink-Flow

### Product:

Bio-Ink® Flow  
Sore Spot Indictaor  
BK 206

**REF**

### Contents:

- Bio-Ink® - Flow Paste (1ml)
- 5 single use applicator needles

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