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BRUXI+

Instructions for use for the production of a night guard

BRUXI+ is an immediate help against teeth grinding and also helps 3- to 12-year-old children with mandibular joint problems.

The contourless BRUXI+ resembles an impression tray and has the external appearance of a splint that can be made individually. For this purpose, an impression is made directly in the patient's mouth or with a jaw model using "DIRECT for BRUXI+" – a fast hardening silicone.

Bruxi+ is available in only one size for the lower jaw.

For use in milk teeth, cut about 1 cm of each side of the silicone splint with scissors. For this reason, a fine marking extending diagonally forward has been incorporated.

Shortening the Bruxi+

1 Seal it with SILCOfix

2 Fill it with the DIRECT silicone

**3 Impression
(on the model or jaw)**

Before the „impression“ is made, the inner sides and edges of BRUXI+ have to be generously sealed several times with SILCOfix.

It hardens in the mouth after only about 90 seconds; if the jaw model is used, it takes about 3 minutes. After approx. 5 minutes, the excess material can be cut off with scissors and the edges smoothed out with sanding discs.

Result of the impression

Further questions or problems? We will be happy to advise you.

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