CleanJoy®

MD EU Medical device

Carefully read the instructions prior to use**Product description:**

CleanJoy is a tooth cleaning and polishing paste used for professional tooth cleaning and polishing within the framework of a prophylactic treatment. The fluoride-containing paste (700 ppm fluoride) is offered in three different cleaning strengths, so that, depending on the degree of contamination, each patient can receive a treatment which addresses their individual situation.

The safe application and easy identification of a particular paste with its own distinct level of cleaning is ensured by the colour coding of the tubes, *SingleDose* and pastes:

Red (RDA* = 195) = High cleaning strength

Yellow (RDA* = 127) = Medium cleaning strength

Green (RDA* = 16) = Polishing paste

*RDA value as per DIN EN ISO 11609

Indications:

- Removal of surface, soft and hard plaque.
- Removal of extrinsic discolouration (e.g., staining from coffee, tea or tobacco).
- Polishing tooth surfaces within the framework of a professional tooth cleaning, prior to bleaching and following the removal of orthodontic devices.
- Cleaning and polishing restorations within the framework of a professional tooth cleaning.
- Removal of residues of temporary luting material prior to definitive bonding (cementation).

Contraindications:

CleanJoy contains artificial flavouring. **CleanJoy** should not be used in patients with a known hypersensitivity (allergy) to these constituents.

Patient target group:

CleanJoy is suitable for use in all patients without any age or gender restrictions.

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User:

CleanJoy should only be used by a professionally trained dental practitioner.

Use:**Processing:**

- **Tube:** Remove the tube cap and dispense some of the material onto a mixing pad. **CleanJoy** can be applied with any commercially available cup or small brush. After use, replace the tube cap.
- **SingleDose:** Use the tab to remove the protective foil. Remove the material with a commercially available cup or small brush.
- In order to avoid potential heat generation and splashing, an appropriate rotational speed should be used when working with the paste (2,000 to 3,000 rpm).
- A final treatment with the polishing paste (green paste) must follow each and every application of a higher strength cleaning paste (red or yellow paste). The polishing paste can also be used following treatment with a powder jet device.
- After the application, rinse the cleaned surfaces thoroughly with water, remove remaining material by means of suction and ask the patient to rinse out their mouth thoroughly.
- Each *SingleDose* contains sufficient material for a single use. It must only be used once and only on a single patient. Repeated use can result in cross-contamination.

Warnings, precautionary measures:

- In case of contact with the eyes, rinse thoroughly with water and consult a doctor.
- An excessive intake of fluoride may cause nausea. Do not swallow **CleanJoy**.
- The information and/or advice we provide does not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at 39°F to 73°F (4°C to 23°C). Do not use **CleanJoy** after the expiry date.

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Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **CleanJoy** must be reported to VOCO GmbH and the responsible authority.

PRODUCT ORDERING INFORMATION:

Tube 100 g fine mint	REF 2090
Tube 100 g medium mint	REF 2091
Tube 100 g coarse mint	REF 2092
Tube 100 g fine cherry	REF 6098
Tube 100 g medium cherry	REF 6099
Tube 100 g coarse cherry	REF 6100
Tube 100 g fine caramel	REF 6090
Tube 100 g medium caramel	REF 6091
Tube 100 g coarse caramel	REF 6092

<i>SingleDose</i> 200 x 2 g fine mint	REF 2096
<i>SingleDose</i> 200 x 2 g medium mint	REF 2097
<i>SingleDose</i> 200 x 2 g coarse mint	REF 2098
<i>SingleDose</i> 200 x 2 g fine cherry	REF 6102
<i>SingleDose</i> 200 x 2 g medium cherry	REF 6103
<i>SingleDose</i> 200 x 2 g coarse cherry	REF 6104
<i>SingleDose</i> 200 x 2 g fine caramel	REF 6094
<i>SingleDose</i> 200 x 2 g medium caramel	REF 6095
<i>SingleDose</i> 200 x 2 g coarse caramel	REF 6096

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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Instructions for use

Remin Pro is a water-based cream that contains hydroxyapatite, sodium fluoride (1,450 ppm fluoride) and xylitol. **Remin Pro** is available in melon, strawberry and mint flavour.

Remin Pro provides extra protection for teeth and, in doing so, it helps neutralise acids from acid-forming bacteria in plaque. **Remin Pro** additionally aids the neutralisation of other acids in the mouth.

Indications:

- After tooth whitening
- After professional tooth cleaning
- For the prevention and control of hypersensitivities
- During orthodontic treatment

Application:

1. Apply a bean-sized amount of **Remin Pro** to the teeth with a finger or suitable instrument (toothbrush or swab) and distribute.
2. The patient should distribute the remaining **Remin Pro** cream in his or her mouth with the tongue. **Remin Pro** and the saliva should remain in the mouth for as long as possible (minimum of 3 minutes) for an optimal result.
3. The patient should spit out the remaining amount. Rinsing should be avoided if possible. The patient should wait at least 30 minutes after the treatment before eating or drinking.

Also for daily use after oral hygiene.

Storage:

Store **Remin Pro** in a cool, dry area protected from direct sunlight (4 °C - 23 °C / 39 °F - 73 °F). Do not use the material after the expiry date.

Precautions:

1. **Remin Pro** contains parabens. **Remin Pro** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
2. Always close the tube immediately after use.
3. The application of **Remin Pro** is not recommended for patients under 6 years or for home use on children under 12 years.
4. In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention.
5. Immediately discontinue treatment and seek medical attention if symptoms of angioedema occur that could be attributed to the ingredients in **Remin Pro**.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognises its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

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For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.com/us/symbols

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Also for daily use after oral hygiene.

Storage:

Store **Remin Pro** in a cool, dry area protected from direct sunlight (4 °C - 23 °C / 39 °F - 73 °F). Do not use the mate-

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Our preparations have been developed for use in dentistry.

As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

Last revised: 12/2016

VOCO GmbH
Amon-Flettner-Str. 1-3
27472 Cuxhaven
Germany
Phone +49 (4721) 719-0
Fax +49 (4721) 719-140
e-mail: marketing@voco.com
www.voco.com

voco Grandio® Seal

EN Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 6874

Product description:

Grandio Seal is a light-curing, highly filled, fluoride-containing nano-hybrid fissure sealant with outstanding mechanical properties.

Grandio Seal contains 70 % w/w inorganic fillers in a methacrylate matrix and cures under blue light (halogen or LED).

Indications:

- Sealing/filling of pits and fissures
- Sealing/facing of damaged enamel surfaces
- Covering of caries predilection sites during orthodontic treatments
- Sealing of deciduous teeth

Contraindications:

Grandio Seal contains aromatic and aliphatic dimethacrylates as well as BHT. Grandio Seal should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Grandio Seal is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Grandio Seal should only be used by a professionally trained dental practitioner.

Use:

Fissures should be sealed as early as possible, as soon as the eruption of the tooth allows treatment and moisture control is possible. After isolating the operating site, thoroughly clean the surfaces to be sealed (rotating brush/cleaning paste), spray and dry.

In case of incipient carious lesions or unclear diagnosis, extend the fissure with a fine instrument, if necessary.

Etching:

Apply Vococid etching gel (or liquid) (blue colour for control), work in with a short-haired disposable brush and let it act for 15 - 30 s according to the condition of the enamel. Thoroughly spray and dry. The etched enamel should have an opaque and matt-white appearance (avoid contamination at all costs!). Follow the Vococid instructions for use.

Application:

Remove the protective cap from the syringe. Screw a new cannula onto the Grandio Seal syringe.

Grandio Seal flows homogeneously from the cannula upon medium pressure and thus is very easy to dispense accurately.

Apply Grandio Seal from the margin into the fissure or cavity and allow it to penetrate for 15 - 20 s. Remove any air bubbles with the fine cannula.

In order to ensure the functioning of the non-dripping syringe do not pull back the plunger during or after use.

The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap. The use of a protective barrier is recommended to protect the syringe against contamination with bodily fluids, dirt from hands or oral tissue.

Light-curing:

Polymerise Grandio Seal for 20 s per surface and check the sealing. Conventional polymerisation devices are suitable for light-curing this material. The light output should be at least 500 mW/cm² on halogen polymerisation devices and 300 mW/cm² on LED devices. Re-seal undercuts, remove excess cautiously (e.g. finishing diamond), polish if necessary (rubber polisher). Then apply a fluoride varnish on the treated and neighbouring teeth.

Warnings, precautionary measures:

- An operating site which cannot be kept sufficiently dry involves the risk of a (partial) premature loss of the sealing. In such cases, the application of a fluoride varnish or sealing with a glass ionomer cement is recommended until perfect sealing is possible.
- If the etched surface is contaminated (e.g. by saliva), the adhesion of Grandio Seal may be reduced. In case of contamination, clean and etch again for approx. 10 - 20 s, spray and dry.
- Adhesion can be increased by using a bonding agent after etching (e.g. in case of difficult moisture control/extended sealing). Follow the bonding agent manufacturer's instructions for use. Then apply Grandio Seal.
- Only expose Grandio Seal to the operating light for a short time as otherwise polymerisation will start (viscosity increases).
- Check the sulcus after application and remove residues of material.
- Avoid contact with the mucosa as far as possible. Do not use cleaning or drying agents containing phenolic substances (e.g. eugenol, thymol) as they inhibit complete polymerisation.
- Carious lesions (e.g. around fillings) can be stopped by appropriate sealing if renewal of the filling is not desired or possible. For this purpose, clean the areas affected, prepare as minimal as possible and etch in the enamel area. In such cases, sealing of the complete filling is recommended.
- In the case of orthodontic treatment in areas of reduced oral hygiene, sealing these areas can provide good and permanent protection. Fluoridation should be continued in all cases to protect the unsealed tooth areas from caries.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, TEGDMA, silicon dioxide, BisGMA, fumed silica, PEGDMA, initiators, stabilisers, pigments

Storage:

Store Grandio Seal away from light at temperatures between 4 °C and 23 °C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Grandio Seal must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of Grandio Seal can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA2090 E1 0920 99 © by VOCO