CYANO EX

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GB Instructions for Use

Cyanoacrylate remover

Removes fresh and old cyanoacrylate stains from surfaces or textiles.

Instructions for use:

Apply Cyano Ex to the stain and rinse with water. In case of strong and older stains, leave Cyano Ex on for some time and repeat the procedure, if necessary.

Tip:

Test on colored and delicate materials in a concealed area prior to use.

Important:

Use strictly as cleaning agent to remove cyanoacrylates, trays and textiles. Do not use in the oral cavity or on wounds under any circumstance – damaging to health!

Contains: Acetone, Ethyl acetate

Hazard statements:

Highly flammable liquid and vapour. Causes serious eye irritation. May cause drowsiness or dizziness. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Use only outdoors or in well-ventilated area. Wear protected gloves/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye persists: Get medical advice/attention.

Store in well-ventilated place. Keep cool. Store locked up. Dispose of contents/container to in accordance with local/regional/national/international regulation.

Repeated exposure may cause skin dryness or cracking.

Cyano Ex, 20 ml solution

REF 152 272

Danger