

Operating Instructions

1. General terms:

- 1.1. Product description:** DIA-GLACE **No. 300-0000**
1.2. Delivery form: Plastic tin containing 5 gr. Diamond paste hard / pink
1.3. Manufacturer: YETI Dentalprodukte GmbH - Industriestraße 3 -
D-78234 Engen/Germany
Tel.: +49-7733/9410-0 FAX: +49-7733/94 10 22
- 1.4. Storage:** store cool and dry / protect from UV radiation / protect from any blow or dropping

2. Use according to determinations:

For polishing of ceramic and synthetic materials used by dental technicians, especially for polishing of crowns and bridges **outside the patients mouth**. To be used with felt polishing wheels or polishing brushes which are suitable for a speed of rotation of 5.000 - 10.000 Upm.
Any claims on warranty or liability will not be accepted if not used according to determinations !

3. Caution:

- Use felt wheels only with suitable mandrels.
- Speed of rotation is of 5.000 - 10.000 Upm during usage.
- Ensure that unqualified persons (e.g. children) do not come into contact with this product (danger of swallowing).
- To avoid foreign elements in mouth or eyes do not work without safety glasses and mouth protection.

4. Operating Instructions:

4.1. Preparation

Put the felt wheel (or brush) onto a suitable mandrel corresponding speed of rotation and form of the felt wheel.

4.2. Application

- Pick up the diamond paste with felt wheel or brush with the suggested speed of rotation.
- **Attention: Consistency of the diamond paste Dia-Glace is hard and not pasty as comparable products. Due to this hard consistency Dia-Glace does not splash and is much more economic than other comparable polishing systems. Please observe that the hard consistency is normal and does not imply a claim for return !**
- Polish the desired surface with moderate pressure observing the suggested engine speed range.
- Hold the object which is to be polished firmly with your hand.
- Change the polishing direction of the surface again and again, we recommend to work in circles.
- As soon as polishing effect is reduced pick up some more diamond paste.

4.3. After application:

Clean the polished object under running water.

Used felt wheel or brush can be stored in suitable support until next application.

5. Disposal: According to current local regulations.

6. Durability: Consumer goods should be used rapid after opening and used up within a few months.

Dokument:	Erstellt am/von:	geändert am/von:	Revision:	freigegeben am/von:	Seitenzahl:
BA	09.10.1996/TB	08.04.2019/QM	0	08.04.2019/TB	Seite 1 von 1