Duplikat

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EN Warranty

Dentaurum guarantees a faultless quality of the products manufactured by us. The information in these instructions for use is based upon our own experiences. The user himself is responsible for the correct processing or use of the products. We cannot be held responsible for failures, as we have no influence on the processing. However, any claim for compensation only applies to the commercial value of our products.

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REF **008-030-00**

Dear Customer

Thank you for having chosen a quality product from Dentaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Our products are regularly modernized to the latest technical development. For this reason we advise examining the most up-to-date operating instructions supplied for any possible changes, even if the product is frequently used.

Instruction for use Duplikat

A reversible hydrocolloid duplicating material which produces precise refractory models. Is especially suited for the combination technique using phosphate bonded investments, such as rema® Exakt or rema® dynamic (top speed) and ethyl-silicate investments. Using Duplikat ensures that all secondary metal components will fit precisely with the primary structure. Preservatives protect Duplikat against fungal attack and guarantee long life when handled correctly.

Initial melting

1. Cut Duplikat into small pieces and put it into the duplicating unit without adding water.
2. Set the melt temperature on automatic duplicator to 93 °C / 199 °F and the pouring temperature to 50 °C / 122 °F.
3. If the double boiler method of preparation is used, the mixture must be stirred constantly. Do not bring the mixture to a boil.

Melting temperature: 93 °C / 199 °F

Pouring temperature: 50 °C / 122 °F

Duplicating – general notes

Remelting after first use

Clean Duplikat carefully under running water. Cut into small pieces and place into automatic duplicating machine or double boiler. Do not add water to gel.

Model preparation

Prepare model in normal fashion using heat resistant preparation and/or block-out waxes such as Dentaureum's preparation wax, REF 120-025-00, and/or Block-out Wax, REF 110-310-00.

Soak model prior to duplication

Clean model and place in warm (40 – 45 °C / 104 – 113 °F) water, teeth down, for 30 minutes.

Duplicating

After soaking remove excess water from model using a clean tissue or paper towel. Do not use compressed air. Secure model to metal duplicating flask base and put top into place. Use Dentaureum's metal flask, REF 165-700-00.

Duplicating material should be stirred continually during melting and cooling.

Pouring temperature to 50 °C / 122 °F (± 1 °C / 1.8 °F) .Carefully pour duplicating material into one side only of the metal flask, REF 165-700-00.

Allow flask to bench cool for 15 minutes before placing in water bath. Water level should be approx. 2 cm and the temperature between 18–25 °C/64–77 °F.

Allow flask to remain in water bath for approx. 30 minutes.

Note:

If a plastic duplicating flask has been used the setting time should be extended to approx. 40 minutes.

After setting, the master model is carefully removed using model tongs, REF 008-030-00. The model should be lifted straight up out of the mold to avoid damaging the duplicating gel. Investment material is then mixed according to the manufacturer suggested ratio and after max. 1 hour poured at room temperature (20–24 °C / 68–75 °F) into the clean, dry mold. After the required setting time of 40 minutes, the duplicating material is removed from the flask and then carefully peeled away from the model.

Important!

Duplikat and the mixed investment material should not be too cold (room temperature) otherwise the investment material will not set properly.

Remelting Duplikat

Clean Duplikat after each duplication under running water. Keep used material in a closed container.

Duplikat may be remelted up to 10 times. After that the material will begin to thicken until it reaches a syrupy consistency. Adding approx. 30 ml of water per kg will restore

the Duplikat to near new property values. This reconstituted mixture is suitable for further use but there may be a slight loss of quality.

Note:

Never mix new Duplikat with previously melted material.

Sources of faults and their remedies

Fault: Surface of refractory model is defective.

Cause: The hydrocolloid mold was too moist.
The mixing time for the investment material was too short.
The investment material was not of the proper consistency.
The investment material was not given sufficient setting time.
A longer setting time was needed because the duplicating material and/or the investment material was too cold.

Remedy: Always remove excess moisture from the mold before filling with investment material.
Observe all instructions for use of the investment material being used.

**Fault: Material tears easily.
Loss of transparency.
Smeary consistency.
Poor precision of fit.**

Cause: Duplicating material is old.
Duplicating material has been boiled.
Contamination from foreign material, e.g. cleaning agents, alum or electrolyte.

Remedy: Material is permanently damaged and should be discarded.

Fault: Casting does not fit properly.

Cause: Duplicating material has been reused too many times.

Remedy: Should be discarded

Fault: Duplicating material has become too thick.

Cause: Water loss in mixture due to evaporation.

Remedy: Add 10–30 ml of tapwater (per kg duplicating material).

Recommended melting units

duplicating unit (Dublitherm), microwave unit

Change of duplicating materials

Never mix old and new materials. When changing materials, clean the melting unit completely and fill it with new material. Do not use cleansing solvents.

Shelf life of Duplikat

60 months in the unopened original packaging.

Waste disposal

The duplicating gel may be disposed together with normal domestic refuse.
The empty packaging should be disposed after use according to local regulations.

Availability

Duplikat – 6 kg	REF 108-600-00
Duplikat – duplicating material liquid concentrate, 3 litres	REF 108-605-00

Accessories for duplicating

Duplicating flask Non-oxidizing metal flask for short gelling times with holder for sprueformer.	REF 165-700-00
--	-----------------------

Model lifting tongs	REF 008-030-00
---------------------	-----------------------

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