# Eugenol

Reactive liquid for the preparation of ZOE cement

**Composition** Eugenol 100%

## Properties

Mixed with Zinc Oxide, it reacts to form a Zinc Oxide-Eugenol cement (ZOE Cement) used in dentistry.

## Indications (ZOE Cement)

- ▶ Temporary obturation
- ▶ Indirect pulp capping

## Directions for use

Mix the zinc oxide powder with eugenol in the proportions of mass / liquid (P/L) between 1.1 and 1.7 (g/g).

Setting starts in the presence of moisture. An increase in humidity or temperature accelerates the setting.

The setting time (tested according to ISO 3107) is 6h50-16h according to the P/L ratio used.

## Storage

Store at room temperature between 5°C and 30°C, Protected from direct light and humidity. Close the jar carefully after use.

## Warning

Due to the presence of eugenol, avoid using cement with resin-based fillings.

When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients. Do not use in case of allergy to Eugenol.

## Packaging

Bottle of 15 ml	<b>[REF]</b> 11420
Bottle of 30 ml	11421
Bottle of 100 ml	11422

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© PD 200302