# DE

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# EN

## Specific advices:

- only for dental use
- FIXOPHAT powder and liquid should be used only in combination (or with components formerly known as Kulzer Phosphate Cement powder and liquid)
- components must always be stored in firmly closed containers
- protect the powder from moisture
- protect the liquid from sun light
- storage between 2–28 °C
- do not use after expiration
- additional information and safety data sheets (MSDS) on [www.favodent.de](http://www.favodent.de)

## Important safety instructions:

Liquid contains phosphoric acid. It causes burns. In case of contact with eyes and skin, rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection. Only completely discharged containers are meant for recycling. Etchant residues may be disposed with the waste either in a highly diluted form with water or in neutralized form.

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FIXOPHAT slow Phosphate Cement normal setting    
 FIXOPHAT rapid Phosphate Cement quick setting  

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## Composition:

Powder: zinc oxide, magnesium oxide, calcium fluoride, feldspar  
 Liquid: orthophosphoric acid, zinc oxide, aluminum hydroxide

## Indications:

- permanent fixations of crowns, inlays, root posts and orthodontic bands
- cavity linings
- temporary fillings

## Handling and dosage:

In order to mix the powder and liquid correctly, a glass plate with a temperature of ideally 18–20°C (64–68 F) and a spatula (classically made of tantalum or agate) should be used. Ensure that spatula and glass plate are clean.

### 1. Dosage:

- FIXOPHAT slow: 1.20 g powder : 0.50 ml (0.888 g) liquid
- FIXOPHAT rapid: 1.20 g powder : 0.50 ml (0.858 g) liquid

### 2. Mixing:

In order to achieve a homogeneous consistency when mixing, the powder should be divided into several portions. Using the smallest amount first, the powder is added to the measured liquid in portions and mixed quickly. Residual liquid and contaminated liquid should be discarded.

Mixing time: 1 minute 30 seconds

### 3. Working time:

Setting-time:

- FIXOPHAT slow: 3 minutes 30 seconds (at 21°C)    ■ FIXOPHAT slow: 5.5-7.5 minutes (at 37°C)
- FIXOPHAT rapid: 3 minutes (at 21°C)            ■ FIXOPHAT rapid: 3.5-5.5 minutes (at 37°C)

All times are calculated from the end of mixing. The cement should be applied to the dry tooth surface. It should be protected from moisture. Close containers immediately after use.

## Specific demands:

- avoid direct application to dentin which is adjacent to the pulp or to the pulp itself
- pulp irritation may occur after cementation and if used to line deep cavities
- dentine areas near the pulp must be protected