FLEXISTONE® Plus

DE

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FLEXISTONE® Plus

Instructions for use

Insulating silicone material for use in dental technology, addition curing, low viscosity, high final hardness

Fig. 1

1. Dosage and mixing

Fill mixing bowl with base to the first marking, then with catalyst to the second one (Fig. 1). Spatulate **FLEXISTONE® Plus** homogeneously, until a uniform colour is achieved (Fig. 2).

Fig. 2

2. Application

Beginning from the interdental spaces, apply mixed **FLEXISTONE® Plus** uniformly with a spatula reaching a thickness of approximately 3 mm. Prior to setting of **FLEXISTONE® Plus** provide undercuts by means of a wax-knife (Fig. 3). Thus an optimum bonding to the plaster is ensured. No retention crystals are necessary.

3. Deflasking


After polymerisation deflask as usual and remove **FLEXISTONE® Plus**.

Fig. 3

Important working hints

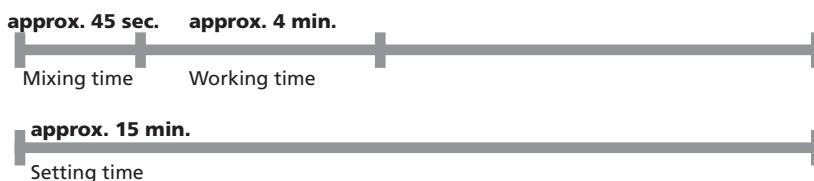- Avoid thin-tapered borders of **FLEXISTONE® Plus** – cutting of a small shoulder is recommended to prevent separation in the flask-counter.
- Increased temperatures accelerate, decreased temperatures retard the setting time.
- Containers should be tightly closed immediately after each use!
- Cured materials are chemically inert – spots on clothing should be avoided!
- **FLEXISTONE® Plus** is recommended for the IVOCLAR SR-Ivocap-System.
- Please follow the instructions of the safety data sheet!


Indications for use:

- Insulating material, A-silicone based, for the application with injecting technique and flask pressing technique.
- For the insulation and protection of acrylic or porcelain teeth during investing.
- Can be easily applied in one single procedure onto acrylic teeth and wax, forming a thin, very hard insulating layer for fixing of teeth during pressing.

Technical data:

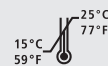- **Mixing ratio:** 1:1
- **Colour code:** **FLEXISTONE® Plus base:** clear blue **FLEXISTONE® Plus catalyst:** white
- **Mixing time:** approx. 45 sec.
- **Working time:** approx. 4 min.*
- **Setting time:** approx. 15 min.*
- **Final hardness:** 85 Shore A
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard the indicated times.

Ordering information:

FLEXISTONE® Plus
Standard packing 02383
 2 Plastic bottles, ea. 160 ml
 base + catalyst 1:1
 2 Stands
 1 Mixing bowl

Eco-packing 02329
 2 Plastic bottles, ea. 1000 ml
 base + catalyst 1:1
 4 Mixing bowls

DETAX

GmbH & Co. KG
 Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
 Telefon: 07243/510-0 · Fax: 07243/510-100
 www.detax.de · post@detax.de

Made in Germany

02/2016

FLEXISTONE® Plus

Two chemical systems available

recommended by
IVOCLAR for the
SR-Ivocap-System

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