GC Fuji TRIAGE™

RADIOPAQUE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Fissure protection.
2. Root surface protection.
3. Hypersensitivity prevention and control.
4. Protection for immature enamel.
5. Temporary filling of endodontic access.
6. Intermediary restorations.

CONTRAINDICATIONS

1. Pulp capping.
2. Avoid use of this product in patients with known glass ionomer cement allergies.

DIRECTIONS FOR USE

PINK Shade is command set, WHITE Shade is chemical set only.

Powder / Liquid Ratio (g / g)	1.8 / 1.0
Mixing Time (sec.)	20-25*
Working Time (min., sec.)	1' 40"
Net Setting Time (min., sec.)	2' 30"
Final Finishing Commencing Time	6'00"
Final Finishing Commencing Time if light cure (PINK Command set)	4'00"

Test conditions : Temperature (23±1°C), Relative humidity (50±5%)

A. FISSURE PROTECTION

ROOT SURFACE PROTECTION HYPERSENSITIVITY PREVENTION AND CONTROL PROTECTION FOR IMMATURE ENAMEL

1. Preparation of the tooth surfaces (e.g. for fissure protection or root surface protection) with NOT DESSICATE. Best results are obtained when prepared surfaces appear moist (glistening).

a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid aggravating the operculum.

Note: If extra retention is desired, application of CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) is recommended. The conditioner should be rinsed off thoroughly after application.

b) Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESSICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Powder and Liquid Dispensing

a) The standard powder to liquid ratio is 1.8g / 1.0g. (1 level scoop of powder to 1 drop of liquid).

b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.

c) Make sure the liquid nozzle is clean and dry before dispensing liquid. Turn the liquid bottle horizontal and hold in this position briefly to remove any bubbles.

d) Then invert and hold the liquid bottle vertically and squeeze gently to dispense a bubble free drop of liquid. After dispensing, wipe any residual liquid off the nozzle.

e) Close bottle caps tightly immediately after use.

3. Mixing

Dispense the required amounts of powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Spread the liquid over the pad (Fig. A-2) and mix the first portion with all the liquid for 10 seconds. Incorporate the remaining portion and mix the whole amount thoroughly for 15 seconds (total time within 25 seconds).

Working time is 1 minutes 40 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.

a) Take the mixed material using a suitable placement instrument or brush and apply to the tooth surface (Fig. A-3). Then use a bristly spreader to spread the material. Do not use a brush on the root surface or hypersensitive area or over the occlusal surface and into the pits and fissures.

Note: If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

b) After placement, when the material starts to lose the glossy appearance or after curing with the light curing device, apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge (Fig. A-4).

c) Finishing under air spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cured. Use a superfine diamond bur or a silicone finishing putty (Fig. A-5).

d) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig. A-6).

B. TEMPORARY FILLING OF ENDODONTIC ACCESS

1. Cleaning the root surface

a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

b) Fill the chamber with a cotton pellet (Fig. B-2).

2. Powder / Liquid Dispensing and Mixing

See directions in A (above), Section 2 and 3.

3. Placement

a) Using a suitable instrument, place the mixed material over the cotton pellet (Fig. B-3).

Note: If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

b) Moisture protection and Finishing

See directions in A, Section 4, b, c, and d.

C. INTERMEDIATE RESTORATIONS

1. Caries removal

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with CAVITY CONDITIONER for 10 seconds or DENTIN CONDITIONER for 20 seconds.

c) Rinse chamber with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1). DO NOT DESSICATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Powder / Liquid Dispensing and Mixing

See directions in A (above), Section 2 and 3.

3. Placement

a) Take the mixed material using a suitable placement instrument or brush and apply to the prepared lesion or tooth surface without incorporating any bubbles (Fig. C-2). Form the contour and if possible cover with a matrix.

Note: If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

b) See directions in A, Section 4, b, c, and d.

SHADE

Pink, White

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39-77.0°F).

PACKAGE

15 g powder, 10 g (8.0 mL) liquid, Dentin Conditioner 6 g (5.7 mL), powder scoop, mixing pad No.22, plastic spatula

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. DO NOT mix powder or liquid with any other glass ionomer components.

4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

5. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.georgepoc.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:


MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects-Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal equipment system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

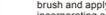MANUFACTURED BY: GC CORPORATION

70-1, Hoshinaka 2-chome, Hoshinaka-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY: GC CORPORATION

70-1, Hoshinaka 2-chome, Hoshinaka-ku, Tokyo 174-8585, Japan

EU: GC Europe NV

1800, Leuven, Belgium

8-2001 Leuven, Belgium

PRINTED IN JAPAN

Prior to use, carefully read the instructions for use.

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RECOMMENDED INDICATIONS

1. Occlusal bruxism.
2. Root surface protection.
3. Hypersensitivity prevention and control.
4. Protection on restorative enamel.
5. Temporary filling of endodontic access.
6. Intermediate restorations.

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DIRECTIONS FOR USE

PMK Shade is command seal. WHITE Shade is chemical seal only.

Powder / Liquid Ratio (g/g)	1.8 / 1.0
Mixing Time (min. sec)	20-25
Working Time (min. sec)	1'40"
Net Filling Time (min. sec)	2'30"
Final Finishing Composing Time	6'00"
Final Finishing Composing Time at Light cured (PMK Command seal)	4'00"
Test conditions	Temperature (23±1°C) Relative humidity (50±5%)

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REPOSICIONES INTERMEDIARIAS

1. Preparation of the tooth surfaces (e.g. for fissure protection or root surface protection)

After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Remove any residue of toothpaste.

Note: If a patient is allergic to fluoride, use a fluoride-free toothpaste. Rinse with water. Remove any residue of toothpaste.

When a restoration is desired, application of CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) is recommended. The conditioner should be rinsed off thoroughly after application.

a) Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESICCATÉ. Do not use a high speed air syringe on prepared surfaces; appear moist (glistening).

2. Powder application

a) The standard powder to liquid ratio is 1.8g / 1.0g. (1 level scoop of powder / 1 drop of liquid).

b) For accurate dosing of powder, lightly tap the bottle against the hand. Do not shake or invert.

c) Make sure the liquid nozzle is clean and dry before dispensing liquid. Turn the liquid bottle vertical and squeeze gently to dispense a bubble free drop of liquid. After the drop has formed, quickly withdraw the nozzle.

d) Do not shake cap tightly immediately after use.

3. Mixing

Dispense the required amounts of powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Spread the liquid over the pad (Fig. A-2) and mix the first portion with all the liquid for 10 seconds. Then mix the remaining portion and mix equal amounts thoroughly for 10-15 seconds (total time within 25 seconds).

4. Application

Apply the mixed material to tooth surface with a brush or apply to both tooth surface with a brush and apply to restorative area over the occlusal surface and into the pits and fissures.

a) Take the mixed material using a suitable placement instrument or brush and apply to both tooth surface (Fig. A-3). Use a brush in restorative area over the occlusal surface and into the pits and fissures.

b) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again.

2. Powder application

a) The standard powder to liquid ratio is 1.8g / 1.0g. (1 level scoop of powder / 1 drop of liquid).

b) For accurate dosing of powder, lightly tap the bottle against the hand. Do not shake or invert.

c) Make sure the liquid nozzle is clean and dry before dispensing liquid. Turn the liquid bottle vertical and squeeze gently to dispense a bubble free drop of liquid. After the drop has formed, quickly withdraw the nozzle.

d) Do not shake cap tightly immediately after use.

3. Mixing

Dispense the required amounts of powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Spread the liquid over the pad (Fig. A-2) and mix the first portion with all the liquid for 10 seconds. Then mix the remaining portion and mix equal amounts thoroughly for 10-15 seconds (total time within 25 seconds).

4. Application

Apply the mixed material to tooth surface with a brush or apply to both tooth surface with a brush and apply to restorative area over the occlusal surface and into the pits and fissures.

a) Take the mixed material using a suitable placement instrument or brush and apply to both tooth surface (Fig. A-3). Use a brush in restorative area over the occlusal surface and into the pits and fissures.

b) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again.

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6. Temporary filling of endodontic access

Use a fine-grained finishing instrument or brush and apply to both tooth surface with a brush and apply to restorative area over the occlusal surface and into the pits and fissures.

a) Use a fine-grained finishing instrument or brush and apply to both tooth surface with a brush and apply to restorative area over the occlusal surface and into the pits and fissures.

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7. Shade selection

Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

a) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

b) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

7. Odabir nijanse

Upotrijebite vodič za odabir nijanse da biste odabrali nijansu proizvoda za uporabu. Vodič za odabir nijanse treba koristiti u dobro osvijetljenoj prostoriji.

a) Upotrijebite vodič za odabir nijanse da biste odabrali nijansu proizvoda za uporabu. Vodič za odabir nijanse treba koristiti u dobro osvijetljenoj prostoriji.

b) Upotrijebite vodič za odabir nijanse da biste odabrali nijansu proizvoda za uporabu. Vodič za odabir nijanse treba koristiti u dobro osvijetljenoj prostoriji.

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Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

a) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

b) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

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Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

a) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

b) Use a shade guide to select the shade of the product to use. The shade guide should be used in a well-lit area.

8. Storage

Store in a cool, dry place. Do not store in a humid or hot environment.

a) Store in a cool, dry place. Do not store in a humid or hot environment.

b) Store in a cool, dry place. Do not store in a humid or hot environment.

8. Tárolás

Store in a cool, dry place. Do not store in a humid or hot environment.

a) Store in a cool, dry place. Do not store in a humid or hot environment.

b) Store in a cool, dry place. Do not store in a humid or hot environment.

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Store in a cool, dry place. Do not store in a humid or hot environment.

a) Store in a cool, dry place. Do not store in a humid or hot environment.

b) Store in a cool, dry place. Do not store in a humid or hot environment.