





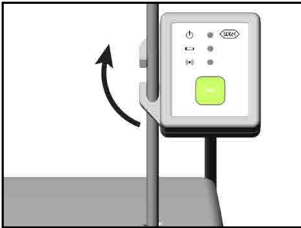
Quick guide for SPI dongle

Connecting and disconnecting the SPI dongle

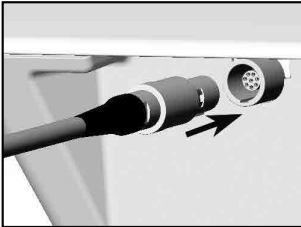


 Pay attention to the positioning!

❶ Plug in the SPI dongle or disconnect the SPI dongle from the control unit.



❷ Attach the SPI dongle to the irrigant support or remove the SPI dongle from the irrigant support.

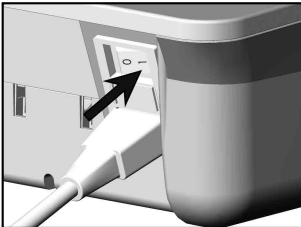

❸ Switch on the control unit at the power switch.

Quick guide for SPI dongle

Using multiple W&H control units simultaneously

Press and hold the orange button on the foot control S-NW for 3 seconds to switch between the various control units.

GREEN – SPI dongle activated

LED on if the SPI dongle is connected and the control unit is switched on.

ORANGE – battery

LED flashes if the batteries on the foot control need to be replaced.

BLUE – pairing



The foot control S-NW and the SPI dongle are paired in default status.

If pairing is active: LED indicator flashes

If pairing is inactive:

- 1 Press and hold the button on the SPI dongle for 4 seconds.
- 2 LED indicator flashes. SPI dongle is in pairing mode for 30 seconds.
- 3 Press and hold the green and orange buttons simultaneously on the S-NW foot control.
- 4 LED flashes three times when pairing is successful.

W&H Dentalwerk Bürmoos GmbH
Ignaz-Glaser-Straße 53, Postfach 1
5111 Bürmoos, Austria

t + 43 6274 6236-0
f + 43 6274 6236-55
office@wh.com wh.com

