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EN

Dear Customer

Thank you for choosing this high-quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaaurum.com.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General information

The Face Bow consists of an intraoral inner bow and an extraoral outer bow. Both bows are joined in the middle with laser welding.

3. Application

The Face Bow serves as an extraoral anchorage and/or the dental or skeletal correction of vertical and sagittal anomalies.

4. Safety precautions for the therapist

You have prescribed the patient a Face Bow and headgear for orthodontic treatment.

The Face Bow and headgear must be used according to your instructions.

Please advise your patient that improper use and removal of the Face Bow, without having loosened the elastic bands of the headgear, may result in serious injury if the Face Bow snaps back.

For safety reasons and for the protection of your patients only headgears with safety modules must be used!

The headgear, including the Face Bow must be fitted and handed out by the therapist only.

5. Composition

The composition is included in the materials list, refer to the catalog or www.dentaaurum.com.

6. Safety information

The product should not be used, if there is a known allergic reaction to one or more of the material components.

7. Information for single-use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

8. Instructions for disinfection

A disinfection prior to the first use is not necessary.

9. Instructions for disposal

There are no special precautions to be taken for the disposal of the Face Bows after treatment.

10. Delivery program

The Face Bows are available in the sizes mini, medium and maxi. For details please refer to the latest orthodontics catalog.

11. Additional information

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident, are informed accordingly.

12. With reference to our quality standards

Dentaaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

13. Explanation of symbols used on the label

 Please refer to the label. Additional information can be found at www.dentaaurum.com (Explanation of the symbols REF 989-313-00).

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Safety precautions for the patient / accident prevention

1. Face Bow and headgear must be used according to the therapist's instructions. When putting on headgear and Face Bow, strictly follow the instructions of the therapist.
2. Never put on or take off the Face Bow before loosening the elastic bands or the plastic chains.
3. Do not allow anyone to pull your headgear or your Face Bow. There is a serious risk of injury.
4. Never pull the Face Bow vertically, always remove from your mouth horizontally.
5. Never wear Face Bow and headgear during sports.
6. Should you have any questions regarding proper use of the Face Bow and headgear, please contact your therapist.
7. Improper use may result in serious injury caused by the Face Bow ends.
8. Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident, are informed accordingly.

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