SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

**EC REP**

SHOFU DENTAL GmbH  
An der Pönt 70, 40885 Ratingen, Germany

04.0:2020-09  
72404

**en**

READ CAREFULLY BEFORE USE

### PRODUCT DESCRIPTION

SHOFU GingiCurette Diamonds are designed to establish a chamfered finish to the tooth surface in crown and bridge preparations.

### PRECAUTIONS AND WARNINGS

●Never exceed the maximum permissible rotation speeds described in Table A. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Use intermittently with gentle touch and water spray. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

### STORAGE AND STERILIZATION

●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●For the details about REPROCESSING procedures, visit [www.shofu.de](http://www.shofu.de).

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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