# GUTTAFUSION®

## Oven

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**EN** Operating Manual

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**IT** Manuale d'uso

**ES** Manual de instrucciones

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**DA** Brugsanvisning

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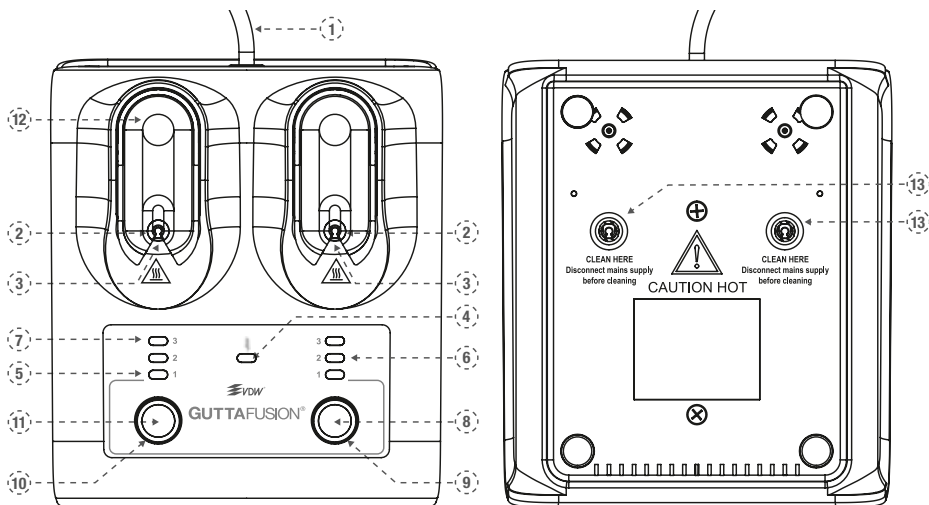
3) Heating chamber

4) Cleaning mode indicator light

5) Heat setting "1" for all sizes of GUTTA FUSION® Obturators and for other VDW obturator brands sizes 20-25

6) Heat setting "2" for other VDW obturator brands sizes 30-60

7) Heat setting "3" for other VDW obturator brands sizes > 60

8) Press button to select obturator size for right obturator heater

9) Right heater "ON" indicator light

10) Left heater "ON" indicator light

11) Press button to select obturator size for left obturator heater

12) Press here to raise the obturator holder.

When selecting the size of the obturator to be used press here to lower the obturator holder and start heating. Press here again to stop heating and raise the obturator holder

13) Clean heating chambers from here with supplied tool

**WARNING:** Disconnect main power cord before entering heating chambers

## **Congratulations on your purchase of the GUTTAFUSION® Oven.**

Endodontic products and services are the sole focus of VDW GmbH's business operations. Constantly remaining in close cooperation with international universities and endodontists enables us to develop innovative concepts, products and systems which make dentists' work simpler, easier and more effective.

By having purchased the GUTTAFUSION® Oven, you have acquired a product which has been developed and tested with the utmost care, meeting even the highest expectations in terms of functionality and operation.

VDW GmbH reserves the right to edit the information and data in this manual without prior notice.

The manual is available in additional languages upon request.

This manual has been created with the greatest care, but even with all our efforts, errors can never be entirely excluded. Suggestions for improvements are welcome anytime. Please contact VDW GmbH with any related comments or suggestions you may have.

## **Indications for use**

The GUTTAFUSION® Oven has been specially developed for heating GUTTAFUSION® Obturators for use in root canal therapy. Do not use this oven for any other purpose.

## **Contraindications**

There are none known.

## **Warning**

During heating process, surface gets hot.

Look for these  symbols.

## **Precautions**

1. To prevent fire or electric shock, do not expose this product to moisture.
2. Unplug the oven when not in use.
3. To reduce the risk of electric shock, do not remove cover; there are no user-serviceable parts inside.
4. This oven is designed for use in the dental office. Do not use in any other environment.
5. This oven is intended for heating guttapercha coated obturators only. Do not insert anything other than an obturator into the oven.

## Step-by-step instructions

1. Turn the power on by plugging the oven into a 115/230 VAC outlet. Both indicator lights around the buttons will light up. To wake-up the oven from stand-by mode, push and hold either one of the buttons until the indicator light around the button is illuminated and two signal “beeps” are heard, and release the button. On both sides the smallest size “1” is selected as shown by indicator lights. The oven is now ready for use. At any time the oven can be turned off. Push and hold either one of the bottom buttons until the indicator light around the button turns off.

2. Put both obturator holders in the upper position by pressing the back of the holder.

3. DISINFECT the obturator in a sodium hypochloride solution for one minute. RINSE the obturator in 70% alcohol. DRY the obturator for several seconds on a clean work surface to allow evaporation of excess alcohol.

4. After disinfecting the obturator and setting the silicone stopper, place the GUTTAFUSION® Obturator in one of the obturator holders, lightly press on top to have it firmly seated. Regardless of what working length you have determined, the silicone stopper must be under the holder. Make sure the obturator is hanging straight, not at an angle. The obturator must clear all sides of the heating chamber.

5. **NOTE:** Heat all GUTTAFUSION® Obturators on the heat setting “1”. When heating other obturators with the GUTTAFUSION® Oven, push the button which corresponds to the selected obturator holder. Select the size of the obturator you wish to heat, use the following guide for the recommended heat setting and repeatedly push the button to select the correct size.

Obturator brand	Obturator size	GUTTAFUSION® heat settings
GUTTAFUSION®	20-55	1
GUTTAFUSION® for RECIPROC® blue	R25 R40 R50	1 1 1
GUTTAFUSION® for RECIPROC®	R25 R40 R50	1 1 1
GuttaMaster®	20-25 30-60	1 2

6. To start the heating process push the obturator holder down until you hear it click. It will raise a little, but keep in a steady down position. The indicator light around the button will show that the heater is on (constantly on). The obturator is now being heated.

7. After the first signal “beep,” the obturator is ready for use. Push the back of the obturator holder to let it raise slowly. The indicator light around the button will go off. Take the obturator carefully out of the holder by pulling it towards you, making sure not to scrape the obturator on any part of the holder.

8. **NOTE:** You may leave the obturator in the GUTTAFUSION® oven for up to 90 seconds after the first signal “beep.” It will keep the obturator at the appropriate temperature and ready for use. The oven will “beep” every 15 seconds to remind you that the obturator is still in the oven. After 90 seconds, the heating element will switch off automatically.

9. If you want to heat more obturators, you may use one holder or both to continue your work efficiently. Each holder is operated by its own button and indicators. If you hear a signal “beep” then look at the indicator lights around the buttons which indicate the obturator heater is ready (flashing). When both heaters have finished their cycle and no buttons are pushed, the oven will go into a power-saving, stand-by mode after 5 minutes.

10. See table on page 13 for oven specifications.

The equipment is designed to be safe for indoor use under the following environmental conditions:	
Max. altitude	2 000 m
Storage temperature	-20 to +65 °C -4 to +149 °F
Storage humidity	20 to 80 %, no condensation
Operating temperature	+5 to + 40 °C +41 to + 104 °F
Max. operating humidity	80 % at +31 °C to 50 % at 40 °C (lin.) 80 % at +87 °F to 50 % at 122 °F (lin.)
Mains voltage fluctuations	Up to ±10% of the nominal voltage
Transient overvoltages	Category II
Rated pollution degree	2
Weight	0.56 kg
Measurement (W x D x H)	155 x 178 x 81 mm
If there is any doubt about the compatibility of decontamination or cleaning agents, please contact VDW GmbH.	

## Precision heating alerts

The oven is designed with two precision heating alerts:

### Heat setting “1” flashes:

The targeted temperature of the heating element was not reached within specified time (time-out). Use the other heating element to warm the ob-turator. This may be re-set by turning the power off and turning the power on. If the error occurs again the oven should no longer be used and sent for repair.

### Heat setting “2” flashes:

The hardware safety circuit has been activated due to a failure in the heating element, causing overheating. This may be re-set by turning power off, disconnect the power cord, reconnect the power cord, and turn the power on. If the error occurs again the oven should no longer be used and sent for repair.

## Cleaning instructions

Clean the oven after each use from the bottom side. Any gutta-percha remaining on the holder or in the heating chamber may be effectively removed with a generic dental solvent on a cotton pellet. Press and hold both buttons at once for 2 seconds to put the oven into the cleaning mode. This will cause both heaters to heat up to a medium temperature of 60 °C/140 °F. The indicator lights around the buttons will show that the heaters are on (constantly on) as well as the cleaning mode indicator light with the symbol .

A signal “beep” and flashing indicator lights around the buttons will warn that the desired temperature has been reached. Disconnect the main plug and clean the heater chambers with a wire brush, as directed on page 9.

**NOTE:** To abort the cleaning mode without disconnecting the main plug and to enter the normal mode again press either button for 2 seconds.

## Technical specifications

**Voltage Rating:** 115 Vac - 50/60 Hz/ 380 W, range of the supply voltage +/- 10%, power-save stand-by mode: < 1.5 W, max. Fuse Rating: 4 AT 250 V.

**Voltage Rating:** 230 Vac - 50 Hz/340 W, range of the supply voltage +/- 10%, power-save stand-by mode: < 1.5 W, max. Fuse Rating: 4 AT 250 V.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

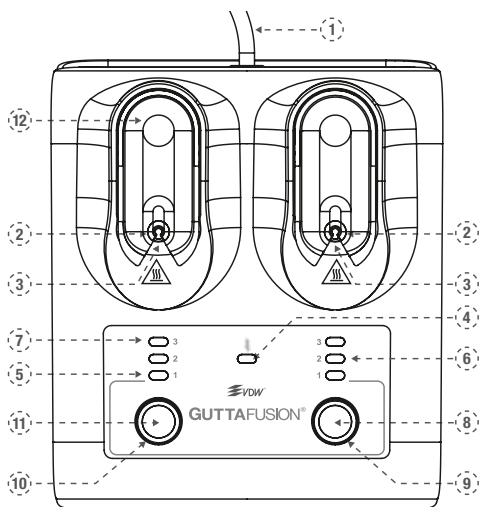
This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.


# Explanation of symbols used on packaging, device and in the operating manual

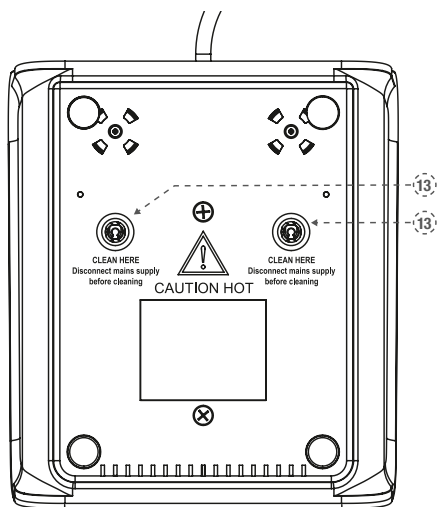	Consult instructions for use
	Warning! Failure to follow all instructions correctly during use can damage the product or endanger users/ patients.
	CE marking
	Class II product
	Alternating current
	Manufacturer
	Special disposal of waste electrical and electronic equipment (Directive 2002/96/EEC)
	Serial number
	Catalogue number (re-order number)
	Hot surface
	Fragile, handle with care
	Cleaning symbol
	Storage humidity

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**Manufacturer: VDW GmbH**