
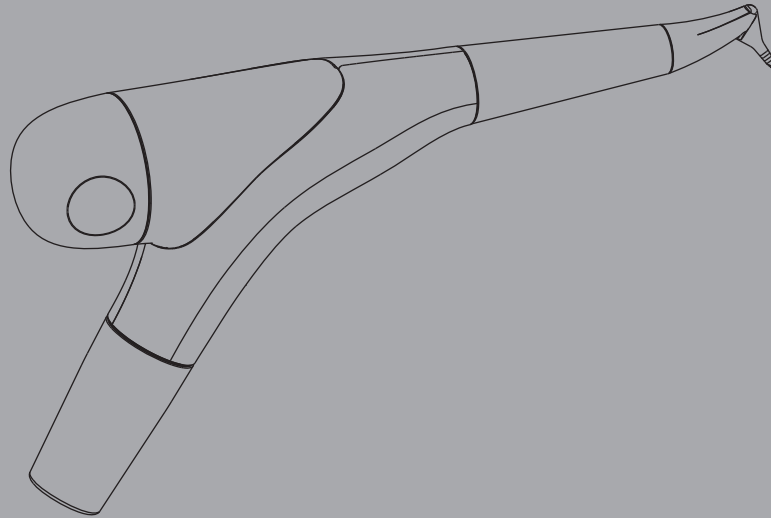
OPERATION INSTRUCTIONS  
MODE D'EMPLOI  
BEDIENUNGSANLEITUNG  
INSTRUCCIONES DE EMPLEO  
ISTRUZIONI PER L'USO  
BRUKSANVISNING

# AIR-FLOW<sup>®</sup> handy 3.0 PERIO

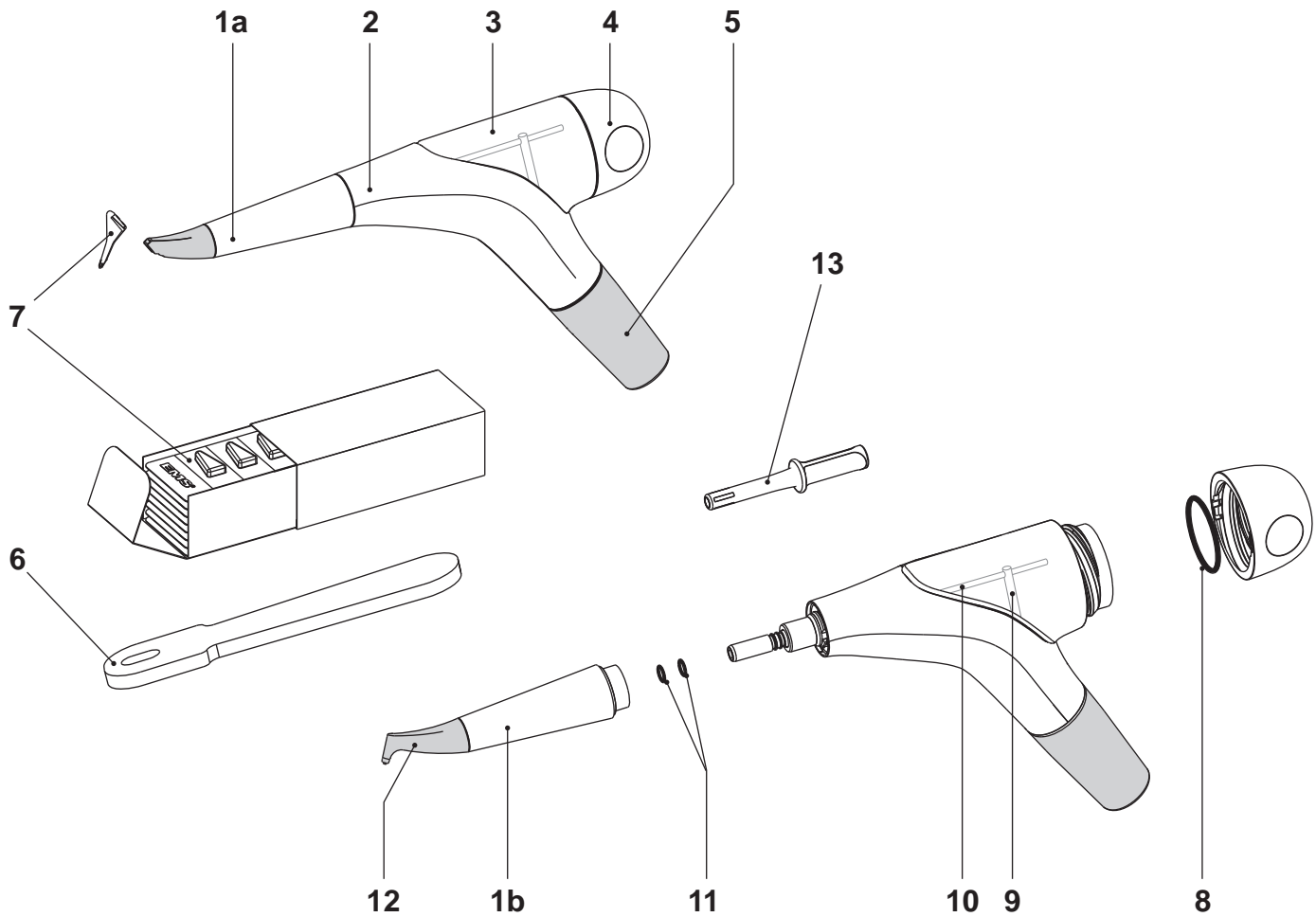**Rx ONLY** Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

**EMS**   
ELECTRO MEDICAL SYSTEMS

## COMPONENTS

- 1 a Handpiece AIR-FLOW PERIO  
b Handpiece AIR-FLOW PLUS
- 2 Body
- 3 Powder chamber
- 4 Powder chamber cap
- 5 Cord adapter
- 6 Nozzle remover
- 7 Nozzles
- 8 Cap ring
- 9 Air tube
- 10 Air/powder tube
- 11 O-rings for handpiece connection
- 12 Nozzle
- 13 EasyClean

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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## **DEAR CUSTOMERS,**

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

Adaptable on the turbine connection of the dental unit, the AIR-FLOW handy PERIO 3.0 is available in 2 different configurations.

The configuration with the AIR-FLOW PERIO handpiece allows the device to remove subgingival dental plaque from periodontal pockets deeper than 4 mm.

The configuration with the AIR-FLOW PLUS handpiece allows the device to both remove subgingival dental plaque from periodontal pockets up to 4 mm in depth and remove supragingival biofilm and surface stains.

This system is especially recommended for all patients suffering from periodontal diseases (periodontitis, peri-mucositis, peri-implantitis).

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## ABOUT THIS MANUAL

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

These operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all of the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close to hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

**Danger**  
Risk of injury

**Caution**  
Risk of property damage or environmental harm

**Please note**  
Useful additional information and hints

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### **Förbjudet**

### **Tillåtet**

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

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EMS

## **Technical modifications**

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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## CONTENTS

AIR-FLOW PERIO treatment.....	2
Indications .....	2
Contra-indications.....	2
AIR-FLOW PLUS treatment.....	6
Indications .....	6
Contra-indications.....	6
Installation and setting up.....	10
Water supply .....	10
Compressed air supply .....	10
Checking the turbine connection (Typical is shown).....	12
Preparing for treatment.....	14
Nozzle installation .....	14
Handpiece connection .....	16
Drying the sterilized handpiece .....	16
Connecting to the dental unit .....	18
Setting the water flow rate .....	20
Filling the powder chamber with EASY FILL.....	22
Closing the cap .....	24
Precautions .....	26
Wear a mask and eye protection .....	26
AIR-FLOW PERIO treatment.....	30
Power setting air/powder and water flow rate adjustment.....	30
Working Technique .....	34
End of the treatment.....	40
AIR-FLOW PLUS treatment.....	42
Working Technique .....	44
Working Technique in periodontal shallows pockets .....	48
Precautions to be taken at the end of the treatment .....	50

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Fluoride Application .....	52
Patient Information .....	54
Start the cleaning .....	56
Cleaning, disinfecting and sterilizing .....	58
Cleaning the handpiece .....	58
Stérilization of the handpiece .....	60
Cleaning the unit .....	62
Daily cleaning of the device .....	64
Daily cleaning of the cap .....	66
Maintenance.....	68
Replacing the cap and the O-rings.....	68
Annual maintenance .....	70
Safety precautions .....	72
Storing the product .....	76
Product disposal .....	78
Warranty.....	78
Accessories.....	78
EMS-Service.....	80
Symbols .....	82
Technical data .....	86
Troubleshooting .....	90

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# AIR-FLOW PERIO TREATMENT

## Indications

The AIR-FLOW handy 3.0 PERIO use with the handpiece AIR-FLOW PERIO and the nozzles is recommended for the following cleaning procedures :

- Maintenance of periodontal pockets deeper than 5 mm up to 10 mm by biofilm removal
- Implant maintenance

## Contra-indications

Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients (endocarditis, pregnancy, breast feeding, contagious disease, immune deficiency, neutropenia, agranulocytosis, diabetes, haemophilia), patients under treatment (radiotherapy, chemotherapy, antibiotics).

Do not use AIR-FLOW PERIO handpiece with nozzle immediately after subgingival cleaning (Ultra-sonic or hand instrumentation).

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 Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

 The use of any other powder than the EMS powders for subgingival application would reduce the nozzle's service life. As a result the treatment would become ineffective and would increase the risk of emphysema.

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 For more details about the EMS powder, please read the relative operation instructions

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# AIR-FLOW PLUS TREATMENT

## Indications

The AIR-FLOW handy 3.0 PERIO, use with the handpiece AIR-FLOW PLUS, is recommended for the following cleaning procedures :

- Removal of biofilm from pockets up to 4mm
- Removal of biofilm supragingival and colorations

## Contra-indications

In order to avoid any risk of emphysema, hypersensitivity, bleeding and patient uncomfot, the use of supragingival powders (EMS CLASSIC, CLASSIC COMFORT and SOFT) is strictly forbidden during subgingival treatments (contacts with soft tissues, dentine, cementum).

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Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

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Do not direct the jet of powder toward fillings, or crown and bridgework as this could damage the restorations.

Do not use the AIR-FLOW method on severely inflamed tissues.

Never use an abrasive powder i.e. silica or alumina based. This would damage the unit.

The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

For more details about the EMS powder, please read the relative operation instructions

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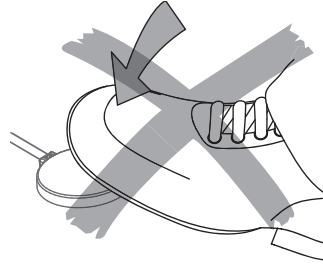
## Checking the turbine connection (Typical is shown)

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⚠ The unit is equipped with an adapter specially designed for the turbine connection of your dental unit. Only use the unit with this special turbine connection. To connect it to another type of turbine will damage it.

⚠ The turbine of your dental unit must not be under pressure when to connect the unit. Do not activate the turbine footpedal. If you have a turbine equipped with light, turn the light off.

⚠ Make sure the o-rings of your turbine connection are in good condition. A turbine connection with o-rings in bad condition can damage the unit.

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
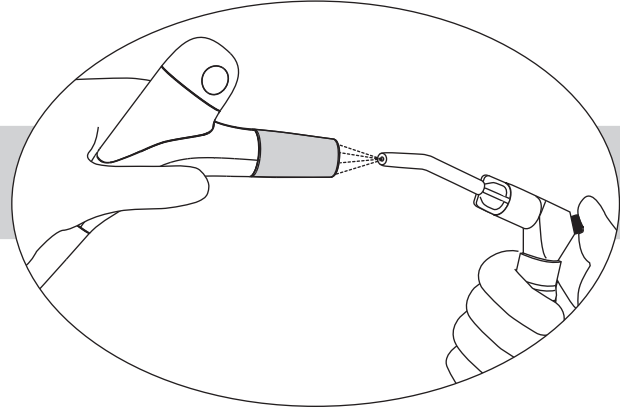
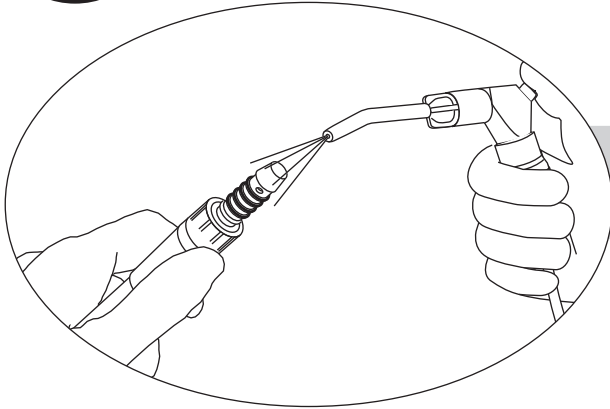
 The turbine connection and the connector must be absolutely dry. Moisture at the connection can cause blockage in the air/powder channels of the unit.

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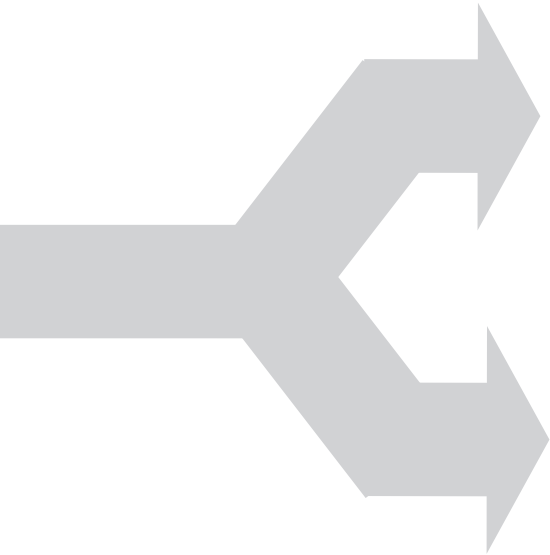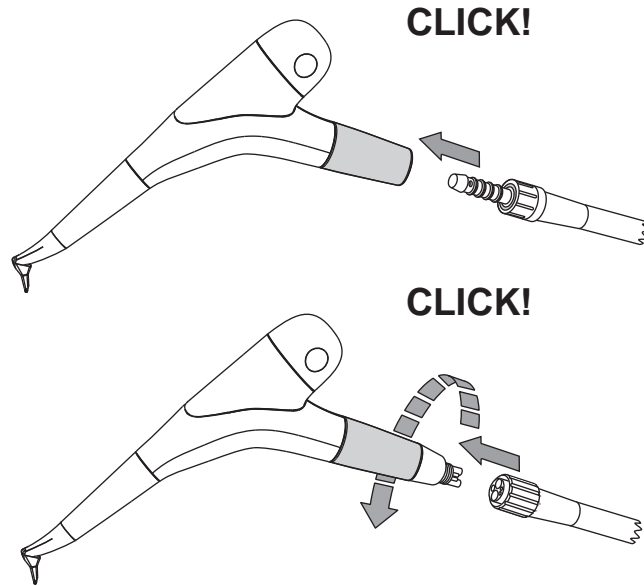
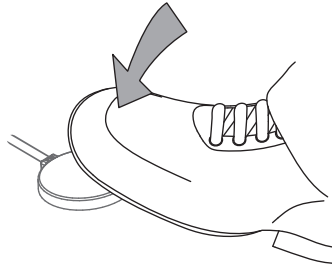
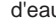
-  Never use the handpiece without water flow.
-  It is better to set the water flow rate before using the device for the first time while the powder chamber is empty.

Set the water flow rate from dental unit to obtain a small continuous spray.

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

## Filling the powder chamber with EASY FILL

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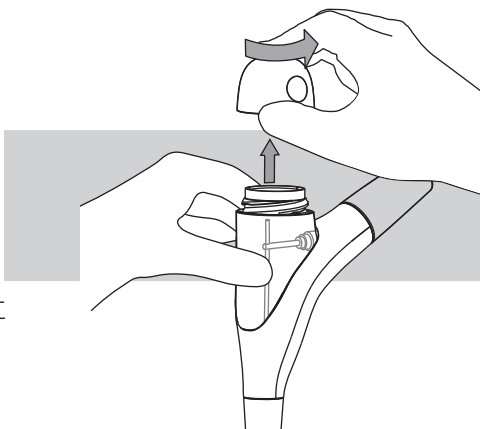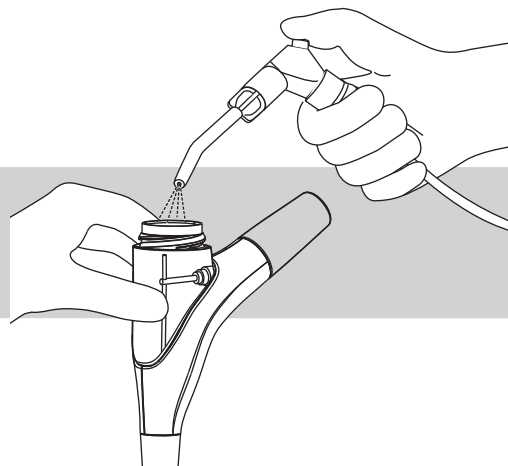 Before using this product with the handpiece, please carefully read and follow the recommendations of the instruction manual originally delivered with this product.

 Make sure that the powder chamber and the cap of the powder chamber are absolutely dry. Moisture can cause the powder to cake.

 Use only powder provided by EMS.

 Do not go over the maximum limit as indicated on the picture above. The openings of the tubes should not be covered with powder. The tubes could get clogged.

 Close the bottle properly to preserve the powder of the humidity.

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 A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.

 To limit the risk of bacterial or viral infections and inhalation of powder, we recommend to the dentist and the hygienist to wear gloves, protective glasses and a safety mask of protection.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

 The nozzle should never be pointed toward the patient before, during and after operation.

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## AIR-FLOW PERIO TREATMENT

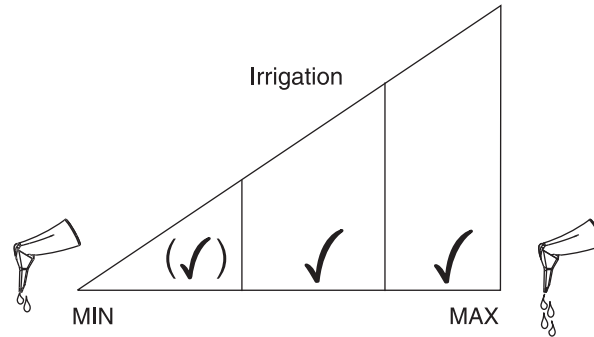
Power setting air/powder and water flow rate adjustment

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Never use the PERIO-FLOW handpiece without water flow; this could cause lesions in the treatment area.

- Start the treatment with water flow rate adjustment control set at medium setting. Then adjust the irrigation to obtain a flow as required for treatment.
- Start the treatment with the power setting air/powder control to the minimum setting. It can then be adjusted as required during the treatment.

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
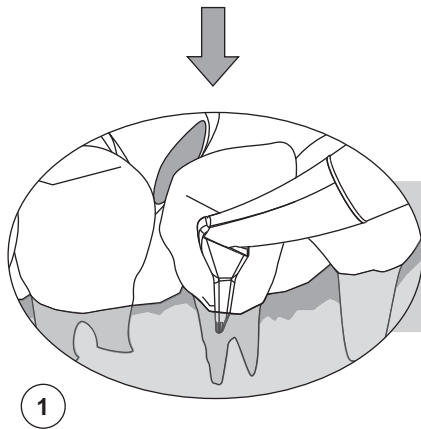
"ON"  
5 SEC

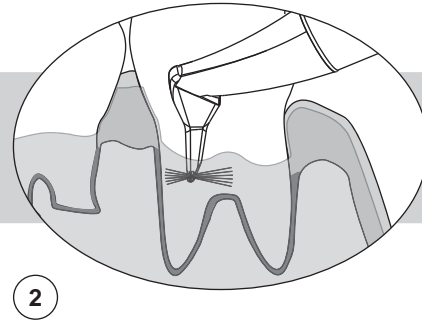1. Insert the nozzle into the periodontal pocket. The penetration depth depends on the pocket depth. Do not force the nozzle into the pocket because this could hurt soft tissues.

2. When the nozzle is inserted in the pocket, press the pedal of your device for 5 seconds.

3. Place the nozzle inside the pocket for 5 seconds maximum per site. Repeated insertion of the nozzle in the same site is not recommended. Carry on the treatment with short tiny vertical motions of the nozzle.

EMS powder/air mixture confined by a water jet is sprayed into the periodontal pocket. Due to the mechanical action of the air, powder and water spray, the subgingival biofilm is eliminated from the root surface.

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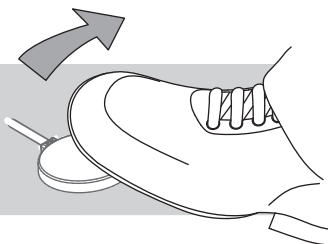
4

4. Release the pedal and wait until the spray stops.

 When removing the foot from the control pedal, the air/powder jet continues for a few more seconds. Take these seconds into account.

5. Remove the nozzle from the pocket and repeat the operation in the next pocket.

 Never spray outside periodontal pockets. External operation accelerates nozzle wear and causes a multidirectional spray jet which can be unpleasant for the patient.

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 The supragingival air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

 During treatment, make small circular motions.

Finally after the treatment polish all dental surfaces by setting the water flow rate to the maximum.

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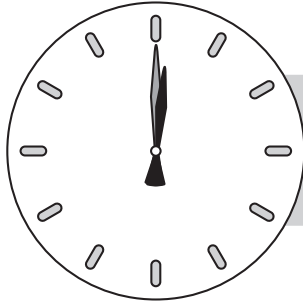
## Information au patient

## Patientenunterweisung

 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any more natural protection with respect to colouring.

 Tell your patient that during 2 to 3 hours following treatment, he should neither smoke, nor consume food or drinks which could strongly colour the teeth (tea, coffee...).

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## CLEANING, DISINFECTING AND STERILIZING

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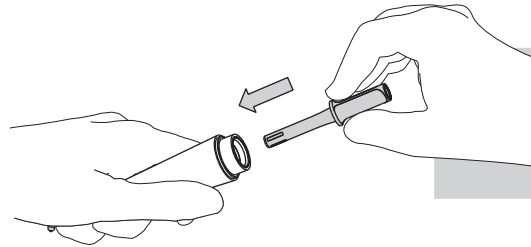
Reinigen des Handstücks


Clean the handpiece with the EasyClean supplied with the unit before disinfecting and sterilizing.

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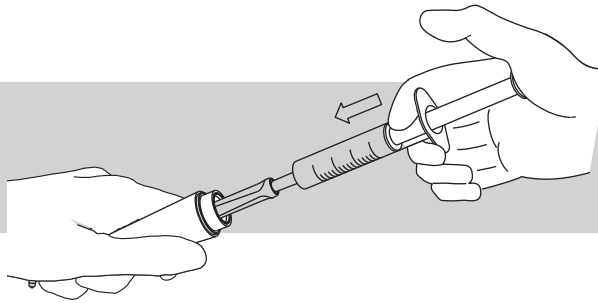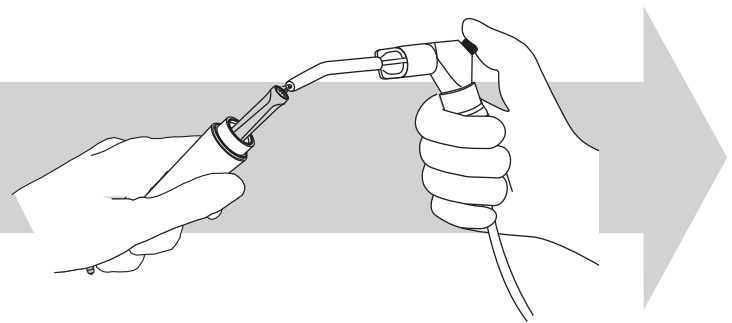
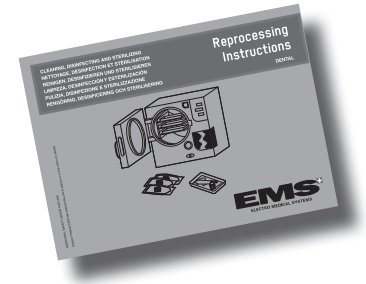
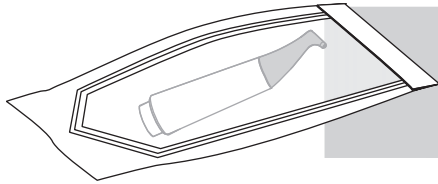
## Sterilization of the handpiece

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 Do not exceed the maximum number of sterilization cycles.

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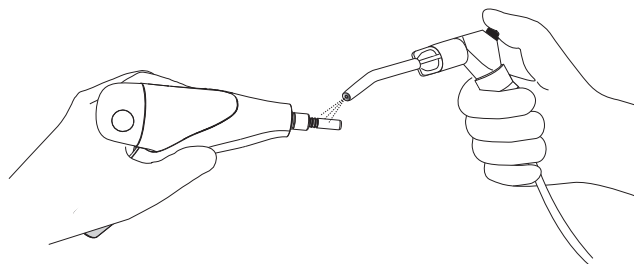
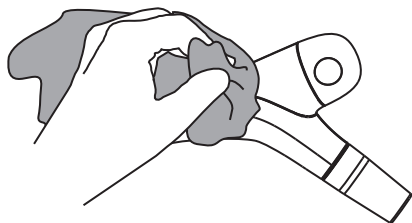
## Cleaning the unit

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 The unit is not protected against water spray.

 Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

 Do not submerge the unit in a disinfectant bath. It would be likely to be damaged.

Make sure the handpiece connection of the cord is dry

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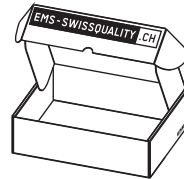
## Annual maintenance

## Maintenance annuelle

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To keep your device in a perfect operating and to extend its life, send your device each year for an annual maintenance to EMS or an approved EMS repair center.

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## SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operation instructions, or improper preparation and maintenance.

Use for the intended purpose only. Before using the product, make sure that the manual has been read. This also applies to any equipment used with this product. Failure to observe the operation instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

This product must be used only by trained and qualified personnel (dentist or dental hygienist).

Before each use, clean, disinfect and sterilize the different pieces and accessories of this product (nozzles excepted). Please refer to the information provided in the operation instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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 Use only the powder recommended by EMS for the subgingival application when using the AIR-FLOW handy PERIO. Any other powder could cause premature wear of the nozzle and would increase the risk of emphysema and damage to the unenameled dentine.

 The single use nozzle must be used for one single patient only. Never reuse a nozzle because treatment will be ineffective and the risk of emphysema will increase.

 In order to avoid the use of any inappropriate powders, it is highly recommended to always empty the powder chamber after use.

 The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use "EMS Abrasive Powder" in the unit as this would damage it.

 Always examine the product for damage before starting the treatment. Damaged accessories or a damaged device must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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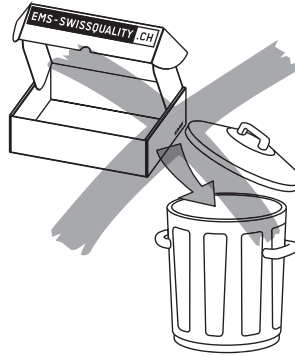
## STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to store the product for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing".
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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## PRODUCT DISPOSAL

The product shall not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

## WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

## ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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## EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

The connector must be dismantled or changed only by an approved EMS repair center.

You can ask for a modification of the type of adapter of the apparatus, in the range available near an approved EMS repair center only.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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## TECHNICAL DATA

### Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW handy 3.0 PERIO
Classification 93 / 42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection : Water	1 to 2.2 bar (1000-2200 hPa) with a service flow of 50-80 ml/min.
Service Pressure to the turbine connection : Air	Static pressure 2.7 to 3.5 bar (2700-3500 hPa)
Weight	Approx. 0.150 kg
Operating conditions	+10°C to +40°C 30% to 75% relative humidity 700 hPa to 1060 hPa air pressure Altitude max. 3000 m
Validated life time of the handpiece	800 sterilization cycles
Storage conditions	+5°C to +40°C 5% to 85% relative humidity 500 hPa to 1060 hPa air pressure
Transport conditions	-10°C to +40°C (ventilated) or 60°C (non ventilated) 75% relative humidity min. 700 hPa air pressure

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# TROUBLESHOOTING

Symptom	Solutions
Water comes into the powder chamber or leaks from the cap	<ul style="list-style-type: none"><li>• Check the dental unit connection</li><li>• Check the state of the o-rings of the turbine connection</li><li>• Connect the unit to the dental unit (be careful, the powder chamber must be empty)</li><li>• Dry and fill the powder chamber</li><li>• Send the unit to an approved EMS repair center</li></ul>
No powder/air jet coming from the unit	<ul style="list-style-type: none"><li>• Cut immediately the air input by releasing the foot pedal</li><li>• Wait for 1-2 minutes for the system to decompress</li><li>• Disconnect the unit from the dental unit</li><li>• Press the foot control pedal of the dental unit</li><li>• If no air comes out of the turbine connector, it is your dental unit that causes the problem</li><li>• If air comes out, it is the device that causes the problem</li><li>• Unscrew the cap above the wash-hand basin, because the powder can be ejected. A sealed apparatus, even disconnected, can indeed remain under pressure</li><li>• Empty the powder chamber and screw the cap back on</li><li>• Connect the unit to the dental unit (be careful, the powder chamber must be empty)</li><li>• Connect the handpiece to the unit</li><li>• Press the foot control pedal of the dental unit</li><li>• If air comes through the handpiece connector, the handpiece is blocked. Clean the handpiece</li><li>• If no air comes out of the handpiece connector, the unit is blocked</li><li>• Send the unit to an approved EMS repair center</li></ul>
No water spray is coming from the nozzle	<ul style="list-style-type: none"><li>• Change the nozzle</li><li>• Remove the nozzle and verify that a water jet comes out from the handpiece</li><li>• Send the device to a certified EMS Repair Center</li></ul>
Water leakage between the handpiece and the nozzle (unusual leak of droplets)	<ul style="list-style-type: none"><li>• Check the position of the nozzle and its position on the handpiece</li><li>• Change the nozzle</li><li>• Check for residues between the nozzle and the handpiece</li><li>• Check that nothing affects the alignment of the nozzle</li><li>• Send the device to a certified EMS Repair Center</li></ul>
Water leakage between the nose connection and handpiece	<ul style="list-style-type: none"><li>• Check the O-rings on the nose connection of the device, replace them if necessary</li><li>• Change the handpiece</li><li>• Send the device to a certified EMS Repair Center</li></ul>
Air and/or powder is leaking from the threads of the cap	<ul style="list-style-type: none"><li>• Check the seal and the cleanliness of the thread on the powder chamber and on the cap</li><li>• Replace the seal or the cap if necessary</li><li>• Send the device to a certified EMS Repair Center</li></ul>

---

The unit efficiency diminishes

- Check the amount of powder in your device
  - Check the air pressure of the dental unit
  - Change the nozzle
  - Remove the residual powder in the channels of the handpiece with the EasyClean
  - Send the unit to an approved EMS repair center
- 

If the handpiece falls

AIR-FLOW PERIO handpiece

- Replace the nozzle and check the fit of the nozzle
- If the nozzle doesn't hold or it is not watertight, send the handpiece to a certified EMS Repair Center

AIR-FLOW PLUS Handpiece

- Check the state of the nozzle
  - Replace the handpiece
- 

If there is leakage of powder through the nozzle

- Fill with the Easyfill
  - Reduce the volume of powder in the powder chamber
-

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EMS Italia S.r.l.
EMS Corporation