# Honigum Pro

Light

## Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	12
Mode d'emploi – FRANÇAIS	20
Istruzioni per l'uso – ITALIANO	28
Instrucciones de uso – ESPAÑOL	36
Instruções de utilização – PORTUGUÊS	44
Gebruiksaanwijzing – NEDERLANDS	52
Brugsanvisning – DANSK	60
Bruksanvisning – SVENSKA	67
Instrukcja użycia – POLSKI	74
Инструкция по применению – РУССКИЙ	82

## Produktbeschreibung

---

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Vinylpolysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

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## Product description

---

Honigum Pro/Light is a correction impression material based on addition curing silicones.

The material is available in the Automix safety cartridge.

## Indications

---

- Crown and bridge impressions
- Inlay and onlay impressions

## Basic safety instructions

---

- For dental use only!
- Keep away from children!

## Advice on application

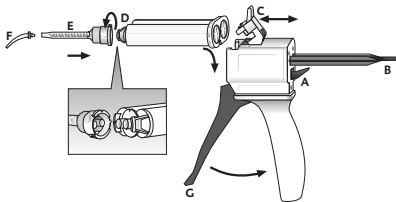
---

- The working times (refer to “Technical data” section on page 16) apply at a room tem-

perature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

- Rinse off residues of retraction solutions thoroughly with water because they can increase the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.
- Do not mix Honigum Pro/Light with condensation curing silicone or polyether impression materials.

## Using the Automix cartridge

### Insert cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it by pushing down the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the dispensing gun.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

### Apply material

- ▶ When using a cartridge for the first time, express a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [G] of the dispenser and it is then applied directly.

**Note:** Leave the used mixing tip on the cartridge after use. It serves as a cap.

## Remove cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

## Recommended use

---

- In order to improve the adhesion of the pre-impresion material to metal or plastic trays, a tray adhesive for A-silicon (e.g. Tray Adhesive, DMG) should be applied.
- The intraoral tips are to be used for direct, intrasulcular application around the preparation.

## Double-mix impressions

- ▶ For the individual times applicable to the materials used refer to the “Technical data” section. Follow the associated instructions for use!

- ▶ Fill the tray with Honigum Pro/Putty, Heavy or Mono. When doing so, hold the impresion tray at a slight angle beneath the mixing tip.

**Note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impresion material throughout the process.

- ▶ Syringe the preparation with Honigum Pro/Light.

**Note:** The filling of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and allow the impresion to form.
- ▶ Remove the impresion from the mouth once the setting time has ended, refer to “Removal” section on page 15.

## Correction impression

- ▶ For the individual times applicable to the materials used refer to the “Technical data” section. Follow the associated instructions for use!

**Note:** No more than 30 minutes are permitted to pass between the pre-impression and correction impression, otherwise it is not possible to guarantee a firm bond between the two impression materials. Cleaning with alcohol is also not permitted between pre-impression and correction impression.

- ▶ Cut generous drainage channels into the pre-impression for the wash and remove imperfections.
- ▶ After experimental repositioning, thoroughly rinse the impression with water and then dry.
- ▶ Apply Honigum Pro/Light around the preparation and on the pre-impression.

- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and exert brief initial pressure.
- ▶ Remove the impression from the mouth once the setting time has ended, refer to “Removal” section on page 15.

## Removal

---

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ After removal, proceed as described in the section “Cleaning and disinfecting the final impression” section on page 15.

## Cleaning and disinfecting the final impression

---

- ▶ Only disinfect the final impression with dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.

- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- ▶ Follow the instructions of the disinfectant solution's manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting, refer to "Storage of the impression" section on page 16.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; [www.dgzmk.de](http://www.dgzmk.de)
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; [www.ada.org](http://www.ada.org)

## Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for

the forms (ISO 6873, Type 4; e.g. Fujirock II, GC). The impression can be copper or silver-plated.

## Storage of the impression

- ▶ Store the impression in a dry place at room temperature (15–25 °C/59–77 °F)!
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

## Technical data

Honigum Pro/Light		 Fast
Working time <sup>1</sup> [min]	≤ 2:45	≤ 1:30
Intraoral working time <sup>2</sup> [min]	≤ 1:00	≤ 0:30
Time in the mouth [min]	≥ 3:15	≥ 2:00

<b>Honigum Pro/Light</b>		 Fast
Dimensional change after 24h	≤ 0.35	≤ 0.35
Compression set [%]	≤ 0.25	≤ 0.25
Strain in compression [%]	≈ 4.0	≈ 4.0

<sup>1</sup> The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

<sup>2</sup> The times apply from start of mixing time for a temperature of 35 °C.

## Disinfection of the Automix-Dispenser

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG rec-

ommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser can also be autoclaved.

## Composition

Vinyl polysiloxanes, silicium dioxide, pigments, additives, platinum catalyst

## Classification

ISO 4823:2000, type 3: Light-bodied consistency

## Storage and shelf-life

- ▶ Store in a dry place at room temperature (15–25 °C/59–77 °F)!
- ▶ Do not store in the refrigerator or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ Leave the used mixing tip on the cartridge after use. It serves as a cap.
- ▶ Do not use after the expiration date!

## Available in the following forms

### Not available in the USA

4 cartridges with 25 ml paste, 10 Automix-Tips, 10 Intraoral-Tips	
Honigum Pro/Light	
	REF 929539
 Fast	REF 929639
16 cartridges with 25 ml paste, 40 Automix-Tips, 40 Intraoral-Tips	
Honigum Pro/Light	
	REF 929690
 Fast	REF 929691

2 cartridges with 50 ml paste, 10 Automix-Tips, 10 Intraoral-Tips	
Honigum Pro/Light	
	REF 929831
 Fast	REF 929832
8 cartridges with 50 ml paste, 40 Automix-Tips, 40 Intraoral-Tips	
Honigum Pro/Light	
	REF 929835
 Fast	REF 929836
8 cartridges with 50 ml paste	
Honigum Pro/Light	
	REF 929892

## Only available in the USA

4 cartridges with 25 ml paste, 10 Automix-Tips, 10 Intraoral-Tips	
Honigum Pro/Light	
	REF 989539
 Quad Fast	REF 989639

**Caution:** US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH