HydroPrint Premium regular set

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Instrucciones For Use **EN**

Definition
HydroPrint Premium regular set is a high precision, alginate based impression material for use in dentistry.

Material type
Alginate impression material

Indications
HydroPrint Premium regular set is indicated for partial and complete impressions and features excellent stability, elasticity and compatibility to dental stones.

Side effects/Interactions
There are no restrictions when this product is used by a certified professional according to the instructions for use. Avoid using the alginate on any patient with an allergic history to one or more components of the product, or when stipulated technique can not applied

Precautions

- For dental use only.
- Keep out of reach of children.
- Contains silica. The powder inhaling might be harmful to the health.
- Contact with eyes, skin and hands can cause irritation. It's recommended to wear protective equipment (goggles, gloves and mask).
- Do not reuse this product.
- Take the impression with the patient sitting, not lying down.

Composition
Potassium alginate, calcium sulfate, diatomite, magnesium oxide, Tetrasodi-um pyrophosphate, potassium fluoroantate, PEG, pigment and flavour.

Instruccions for Use

- Transfer the content of this pack to a closed container.
- Shake the container several times to mix the contents thoroughly.
- Always pour the powder into the water, never reverse. For each powder measure, use 1/3 of a water vial. If the water temperature is lower than 23 °C/73 °F, the total setting time will increase. If the water temperature is higher, the setting time will decrease. High ionic content of the mixing water may affect the setting time.
- For a partial impression 1 scoop = 1/3 beaker of water
- For a full impression 2 scoops + 2/3 beaker of water
- For an extra-large impression 3 scoops + full beaker of water

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HydroPrint Premium Regular Set

Mischzeit:

sealed container with a wet gauze or cotton roll. Do not immerse the impression in water.

Shelf life and storage
HydroPrint Premium regular set fulfils the intended purpose at least until the expiry date, which is displayed on the containers. It should be stored in well sealed packs at temperatures 15 – 30°C / 59 – 86°F, in a dry place, protected from heat. Impressions should be stored at normal room temperature, avoid exposure to heat and sun.

Close container tightly after use. Humidity can adversely affect the properties of the product.

Caution
Federal law restricts this device to sale by or on the part of a dentist.

Marking
The expiry date and ☞ number are shown on the package.

Note
Only supplied to dentists and dental laboratories or upon their instructions.

Technical data ISO 21563:2013
Measurements are done at 23 °C / 73 °F room temperature and 50 % relative humidity.

Depending on the density the approximate powder/water ratio (g/ml) is 9g/19ml.

HydroPrint Premium regular set	
Mixing time:	0:45 min
regular set	
Working time:	1:10 min
Time in mouth:	
regular set	1:35 min
Total Setting time:	
regular set	2:45 min

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