Intensiv HygienicTray HT3000

IFU-2017/07/2017

DE

Produktbeschreibung

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GB

Product description

- Fully dismantable in 2 parts
- Suitable for the standard sterilisation tray
- Ideal arrangement of the 36 FG and 18 RA perforations for easy access to the instruments
- Stainless steel
- Dimensions:
H x L x W: 30 x 150 x 43 mm

Indications

- Organization of rotary instruments for daily use in therapy protocols
- Bearing of instruments during sterilization process
- Storage of rotary instruments

Practical recommendations

- To open the tray, pull up the cover while holding the base.
- When opening the tray, keep in a horizontal position to avoid the instruments falling out.
- Close the tray by pressing down the cover until a "click" is heard.
- If the cover is too loose, remove it and press it lightly on the ends, in order to reach sufficient pressure.
- To avoid any cross contamination, use disinfected tweezers to take the instruments.

Maintenance and sterilisation

- To facilitate cleaning, dismantle the Intensiv HygienicTray completely (cover, instrument tray).
- The HygienicTray is supplied non-sterile. Prior to first use and immediately after each use, the HygienicTray must be disinfected, cleaned and sterilized.
- During the sterilisation keep the HygienicTray separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

Intensiv HygienicTray
Ref. HT3000

Additional models
Intensiv HygienicTray
Ref. HT100
Ref. HT200
Ref. HT300

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Intensiv HygienicTray
Ref. HT3000

Modelli supplementari
Intensiv HygienicTray
Ref. HT100
Ref. HT200
Ref. HT300

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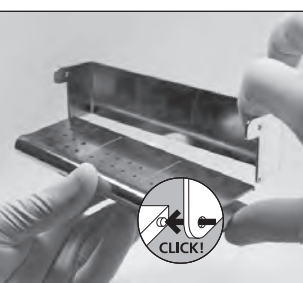
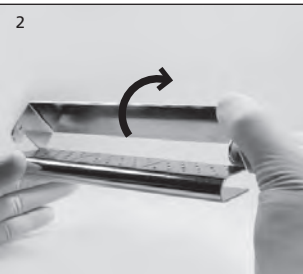
Intensiv HygienicTray
Ref. HT3000

Modelos adicionales
Intensiv HygienicTray
Ref. HT100
Ref. HT200
Ref. HT300

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GB

Product description

- Fully dismantable in 2 parts
- Suitable for the standard sterilisation tray
- Ideal arrangement of the 13 FG and 5 RA perforations for easy access to the instruments
- Stainless steel
- Dimensions
H x L x W: 30 x 50 x 40 mm

Indications

- HygienicTray may be used as an organizer for a specific indication treatment, i.e. prep bur block. In addition, it enables an easier handling of the instruments during the treatment and provides a support during the sterilisation process.

Practical recommendations

- To avoid any cross contamination, use disinfected tweezers to take the instruments.

Maintenance and sterilisation

- To facilitate cleaning, dismantle the Intensiv HygienicTray completely (cover, instrument tray).
- The HygienicTray is supplied nonsterile. Prior to first use and immediately after each use, the HygienicTray must be disinfected, cleaned and sterilized.
- During the sterilisation keep the HygienicTray separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

Intensiv HygienicTray
Ref. HT300

Additional models
Intensiv HygienicTray
Ref. HT100
Ref. HT200
Ref. HT3000

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