**Estimado cliente**

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## EN

### Dear customer

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at [www.dentaurum.com](http://www.dentaurum.com), even though you frequently use the same product.

#### 1. Manufacturer

Dentaurum GmbH & Co. KG  
Turnstr. 31 | 75228 Ispringen | Germany

#### 2. General product description/area of application

The hyrax® screw (Hygienic Rapid Expansion Screw) is a special screw for rapid palatal expansion. The retention legs of the palatal expansion screw are welded onto preformed metal bands or cap splints. Alternatively, they can be polymerized into acrylic splints. The bands or splints are cemented in the usual way.

#### 3. Area of application

Used for rapid palatal expansion with a fixed appliance.

#### 4. Fabricating a hyrax® appliance using the example of preformed bands

Preformed bands are fitted on the maxillary left and right 1st bicuspids and left and right 1st molars. These bands are transferred to a dental stone model using an impression or fitted to a dental stone model. The legs of the hyrax® screw are then bent to fit the bands so that they can be welded into place (Fig. 1).

To prevent the legs from breaking during bending, we recommend using our universal bending tool (REF 611-122-00, Fig. 4). If one is not available, hold the legs against the body of the expansion screw, using strong flat pliers (Fig. 5).

The hyrax® screw should „float“ freely at least 1-2 mm above the palate roof/gingiva (Fig. 2). Its posterior side should form a straight line with the mesio palatal cusps of the 1st molars. To avoid any leverage effect, it should be ensured that the occlusal level is parallel.

#### 5. Information on use of hyrax® screws

To increase strength, the bands can be joined on the buccal and/or lingual side with a piece of 1.0 mm remaloy® wire (REF 528-100-00). Laser weld these wires above the legs. The palatal strengthening wire should project slightly beyond the bands and be supported by the adjacent teeth to allow better distribution of expansion forces.

Easy, problem-free bending of the retention legs of hyrax® screws with the universal bending tool (REF 611-122-00), without the use of additional pliers (leverage principle).

To prevent the legs from breaking during bending, they should be held against the body of the expansion screw using strong flat pliers e.g. Maxi wire-bending pliers (REF 013-522-00).

When using the soldering technique please protect the body of the screw against excessive heat, e.g. with heat screen (REF 380-070-00).

#### 6. Information about the composition

Information about the composition is included in the material list, see catalog or [www.dentaurum.com](http://www.dentaurum.com).

#### 7. Safety instructions

Always use the blue safety key supplied to prevent swallowing!  
The product should not be used, if there is a known allergic reaction to one or more of the material components.

#### 8. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

#### 9. Availability

Description	straight legs	bent legs	Quantity
hyrax® Mini - 7 (Expansion 7 mm)	602-800-10	602-801-10	1 piece
	602-800-30	602-801-30	10 pieces
	602-800-50	602-801-50	50 pieces
hyrax® Medium - 10 (Expansion 10 mm)	602-808-10	602-840-10	1 piece
	602-808-30	602-840-30	10 pieces
	602-808-50	602-840-50	50 pieces
hyrax® Maxi - 12 (Expansion 12 mm)	602-811-10	602-812-10	1 piece
	602-811-30	602-812-30	10 pieces
	602-811-50	602-812-50	50 pieces

Packs (1 piece / 50 pieces) with one safety key  
Pack (10 pieces) with 10 safety keys

Safety key	611-120-01	10 piece
Universal bending key	611-122-00	1 piece

#### 10. Quality information

Dentaurum guarantees the perfect quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

#### 11. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

The product should not be used, if there is a known allergic reaction to one or more of the material components.  
Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

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#### Information for the doctor / patient

Before the hyrax® appliance is cemented into position, turn the spindle (screw) slightly 2-3 quarter turns in the direction of the arrow to test its function. Then turn back to the original position.

Turn the hyrax® screw only in the direction of the arrow. Opening and closing the screw several times during the treatment can cause it to become too loose and to lose its function.

The finished hyrax® appliance is cemented into position in the usual manner.

Following this, the screw is turned four times by 90° using the key supplied for this purpose. One full 360° turn of the spindle = 0.8 mm expansion.

The patient should turn the screw two times 90° both in the morning and evening (according to your instructions) until the desired expansion is achieved.

**Note:** Use only the blue safety key supplied, as this cannot accidentally be swallowed.

The safety key is designed to hold a maximum force of 35 N ± 3.5 kg. Please note that the force exerted on the safety key wire should always be exerted on the closed side of the wire clamp so that the clamp cannot loosen from the axis. Improper use may lead to overload and consequently to failure thus potentially excluding any guarantee and liability. Regular visits are recommended to monitor progress.

The hyrax® appliance is secured against unwanted reversing, after the desired degree of expansion is achieved, by fitting a ligature or blocking with acrylic. For stabilization the appliance should remain in the mouth for approximately 2-3 months.

A retainer should be worn following removal of the hyrax® appliance.

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