**SPECIAL AREAS**

# GREEN&CLEAN BR

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**METASYS**

# GREEN&CLEAN BR

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**EN** Instructions

**FR** Instructions

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**SV** Instruktioner

**FI** Ohjeet

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**DA** Instruktioner

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**CS** Instrukce

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**SL** Navodila

**HR** Instrukcije

**HU** Utasítás

**ET** Juhendid

**LV** Instrukcijas

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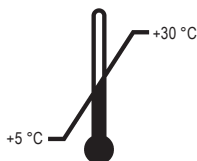
**IS** Leiðbeiningar

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## CE 0044

UFI: 9C00-D050-800R-GA8G

Produktidentifikatoren / product identifiers:

Hydrogen peroxide [7722-84-1]

# Artikelnummern GREEN&CLEAN BR

## Order numbers GREEN&CLEAN BR

---


### Artikelnummern | Order numbers

**REF** 60040300

Biofilm-Reinigungslösung, 2 x 1000 ml Flaschen  
Biofilm removing solution, 2 x 1000 ml bottles

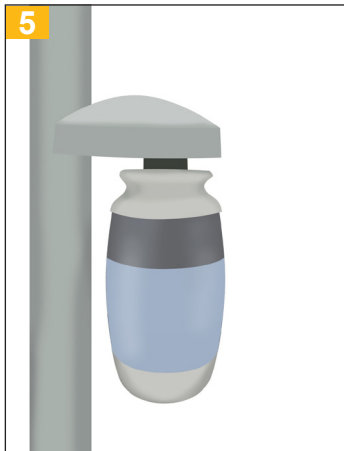---

**Anwendung in der Bottle**

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## **EN GREEN&CLEAN BR**

### **Application using a water bottle system**

4% solution based on hydrogen peroxide for the removal of biofilm in dental units with bottle systems.

In combination with the METASYS BR Biofilm Removing System, GREEN&CLEAN BR is recommended for the refurbishment of dental units as well as before the installation of a water decontamination system.

Attention: Biofilm removal (intensive decontamination) may only be carried out by trained technicians!

Spectrum of efficacy: For the effective cleaning and disinfection of water-carrying pipes as well as for the removal of biofilm from dental units.

Safety recommendation: Avoid contact with skin and eyes. Keep away from children. Do not swallow!

#### Application:

Put on protective gloves and goggles before use!

Remove all water consumers (e.g. turbines, handpieces, couplings; OUTSIDE air/water spray) from the dental unit (protection from clogging).

**1** Fill the empty water bottle of the treatment unit with GREEN&CLEAN BR.

**IMPORTANT:** Before performing the biofilm removal procedure, check whether the dentist element (e.g. instruments) is a stand-alone system, or if it is connected to the assistant and patient elements (e.g. mouth rinse, saliva ejector) of the dental chair: For a stand-alone system use 250 ml of GREEN&CLEAN BR; if the chair elements are connected, use min. 1000 ml of GREEN&CLEAN BR.

**2** Connect the water bottle to the treatment unit and flood the water pipes.

**3** Open all water consumer outlets and wait for the violet liquid to escape. For treatment units without spittoon bowl: Use a suitable receptacle (e.g. bucket or tub) to collect the escaping liquid!

**4** As soon as violet liquid escapes, close all outlets and valves and observe the action time (30 min).

**5** When the action time has elapsed, fill the emptied and washed-out water bottle of the dental unit with clean water, connect it to the dental unit and rinse the water bottle and pipes completely.

**6** If necessary, repeat the rinsing process until all of the GREEN&CLEAN BR has been rinsed out. For intensive decontamination of the bottle itself, fill the water bottle up to the bottleneck with GREEN&CLEAN BR. Seal the water bottle and swirl liquid in it to completely wet the surfaces. The action time is 30 min. Then empty the bottle and rinse with clean water. If necessary, repeat this step several times until the entire GREEN&CLEAN BR has been rinsed out.

**7** If the spittoon bowl and the mouth glass filler are not connected to the bottle system, we recommend carrying out the cleaning and disinfection process with the METASYS BR Biofilm Removing System (METASYS order no. 40050501 METASYS BR Biofilm Removing System + 1 x 1000 ml bottle of GREEN&CLEAN BR Biofilm Removing Solution). Please refer to the directions for use of the device for the relevant instructions. This can be ordered directly from METASYS Medizintechnik GmbH.

The intensive decontamination should be repeated at least once a year and whenever necessary, even if the water pipes are continuously decontaminated.

This mixture is classified as not hazardous according to Regulation (EC) 1272/2008 [GHS].

Composition: 4% hydrogen peroxide, water, colourant

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# ...do you already know?

## Water decontamination & biofilm prevention

### GREEN&CLEAN WK

- > 2% hydrogen peroxide solution for the decontamination of process water and the prevention of limescale deposits in the pipe systems of dental units
- > broad spectrum of efficacy
- > prevention of biofilm
- > with long-term effect
- > limescale inhibitor prevents calcification
- > protects hand and angle pieces
- > also suitable for bottle systems
- > CE 0044

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