INSTRUCCIONES  
ISTRUZIONI  
INSTRUCTIONS  
ANWEISUNGEN  
INSTRUCTIONS

REF 011350-000  
TYPE II, MEDIUM-BODIED

waterpik  
Z.O.E.  
Kelly's  
Impression Paste

#### QUESTIONS?

Please call Water Pik Consumer Affairs at 1-800-525-2020.

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waterpik

Manufactured for  
Water Pik, Inc., a subsidiary of  
Church & Dwight Co., Inc.  
1730 E. Prospect Rd.  
Fort Collins, CO USA 80553-0001  
www.waterpik.com

EC Representative:  
SOFIBEL SAS  
92686 Levallois-Perret Cedex-France  
Church & Dwight UK Ltd.,  
Folkestone, CT19 6PG  
UK: 0800 121 6080

For questions contact Water Pik, Inc.:  
USA: (800) 525-2020  
Canada: (888) 226-3042  
International: (970) 484-1352

Made in U.S.A.  
© 2020 Water Pik, Inc.  
FN: 20017967-F AE

**KELLY'S Impression Paste is an improved zinc oxide - eugenol impression corrective, recommended for use in the preparation of full or partial dentures. It offers many advantages, chief of which are these:**

1. It is easy to mix and to handle.
2. It remains workable in the patient's mouth long enough to permit unhurried impression-seating.
3. It causes no burn in the patient's mouth.
4. It sets quickly after being positioned in the patient's mouth, which minimizes the patient's discomfort.

#### INSTRUCTIONS FOR MIXING

1. Use a clean, dry slab. (A paper mixing pad is ideal for this purpose - use top sheet once, then discard).
2. For an average case squeeze onto the slab equal 1-inch lengths of Base and Hardener. Cut off cleanly from the tube. Always keep tubes tightly capped when not in use.
3. Spatulate the mix and when uniform color is attained (about 1 minute mixing time required), the paste is ready for use.

#### HOW TO USE AS A CORRECTIVE IN THE SNAP IMPRESSION

- a. Remove snap impression from the tray and trim away excess material around the periphery. If it seems necessary, soften the peripheral section in a flame, then return the impression to the mouth and muscle-trim.
- b. Dry the palatal surface of the impression.
- c. Mix KELLY'S Paste and coat the palatal surface of the impression with the paste to a thickness of one to two millimeters.
- d. Have patient rinse mouth thoroughly with cold water, then insert the impression, positioning it very slowly so as to allow time for the paste to flow into every tissue irregularity. Avoid compression by the modeling compound, especially if there are soft or spongy areas in the mouth. When case is properly seated, have patient close mouth and suck in cheeks.
- e. KELLY'S Paste will set in three to four minutes in the patient's mouth provided equal lengths of Base and Hardener are used in the mix. If a faster set is desired, simply increase the ratio of the Red Paste; decrease the ratio to retard the set.
- f. If the impression secured is not completely satisfactory, dry the surface, add more KELLY'S Paste where needed and return the impression to the mouth. Do this as often as necessary to produce a perfect impression.

#### USE AS A CORRECTIVE IN THE FINAL TRY-IN

By following the same directions as shown above for the snap impression, KELLY'S Impression Paste can be used very satisfactorily as a corrective in the final try-in.

#### POURING THE MODEL

No separator is required with KELLY'S. Pour the model in any good stone or plaster. When ready to separate, place the unit in water, approximately 140° F, until thoroughly heated. DO NOT BOIL. After separating, immerse the cast in cold water. This will harden the adhering paste so that it can be easily chipped from the model.

#### CLEANING THE GLASS SLAB

Allow the excess of the mix to harden on the slab, then scrape off with razor-blade scraper or knife.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a dentist.

Non-sterile.

#### IMPORTANT

1. KELLY'S Impression Paste should be kept at room temperature.
2. Keep tubes closed when not in use.
3. The tube caps should not be interchanged to prevent a setting reaction. If a tube cap should stick, warm it slightly.

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