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21 September 2010 - versie 1.0

47 Park Square East, Leeds, LS1 2NL, UK  
Company Number 6779883  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

GBR

## DIRECTIONS FOR USE

BRAND: OptiLube

COMPOSITION: Each 100g contains:

- Glycerine 0.80% • PEG 15,4g • 4-Hydroxybenzoic acid esters 0,0308%
- Sodium hydroxide 0,24% • Purified water 82,73%

USE: Use as Lubricant.

CONTRAINDICATIONS: Sensitivity to Glycerine.

DIRECTIONS OF USE:

SACHETS: Remove sachet from box, tear sachet at one end and apply.

TUBES: Remove cap and peel foil from end of tube apply OptiLube Jelly to instrument or skin, close after use.

RECOMMENDED USE:

OptiLube Jelly is a safe and effective personal and professional lubricant, recommended for:

- Insertion of catheters
- Endoscopy
- Lubrication of condoms
- Rectal thermometers
- General hospital procedures
- During sexual intercourse
- Enemas/douches
- Gynaecological examination
- Digital & instrument examinations.

OptiLube is harmless to human tissue, and will not injure rubber, metal appliances, instruments and natural or synthetic textiles. OptiLube is not a contraceptive.

SIDE EFFECTS AND SPECIAL PRECAUTIONS:

- Precautions: If product is damaged or seal opened, sterility is not guaranteed.  
Warnings: If irritation occurs, discontinue use and consult a doctor. Avoid eye contact. Keep out of the reach of children.

IDENTIFICATION: Sterile, transparent, water-soluble jelly, greaseless and light.

PRESENTATION:

- Tubes: 5g, 42g, 82g, 113g  
Sachets: 2,7g, 5g

STORAGE INSTRUCTIONS:

Store below 30°C  
KEEP OUT OF REACH OF CHILDREN

optimum medical solutions

DATE OF PUBLICATION  
OF THE INSTRUCTIONS:

21st September 2010 - version 1.0

47 Park Square East, Leeds, LS1 2NL, UK  
Company Number 6779883  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

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Kentzler-Kaschner Dental GmbH