Prior to use, carefully read the instructions for use

**EN**

# KOOLINER™

### HARD CHAIRSIDE DENTURE RELINE

Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

#### CONTRAINDICATIONS:

Patients who have shown sensitivity to methacrylates. In case of allergy refer to a physician.

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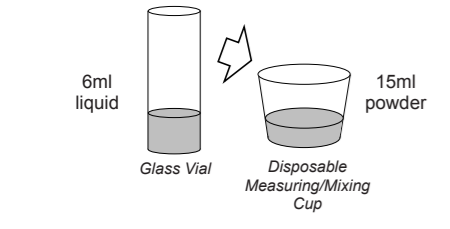
#### DIRECTIONS FOR USE:

1.Preparation of the denture: Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT.

2.Preparation of KOOLINER: Recommended powder/liquid ratio is 15ml powder to 6ml liquid. Pour liquid into the mixing cup and then add the powder slowly. Stir thoroughly for no more than 30 seconds and avoid the introduction of air bubbles.

#### ADD POWDER TO LIQUID AND STIR FOR 30 SECONDS

3. Application. After approximately 1-2 minutes spread the mixture of KOOLINER over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion, and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.


4. Finishing. Peak curing temperature: Approximately 43°C/110°F at 7 minutes when tested according to ADA/ANSI specification number 17. In thicker applications, peak temperature may exceed that stated above, possibly producing a hazardous condition in the mouth during curing. When curing is complete (10 minutes), trim away excess. For smoothing the edges use a hot spatula.

#### STORAGE CONDITIONS:

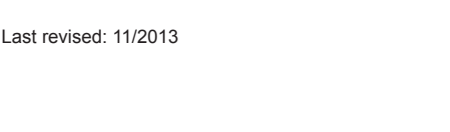Keep dry. Excessive heat and freezing temperatures may impact product performance. For optimal performance store in a cool, dark place (2°C to 37°C / 36°F to 98°F). (Shelf life: 3 years from date of manufacture).

#### PACKAGES:

345001 KOOLINER Professional Package
345002 KOOLINER Powder 3oz (80g)
345091 KOOLINER Liquid (55mL)

MANUFACTURED BY
GC AMERICA, INC.
3737 West 127th Street, Alsip, IL 60803, U.S.A.

DISTRIBUTED BY:
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-Ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

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## KOOLINER™

Prior to use, carefully read the instructions for use.

EN

### HARD CHAIRSIDE DENTURE RELINE

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

A temporary lining for acrylic dentures. For use in chairside procedures.

#### CONTRAINDICATIONS

Patients who have shown sensitivity to methacrylates. In case of allergy refer to a physician. Not intended for permanent lining.

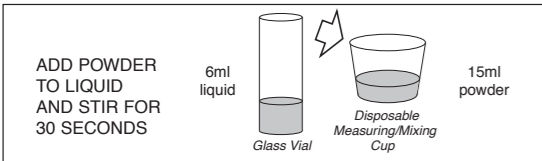
#### DIRECTIONS FOR USE

##### 1. Preparation of the denture:

Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT.

##### 2. Preparation of KOOLINER:

Recommended powder / liquid ratio is 15ml powder to 6ml liquid. Pour liquid into the mixing cup and then add the powder slowly. Stir thoroughly for no more than 30 seconds and avoid the introduction of air bubbles.

##### 3. Application:

After approximately 1-2 minutes, spread the mixture of KOOLINER over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes, instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion, and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.

##### 4. Finishing:

Peak curing temperature: Approximately 40°C/104°F at 6-15 minutes when tested according to ADA/ANSI specification number 17. In thicker applications, peak temperature may exceed that stated above, possibly producing a hazardous condition in the mouth during curing. When curing is complete, trim away excess. For smoothing the edges, use a hot spatula.

#### STORAGE

Recommended for optimal performance; store at temperature of 4-25°C (39-77°F).

#### PACKAGES

**345001 KOOLINER** Professional Package

**345002 KOOLINER** Powder, 80 g

**345091 KOOLINER** Liquid, 55 mL

#### CAUTION

- Patient should clean daily to remove food deposits and plaque. Recommend a commercially available denture cleaner and brush. Do not recommend toothpastes or hard bristle brushes as they may damage the denture liner or denture.
- If patient notices damage to denture (e.g., chipping, delamination, distortion, etc.) or changes in tissue condition (e.g., inflammation, discomfort, allergic reaction, etc.) have patient discontinue use and return for evaluation and consultation.
- Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- Ensure good ventilation/exhaustion at the workplace. Keep ignition sources away.

#### CLEANING AND DISINFECTING RECOMMENDATION

**MULTI-USE DELIVERY SYSTEMS:** To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged.

**DO NOT IMMERGE.** Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <http://www.gceurope.com> or for the Americas: <http://www.gcamerica.com>

They can also be obtained from your supplier.

Undesired effects-Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)  
as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)

In this way you will contribute to improve the safety of this product.

Last Revised: 10/2019

**Rx Only**

**'GC'** **CE** 2797

MANUFACTURED BY  
GC AMERICA INC.  
3727 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

DISTRIBUTED BY  
GC AMERICA INC.  
3727 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

GC EUROPE N.V.  
Researchpark Halbeek-Leeuwen 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32-16-74-10-00

GC SOUTH AMERICA  
Rua Heliodora 399, Santana - São Paulo, Sp - Brasil  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.279.880/0001-00

RESP. TEC. Mayara De Santos Ribeiro - CRO/SP: 105.982

PRINTED IN THE U.S.A.

GC ASIA DENTAL PTE. LTD.  
5 Tampines Central 1, #05-01 Tampines Plaza, Singapore 520541  
TEL: +65-6546-7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1733 Babery Rd, Bundoora, NSW 2019, Australia  
TEL: +61-2-9301-8200

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