# LAMINA VEST II

## ENGLISH REFRACTORY DIE INVESTMENT

### PRECAUTIONS FOR DENTAL PERSONNEL

1. This product contains free silica. Inhalation of free silica for long periods could cause damage to the lung. Use local dust extractor, dust protective mask, etc. to avoid harmful influence of the dust on the human body.
2. Use in well ventilated area with appropriate local ventilation system or fan to avoid inhalation of gas caused by heating this material.
3. Use protective equipment such as glasses while grinding this product.
4. This product is intended for use by dental professionals only.
5. Avoid contact with eyes. In case of accidental contact, immediately flush the eyes with plenty of water and seek medical advice.
6. Use protective plastic gloves to avoid direct contact with this product, since Liquid and the mixture of Powder and Liquid are alkaline.
7. Only dental professionals must be in charge of use and storage of this product.

### INDICATIONS

This product is refractory die investment for the production of porcelain laminate veneers, porcelain jacket crowns and porcelain inlays etc. It is compatible with the SHOFU porcelains, VINTAGE MP, VINTAGE Halo and VINTAGE.

### PHYSICAL PROPERTIES

Item	Specification
Mixing ratio	Powder 40g / Liquid 7mL
Working time	5.5 minutes
Fluidity*	135 mm
Initial setting time*	6.5 minutes
Setting expansion*	1.2%
Linear firing shrinkage*	0.8%
Linear thermal expansion* (600 °C /1112 °F)	0.8%
Compressive strength (2 hours later)*	55MPa

\*Tested in accordance with ISO 15912 (Type 4: Class 1)

<Temperature of the test room : 23 °C /73.4 °F>

### DIRECTIONS FOR USE

1. Mixing: Dilute Liquid with water depending on the case.

- ①For extra coronal restorations (veneers/onlays), the Liquid should be used as it is.  
(Do not dilute Liquid with water.)
  - ②For intra coronal restorations (inlays), mix the Liquid with water in a ratio of 1 : 1.
  - ③Precisely measure 7mL of Liquid (or Liquid diluted with water) and mix with 40g of Powder (1 pack) manually for 15 seconds. Then mix by using a vacuum mixer for 30 seconds.
2. Spray the impression with surface treatment agent for silicone (WAX CLEANER SPRAY etc.). Pour the mixture into the impression and leave undisturbed.
3. Remove the hardened model out of the impression about one hour later.
4. Place the refractory model into the burn-out furnace and hold at 700 °C /1292 °F for 15 minutes.  
Move it to the porcelain furnace preheated at 700 °C /1292 °F and then raise the temperature with the heat rate of 50 °C (90 °F)/min until the temperature reaches 1050 °C /1922 °F, with a hold time of minutes. Take the fired model out of the furnace and allow to cool to room temperature.

### NOTE ON USE

1. Use provided Liquid for LAMINA VEST II only and measure it precisely.
2. For vacuum mixing, use a bowl exclusive for phosphoric acid investment. Do not use a bowl for gypsum investment material.
3. Burn-out furnace is recommended for preheating. Although preheating can be done by porcelain furnace as well, it might contaminate the furnace, as LAMINA VEST II is a phosphoric bonded investment material that releases ammonia gases when heated.
4. Do not use for any purpose other than specifically outlined in the INDICATIONS in these Instructions for Use.
5. Use this product within the expiration date indicated on the package and container.  
※(Example ㊦YYYY -XX → Shelf life: at the end of XX month, YYYY year)

### STORAGE

1. Store at room temperature (15-25 °C /59-77 °F). Avoid high humidity and high temperature. Do not store the Liquid in the cold place at frozen point.
2. Tightly close the Liquid cap after each use.

### COMPOSITION

POWDER : Silica, Ammonium phosphate, Magnesium oxide, Aluminium oxide and others

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### EMBALLAGE

POUDRE : 1.2kg (40g×30)

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<b>Manufacturer</b>
SHOFU INC, 11 Kamitakamatsucho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU DENTAL ASIA-PACIFIC PTE. LTD., 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA
SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD, No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
<b>EC REP</b> SHOFU DENTAL GmbH Am Brill 17, 40878 Ratingen, Germany
www.shofu.de