Lucitone® Intensive Color

Instructions for Use	English (2)
Mode d'emploi.....	French (6)
Istruzioni per l'uso	Italian (8)
Gebrauchsanweisung	German (4)
Instrucciones de uso.....	Spanish (10)
Instruções de utilização.....	Portuguese (13)
Gebruiksaanwijzing	Dutch (14)
Kraftfull hållbar akrylprotesbas	Swedish (15)
Brugsvejledning.....	Danish (16)
Bruksanvisning.....	Norwegian (17)
Návod k použití.....	Czech (18)
Instrucțiuni de utilizare	Romanian (19)

Lucitone®
Intensive Color
Intensivfarbe
Couleur intense
Colore intenso
Color intenso

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
Instrucciones de uso

Indications for Use

Lucitone HIPA Denture Base is a self-curing denture base material designed for use in fabrication, repair, rebasing, or relining of full and partial dentures including implant overdentures or other dental appliances.

Product Information

Developed in conjunction with Lucitone® HIPA (indications given above), Lucitone Intensive Colors are highly pigmented acrylic powders that enable the technician to mix with cold (self) curing acrylic powder/ liquid systems and individually characterize a denture (gingival area and flange area).

Contraindications

- Lucitone Intensive Colors are contraindicated for patients and users who have a history of allergic reaction to methyl methacrylate monomer.

Warnings

- See the Warnings provided for the specific cold (self) curing powder/ liquid system used for mixing and characterization.

Precautions

- See the Precautions provided for the specific cold (self) curing powder/ liquid system used for mixing and characterization.

Additional precautions for use of intensive colors:

- When grinding prosthodontic resins, proper ventilation, masks, and vacuum systems should be used.
- Store at 60 °F - 80 °F (16 °C - 27 °C), away from moisture and direct sunlight.
- Warm materials (powder or liquid)

will reduce working time significantly. The best results are achieved when material temperatures are at or below 73 °F/ 23 °C.

- For disposal please follow the instructions on the safety data sheet or national regulations.

Adverse Reactions

- See the Adverse Reactions provided for the specific cold (self) curing powder/liquid system used for mixing and characterization.
- Particulates will be generated when grinding acrylate resins. Eye, skin and respiratory irritation may occur if appropriate engineering controls are not used.

Symbols on product labels

Manufacturer

Batch Code

Reorder Number

Manufacture Date

Expiration Date

Consult Instructions for Use

Caution

Keep Away from Sunlight

CE European Conformity Marking

Medical Device

U.S. Prescription Only

Store Between

Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

STEP BY STEP INSTRUCTIONS

Properties and Recommended Procedures

The following table describes typical properties of Lucitone Intensive Colors, plus recommended temperatures and other important processing information. Further detail of this information is found later in the text.

Storage temperature for powder	60 °F - 80 °F (16 °C - 27 °C)
Powder / Liquid ratio dependent on denture base material (for HIPA Denture Base Powder and Liquid)	20 g (28 cc) / 15 ml
Temperature (working)	60 °F - 80 °F (16 °C - 27 °C)
Cure time, temperature, and pressure is dependent on denture base material (when using HIPA Denture Base Material).	pressure pot, 30 minutes minimum of 20 psi (1.4 bars), 113 °F ± 2 °F (45 °C ± 1 °C)

CHARACTERIZATION OF DENTURE USING COLD CURE DENTURE BASE ACRYLIC AND LUCITONE INTENSIVE COLOR (CAN BE USED WITH ANY COLD CURING ACRYLIC)

1. Grind the denture to roughen the surface at the location where the characterization will be done.
2. Blow with oil-free, compressed air to clean the surface.

3. Add small portion of the desired Lucitone Intensive Color(s) powder to the cold cure acrylic liquid and stir, then add the cold cure denture base powder and mix thoroughly. The material will become thicker as it sets on the bench.
4. Wet the prepared surface with the cold curing liquid.
5. When appropriate consistency is reached, apply the intensive colored material to the prepared denture.
6. Cure the characterized denture in a pressure pot according to the DFU of the cold cure denture base material.
 - For Lucitone® HIPA Denture Base, cure by placing the characterized denture in a pressure pot with water at 113 °F ± 2 °F (45 °C ± 1 °C) and immediately apply a minimum of 20 psi (1.4 bars) pressure. Cure for 30 minutes.
7. Additions can be made many times in the same way to achieve desired appearance.
8. Finish and polish the characterized denture using normal procedures.

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