## Gebrauchsinformation

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## User information

**Product description:** Tooth cleaning and polishing paste for professional cleaning and polishing of the tooth surfaces in the course of preventative care. The composition of **Mira-Clin hap** makes it suitable for rough to fine polishing and helps desensitise sensitive areas on the tooth and neck of the tooth. Contains xylitol and hydroxylapatite which are beneficial for the teeth. Free from fluorides, gluten and phthalate.

**Indications:** For removing superficial soft and hard plaque and for use after removing dental calculus. For removing extrinsic discolouration (e.g. coffee, tobacco or tea stains).

**Contraindications:** This material may not be used:

- if an allergy to one or more of the ingredients exists or contact allergies exist,
- if the patient doesn't have full control of their swallowing reflex

**Basic safety information:**

- Only for use in dental treatment!
- Keep away from children!
- Do not swallow!
- Avoid skin contact! In case of accidental skin contact, wash the affected area thoroughly with soap and water.
- Avoid eye contact! In case of accidental eye contact, flush immediately with lots of water and consult a physician if necessary.

**Side effects:** Swallowing an excessive amount of the material can lead to nausea and vomiting.

**Usage information:** Suitable for all commercially available cleaning brushes or polishing cups. Use brushes for occlusal surfaces.

**Recommended use:**

- When first opening a tube, pull the tamper-evident safety seal off using the tab and discard it. Make sure to close the tube after use so that the paste does not dry out.
- Place a sufficient amount of paste on a mixing block.
- Pick the material up with the polishing cup or polishing brush.
- In order to reduce the risk of overheating and splattering of the preventative care paste, work with a correspondingly adjusted speed (2000 – 3000 RPM.)
- After use, flush the cleaned areas with lots of water.
- Siphon off excess material and have the patient spit out thoroughly.

**Storing and shelf life:**

- Store at room temperature (15 to 25 °C / 59 to 77 °F)!
- Do not store in the fridge!
- Protect against direct sunlight!

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