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GB
Manufacturer Information on Reprocessing

Hersteller: HAGER & WERKEN GmbH & Co. KG
Ackerstraße 1, 47269 Duisburg, Germany
T +49 (203) 99269-0, F +49 (203) 299283
www.hagerwerken.de

Product:
Miramatic Holder Plus (REF 355 466)
Miramatic Holder (REF 355 465)
Updated: 10/2016 (Rev: 2)

Warning notices: No special notices

Limitation of reprocessing: Frequent reprocessing has only got a small impact on this instrument. The instrument's shelf life is mainly determined by wear and damage by use.

Preparations for cleaning: Clean surface with a disposable cloth or paper.

Cleaning: Only use suitable cleansers, carefully following the instructions of the respective manufacturer. After cleaning, remove cleanser thoroughly under running water. Clean inner part with something convenient (e.g. a cotton stick). An automated cleaning is possible.

Disinfection: Only use suitable disinfecting solutions, carefully following the instructions of the respective manufacturer. Remove disinfecting solution thoroughly under running water. An automated disinfection is possible.

Maintenance: Lubricate the locking button with a drop of oil when needed (Miramatic Holder Plus).

Packaging: Standardised packaging material may and should be used for sterilisation processes.

Sterilisation: Vapour sterilisation at 134 °C for 5 minutes,
vapour sterilisation at 121 °C for 15 minutes.

Storage: No special requirements.

2016012902-1