# MIRATOR 2000 E

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### 1 ALLGEMEINE INFORMATIONEN

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### 1 GENERAL INFORMATION

#### 1.1 Objective of these instructions

This manual was created by the manufacturer and is an integral part of the device. The information contained in this document is addressed to the user and contains all safety recommendations. Before using the unit, this manual should be read carefully to ensure that you are using the controls, their function and position as intended. It is also advisable to carry out performance tests.

This manual has to be kept in order to be able to go back to it later.

#### 1.2 Identification of the device manufacturer

On the device you can find the manufacturer's label and some information about the technical data as shown in Fig. 1

 <small>Ackerstraße 1, 47289 Duisburg</small>		
MOD	ANNO	Matr. N°
<b>Mirator 2000E</b>	<b>2003</b>	<b>6608</b>
Voit <b>230 AC</b>		WATT <b>200</b>
HZ <b>50 - 60</b>		Kg <b>4</b>

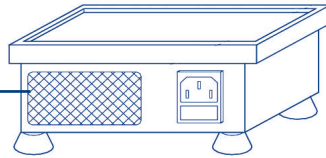
Fig. 1

#### 1.3 Technical information

The Mirator 2000 E is electronically controlled. By pressing the up or down arrow button, you can select the vibration level of the vibration plate.

#### 1.4 Safety devices

Parts of the device that are powered by electricity are protected by fixed devices to prevent access by the user. Only specialized and authorized personnel may remove these parts for unusual maintenance and repairs.

#### 1.5 Technical specifications

Power supply: 230 V - 50/60 Hz  
 Power: 200 W  
 Fuse: 1A

Measures of the  
 Vibrating plate: 13 x 15 cm  
 Weight: 3.5 kg

### 2 TRANSPORT AND INSTALLATION

#### 2.1 Packaging and removal

The packaging is as follows:

- Polystyrene plates K10, which protect the unit on the sides, on the top and on the ground
- Fixed cover

Disposal of packaging waste must be carried out in compliance with the relevant regulations and in an environmentally friendly manner.

#### 2.1 Installation

The device must be placed in a safe place and in a horizontal position. Ensure there is sufficient ventilation. It is the responsibility of the user to ensure that the electrical system is in accordance with the safety regulations. It is particularly important to ensure that the grounding is effective. In addition, it is important to check the line voltage: if the current voltage is too low (less than 210V), the motor will start too slowly and it may be necessary to install a voltage regulator.

After placing the device and the above checks, follow these instructions:

- The main switch 1 must be in the OFF (0) position
- Connect the power cable 4 to the 220-230 V AC output

### 3 APPLICATION INFORMATION

#### 3.1 Performance and application

Please follow carefully the following information, see Fig. 2:

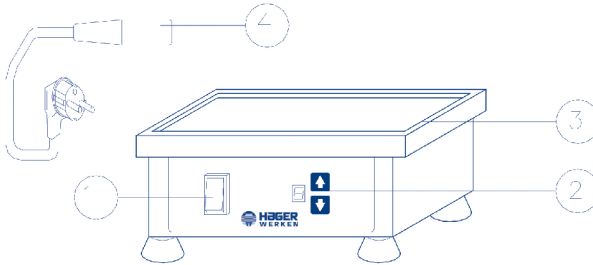
Fig. 2

1. Place the workpieces on the vibration plate 3, make sure that the Mirator 2000 E is switched off (position 0 of the switch 1).
2. Turn on the Mirator 2000 E with the 1 switch. Initially the intensity is low. The display shows the value 1. Press the arrow button 2 (HIGH = Increase and DOWN) to change the intensity of the vibrating plate. When the intensity changes, the value displayed in the display also changes (Maximum = value 9).
3. Pour gypsum or imprinting material, making sure that no material is placed on the plate.
4. Increase vibration to the set point, and keep the vibration for the required time to compress the material.
5. Turn the power off and remove the workpieces from the vibrating plate.
6. At the end of the procedure, clean the plates with a dry or slightly damp cloth.

### 4 MAINTENANCE INSTRUCTIONS

#### 4.1 Normal maintenance

The device does not require any special maintenance. However, it is advisable to remove dust and wipe it with a dry or slightly damp cloth.

**WARNING:** NEVER LET THE UNIT BE IN DIRECT CONTACT WITH WATER WATER OR WITH OTHER LIQUIDS OR CLEANERS. THIS HARMS THE DEVICE AND THE WARRANTY IS INVALID.

The device is not waterproof and the liquids that come into contact with the electrical parts of the unit may cause damage and the risk of electric shock to the user.

#### 4.2 Extraordinary maintenance

To repair or replace parts, the user should consult the manufacturer or specialist technician directly. It is imperative not to remove the solid protective devices without sufficient precautions. Disconnect the power plug from the outlet before any extraordinary maintenance. If the user does not comply with the rules contained in this manual, the manufacturer is exempted from any warranty or responsibility.

### Electrical scheme


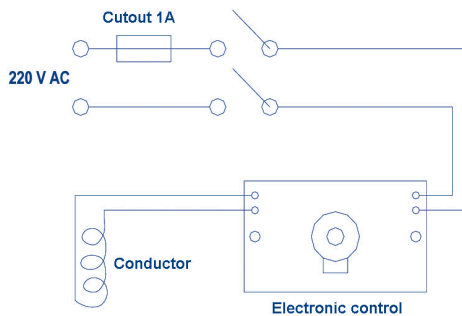
Fig. 3

## Spare parts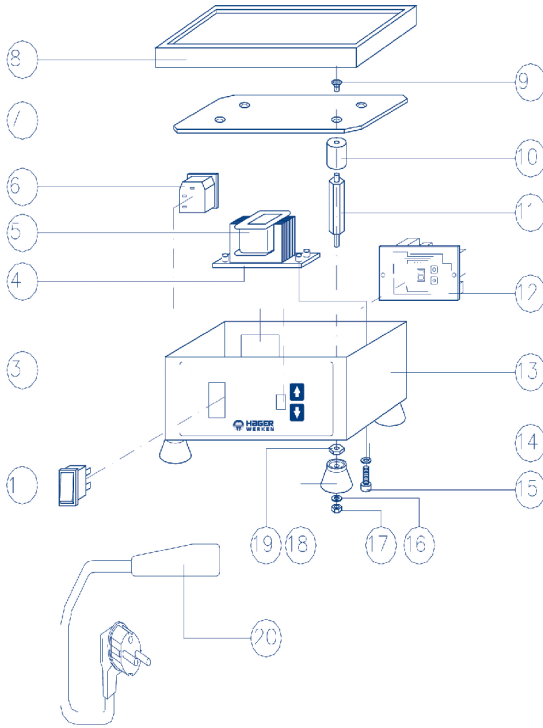

POS	Description
1	Illuminated switch
3	Polycarbonate plate
4	Electromagnetic base
5	Electromagnetic coil
6	Connector IEC
7	Vibrating plate
8	Rubber mat
9	Screws M5x8 TSCE
10	Antivibration
11	Stud bolts
12	Electronic card
13	cars
14	Serrated gasket
15	Screws M4x15 TCCR
16	Sealing
17	Nut M5
18	Mounting
19	Mother
20	Supply cables

### Guarantee:

The producer ensures that the product is free of defects and that it shows all guaranteed features and/or all characteristics agreed upon. The warranty starts with the day of delivery of the product and is valid for twelve months. Excluded from the warranty are faults, careless use of the product, connection to unsuitable power supply, operation with the wrong voltage, acts of God, such as fire, lightning, humidity, etc. In case of justified complaints the producer has the right to repair, to deliver spare parts or to decrease the sales price. Cancellation of the sales contract can only be claimed by the customer after two repairs were carried out without success.