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MUPPY® designs MUPPY® wire guard is available in two different sizes:

- with red ring for deciduous dentition (size I)
- with blue ring for mixed dentition (size II)

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Instructions for use Before using MUPPY® for the first time, clean it thoroughly under warm running water. Check MUPPY® regularly for places that have been bitten through! Replace MUPPY® when necessary.

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Cleaning instructions Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the Dr. Hinz Orofan® dental cleaning sticks. MUPPY® is neither dishwasher- nor microwave-safe. Clean the hygienic MUPPY® Box only with a damp cloth.

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MUPPY® for treatment *In principle, the frontal open bite is the only anomaly that through self-healing, finds its way back to the normal overbite by keeping the tongue away once the cause is weaned off. This also applies to the mixed teeth when the open bite is accompanied by a missing tooth position. The treatment is named "interceptive therapy" because its success is based on the weaning of the habit. In such cases, MUPPY® must also be worn regularly during the day – often over several months.*

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The wearing calendar can be downloaded from dr-hinz-dental.de/muppy

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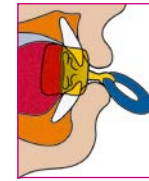
MUPPY® rules *Studies have shown that more than 90 % of children give up damaging sucking with the help of MUPPY®. Your support and a few rules help your child:*

- 1 *MUPPY® is a substitute for thumb and dummy. The substitution must be voluntary.*
- 2 *MUPPY® must always be worn at night.*
- 3 *To get used to it easier children should wear MUPPY® also a few hours during the day (when playing or watching television).*
- 4 *As a rule, weaning is successful within a week. However, MUPPY® should also be worn for 6–8 weeks after successful weaning of the dummy ("recidivism").*
- 5 *MUPPY® can be stored in the hygiene box.*
- 6 *Do not forget to regularly brush the teeth and MUPPY®!*

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How is MUPPY® worn? *MUPPY® is used similarly to a common weaning dummy. The shield is placed between the lips and teeth. The tongue crib lies behind the teeth and prevents the tongue from positioning itself between the rows of teeth.*

If your child is having speech therapy treatment, you should take note of the following:

- Follow your speech therapist's individual wearing and exercise instructions exactly.
- Support your child during this stage, as he/she has to feel the importance of mouth closure in order to take the exercises seriously.
- Make your child aware of the "rules of play" and supervise the necessary exercises.