# AESCULAP®

**en Instructions for use/Technical description**

STERILIT Power Systems oil spray GB600

**USA**

Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at [www.aesculapusaifus.com](http://www.aesculapusaifus.com). If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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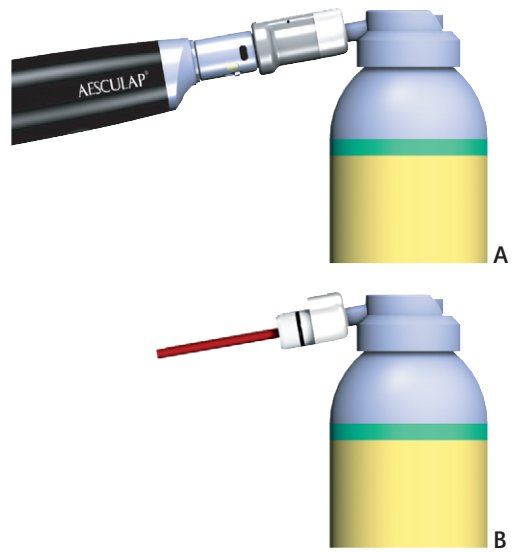
## 1. About this document

### Note

General risk factors associated with surgical procedures are not described in these instructions for use.

### 1.1 Scope

These instructions for use apply to the product below:

Art. no.	Designation
GB600	STERILIT Power Systems oil spray

### 1.2 Safety messages

Safety messages make clear the dangers to patient, user and/or product that could arise during the use of the product. Safety messages are labeled as follows:

#### ⚠ WARNING

Indicates a possible threat of danger. If not avoided, minor or moderate injury may result.

## 2. Clinical use

### 2.1 Product description

Oil for the maintenance of Aesculap Power Systems prior to sterilization. Suitable for steam sterilization methods validated according to EN ISO 17665.

#### Properties

- Oil film permeable to water vapor
- Oil film not affecting the effectiveness of sterilization
- Lubrication and anticorrosion effect
- Silicone-free

### 2.2 Areas of use and limitations of use

#### 2.2.1 Intended use

STERILIT is used within the reprocessing process to lubricate joints and movable parts in order to reduce friction and prolong product life. It is used on the cleaned medical devices prior to steam sterilization validated according to EN ISO 17665.

#### 2.2.2 Indications

##### Note

The manufacturer is not responsible for any use of the product against the specified indications and/or the described applications.

For indications, see Intended use.

#### 2.2.3 Contraindications

No known contraindications.

## 2.3 Safety information

### 2.3.1 Clinical user

#### General safety information

To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:

- ▶ Use the product only according to these instructions for use.
- ▶ Follow the safety and maintenance instructions.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge and experience.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to use, check that the product is in good working order.
- ▶ Keep the instructions for use accessible for the user.

##### Note

The user is obligated to report all severe events in connection with the product to the manufacturer and the responsible authorities of the state in which the user is located.

#### Notes on surgical procedures

The medical professional will make decisions on concrete applicability based on the warranted properties and technical data.

### 2.3.2 Product specific safety information

Risk of injury and/or malfunction!

- ▶ Check product for loose, bent, broken, cracked, worn or missing parts before every application.
- ▶ Extremely flammable aerosol.
- ▶ Pressurised container: May burst if heated.
- ▶ Harmful to aquatic life with long-lasting effects.
- ▶ Keep out of reach of children.
- ▶ Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
- ▶ No smoking.
- ▶ Do not spray on an open flame or other ignition source.
- ▶ Do not pierce or burn, even after use.
- ▶ Protect from sunlight.
- ▶ Do not expose to temperatures exceeding 50 °C / 122 °F.
- ▶ Ensure aerosols are completely empty before recycling.
- ▶ Observe the Safety Data Sheet, see [cifu.bb Braun.com](http://cifu.bb Braun.com)

#### Safe handling

- ▶ Prior to the application of STERILIT, medical products must be cleaned and disinfected according to their respective instructions for use.
- ▶ Only apply prior to sterilization.
- ▶ Ensure adequate ventilation.
- ▶ Store in a dry place.
- ▶ Evaporative cooling effect during use!

#### Emergency telephone number

+49 (0) 6132-84463 GBK GmbH

- 24/7
- 190 languages
- medical advice in case of poisoning
- Advice in the event of an accident or other incidents

#### Unique Formula Identifier (UFI-Code)

UFI: TTSA-H6KK-X70D-22SP

### 2.3.3 Sterility

The product is supplied non-sterile and shall be used under non-sterile conditions.

- ▶ After removing its transport packaging and prior to first use inspect the new product to ensure it is in proper working condition.

## 2.4 Application

- ▶ Observe the instructions for use of the corresponding motor/handpiece.
- ▶ Application using oil spray adapter, see Fig. A:
  - Attach a suitable adapter to the spray can.
  - Attach the adapter to the connector on the handpiece/motor.
  - Hold the oil spray can upright.
  - spray thoroughly for approx. 2s.
- ▶ Apply using extension tube, see Fig. B:
  - Insert the enclosed red extension straw and make sure that it is firmly secured.
  - Hold the oil spray can upright.
  - Oil selected points.
- ▶ Remove excess oil using a lint-free cloth.

## 3. Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 7461 95-1601

Fax: +49 7461 16-2887

E-Mail: [ats@aesculap.de](mailto:ats@aesculap.de)

Other service addresses can be obtained from the address indicated above.

## 4. Accessories

STERILIT Power Systems adapter	For hand piece/motor
GB600840	Acculan 4 Acculan 3Ti
GB600880	Acculan 4 GA344 ELAN 4 GA844
GB600860	ELAN 4 electro
GB600850	ELAN 4 air
GB600870	ELAN 4 MIS shafts microspeed uni XS
GB600820	microspeed uni HiLAN HiLAN XS Hi-Line Hi-Line XS Hi-Line XXS
GB600810	micro-line
GB600830	mini-line

## 5. Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging.

## 6. Symbols on product and packaging

Danger! Extremely flammable aerosol

**REF**

Manufacturer's order number

300 ml

Nominal filling quantity

e

Conformity mark for pre-packaging ordinance

**405**

Full capacity volume

**3**

Conformity mark for aerosol packaging

Use by: see bottom

Emergency telephone number

Observe the instructions for use

Hold the oil spray can upright

Period of time for spraying thoroughly

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