

Okklean and Telescan

Application:

For extraoral application only!

Occlusion spray

To visualize pre contact points and for adjustment work on crowns and bridges.

Telescan

Is for adjustment work of telescopic crowns and attachment work and for preparation of models prior to scanning.

Mode of application:

Shake well before use. Spray at a distance of app. 3-5 cm. Soft, intermittent short bursts while turning the sprayed object. No cover application needed. Can be removed easily with a brush, water vapor or running water. No discoloration of the plaster.

Quality:

Ultrafine spray vapor. Unlike other occlusion sprays Okklean and Telescan are not water soluble, therefore plaster models are not discolored by absorption of colored water.

Risks:

Avoid inhaling spray vapor.

Storage:

- Container is under pressure.
- Protect the bottles from warming above 50°C.
- Do not expose to sunlight, risk of explosion.

Disposal:

Together with general lab waste (metal can).

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/14