Without being held firm by hand, Handpiece could be pushed away by air-pressure of spray. Accidental drop of the equipment could cause damage on it.

2. Surface Cleaning:

Clean with brush and wipe off by cloth. Lubrication should be done every time after surface cleaning. Although it has effect to apply a little alcohol on the cloth, excessive amount, by seeping inside Handpiece, could take off the lubricate oil on ball-bearing. In that case, lubricate again.

3. Inside Cleaning:

Take the same steps with the process of spraying lubricate oil. Operate trial run at low speed every after lubrication. In order to drive dirt out of nose, repeat the process at least twice. Do the cleaning once a day.

Sterilization

Do not put Handpiece through ultrasonic disinfection solution cleaning procedure. Do not use dry heat!

1. Clean with brush and wipe off by cloth. Refer to the section, Maintenance, 2. Surface Cleaning.
2. Lubricate with spray. Refer to the section, Maintenance, 1. Lubrication.
3. Pack every instrument part into an adequate and fitting transparent, sterile pack (sterilisation bag), packed and sealed in accordance with EN 868-5. Strictly adhere to the instructions given by the manufacturer of the sterilisation bag and the manufacturer of the sealing machine, following the normative requirements.
4. Autoclave at the temperature 134°C (2Mpa) for 15 min.
5. Take the Handpiece out of the autoclave and dry it in room temperature.

Do not put operate autoclave dry cycle at the temperature higher than 134 °C!

Warning

1. Do not drop Handpiece nor apply excessive force or impact on the bur connected. Warranty is invalid for the trouble caused by those acts.
2. When water gets inside the Handpiece, lubricate more than three times.
3. Do not run Handpiece without locking bur.
4. Do not run Handpiece with chuck opened.
5. For safety, mount test bur, the standard accessory, on the Handpiece, when it is not in use.
6. In using the bur with its working diameter more than 5 mm, gradually increase the speed in order to assure positive retention.

In high speed use, the bur with large operation diameter could fly off and cause serious damage on human body!

The above mentioned instructions are validated as SUITABLE for the preparation of medical products prior to re-use. The reprocessing employees ensure that the actually performed reprocessing and the utilization of equipment, materials and personnel achieve the desired results. Validation and routine supervision are required for this purpose. Likewise, each deviation from provided instructions performed by the reprocessing employees should be carefully examined and evaluated regarding efficacy and possible negative outcome.

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REF 452 159

Instructions for Use


This product is designed for dental use. Its operation and maintenance should be done under dentist's supervision!

Specification:

1. Maximum Rotation Speed:	40.000 rpm
2. Applicable Bur:	HP type ISO standard (Ø 2.35)
3. Chuck Ability Force to Withdraw:	More than 5 kg
Rotation Direction:	More than 450 g. cm
4. Bur Concentricity:	Less than 0.015
5. Coupling with Motor:	ISO standard (E-type)
6. Sterilization:	134 °C autoklave
7. Gear Ratio:	HST-30A 1 to 1 HST-R 4:1 4 to 1

1. Coupling with Motor:


In order to couple with the motor, push the motor nose into the sheath of the Handpiece all the way until the stopper of the motor is properly hooked in the inside groove of the Handpiece. Verify proper coupling by pulling the Handpiece and the motor. Please make trial run at low speed after coupling.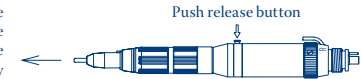

2. Bur Change:

At the bur-locking position, where the dot on chuck locking ring meets the one on Handpiece body, turn the chuck locking ring to the direction indicated by arrow with the Handpiece held fixed, all the way till it couldn't go any further.

Pull off test bur, and insert a clean bur. Turn back the chucklocking ring, with the Handpiece held fixed, all the way till it couldn't go any further, where the two points meet each other.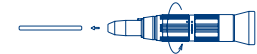

3. Operation:

Use with motor running up to max. 40,000 rpm.

Maintenance

To keep Handpiece under good condition, lubricate oil specifically made for dental handpiece should be used. Use the spray as instructed by its manufacturer. Lubrication should be done every day.

1. Lubrication

Be sure to lubricate after autoclave, and in the case that water and / or dust gets inside. Spray lubrication could also make inside cleaning. It should be done twice each time. Spray lubricate oil at the back of sheath of Handpiece. (It has the same effect with spray lubrication that air, with the hose attached, is supplied after 2 drop of lubricate oil is placed at the back of sheath.)