

PerForm  Proximal Contact Instrument

Garrison<sup>®</sup>

PerForm  Proximal Contact Instrument

Garrison<sup>®</sup>  
Dental Solutions

Garrison<sup>®</sup>

PerForm  Proximal Contact Instrument

Tear open for instructions.

Tear open for instructions.

Garrison<sup>®</sup>  
Dental Solutions

888.437.0032

REORDER TODAY!  
Toll-free 888.437.0032 • [www.garrisdental.com](http://www.garrisdental.com)

Non-sterile, RX only, always use a rubber dam.

For cleaning and sterilization instructions go to:  
[garrisdental.com/sterilization](http://garrisdental.com/sterilization)


USA Office  
150 Dewitt Lane  
Spring Lake, MI, USA 49456  
P 616.842.2244 F 616.842.2430

Europe Office  
Carlstrasse 50  
52531 Uebach-Palenberg, Germany  
P +49.2451.971.409


 D. R.M. Green, Authorized Representative European  
Compliance Services Ltd.  
Cypress Court, Fenalea Square  
Oxford, OX1 1BU, UK

1039r1113


1a. pull (mesial)

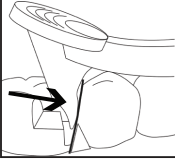

1b. push (distal)

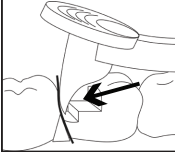

2.

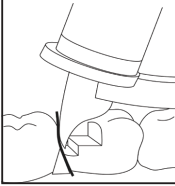

1. Place the first composite increment to the level of the pulpal floor.
2. Press the appropriate end of the PerForm instrument into the uncured composite increment and either push (distal) or pull (mesial) as indicated on the handle until desired contact is achieved. Do not pry! (figure 1a & 1b)
3. Place the curing light directly on the Fresnel lens and cure the composite while maintaining the mesial or distal pressure. (figure 2)
4. Remove the PerForm instrument and fill the small void left by the tip with flowable composite.