Prior to use, carefully read the instructions for use.

## GC PLIERS

PLIERS FOR REMOVAL OF PROSTHETIC APPLIANCES

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### RECOMMENDED INDICATIONS

- Removal of luted temporary prosthetic appliances.
- For removal of all types of prosthetic appliances during trial fitting.

### DIRECTIONS FOR USE

- REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES
  - Use the GC PLIERS without the rubber pads.
  - Hold the cervical area of the crown with the tips of the pliers.
  - Twist the crown alternatively towards the mesial and distal direction.
  - Remove the crown in a labiolingual direction.

### 2. REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING

- Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers.
- Place emery powder on the surface of the pads in order to improve the grip.
- Continue with 1 b).

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Viimati parandatud : 06/2018

**'GC'**

MANUFACTURED and DISTRIBUTED by  
 EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240,  
Interleuvenlaan 33,  
B-3001 Leuven, Belgium  
TEL: +32 16 74 10 00  
PRINTED IN BELGIUM  
06/2018

## GC PLIERS

### PLIERS FOR REMOVAL OF PROSTHETIC APPLIANCES

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

1. Removal of luted temporary prosthetic appliances.
2. Removal of all types of prosthetic appliances during trial fitting.

#### DIRECTIONS FOR USE

1. REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES
  - a) Use the GC PLIERS without the rubber pads.
  - b) Hold the cervical area of the crown with the tips of the pliers.
  - c) Twist the crown alternatively towards the mesial and distal direction.
  - d) Remove the crown in a labiolingual direction.
2. REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING
  - a) Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers.
  - b) Place emery powder on the surface of the pads in order to improve the grip.
  - c) Continue with 1 b).

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

#### CLEANING

Cleaning of the GC Pliers made out of stainless steel, can be accomplished by hand scrubbing, ultrasonic cleaning or thermodesinfection with a non-corrosive, neutral detergent. For further details consult the respective manufacturer's instructions.

#### STERILISATION

GC Pliers may be routinely sterilized by the following methods : autoclave, chemiclave or dry heat.

#### STORAGE

Recommended for optimal performance, store in a dry place.

#### PACKAGES

1. Introductory set: one pair of pliers, 50 rubber pads, 4 g emery powder
2. 50 rubber pads
3. 4 g emery powder

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gcamerica.com>. They can also be obtained from your supplier.

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Kohdent Roland Kohler Medizintechnik GmbH & Co.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.
GC AUSTRALASIA DENTAL PTY. LTD.