# Pro-Tip<sup>®</sup>

---

# *TURBO*

## Replacement 'O' Ring Seal Kit

Installation Instructions

CONTENTS: 2 x 'O' Rings


**REF** 116239

116239A

<b>Index</b>	<b>Pages</b>
EN .....	3-6
NL .....	7-10
FR .....	11-14
DE .....	15-18
IT .....	19-22
PT .....	23-26
ES .....	27-30

## EN Replacing the 'O' Ring Seals


To maintain optimum performance from the Pro-Tip® system it will be necessary to replace the 'O' Rings after a period of time. The Pro-Tip® to converter sealing arrangement is universal on all converter types and contains a black 'O' Ring (a) and a red 'O' Ring (b) **(Figure 1)**.

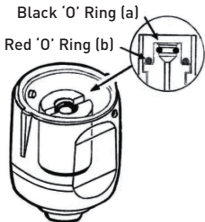**Figure 1**

## Black 'O' Ring


Remove the black 'O' Ring (a) and the red 'O' Ring (b) with a sharp probe. Take the black 'O' Ring (a) and squeeze it into an oval form, between thumb and finger, and place it into the converter bore at a right angle to the cross groove so that it sits upright as shown (Figure 2).

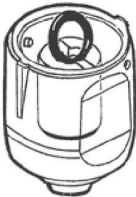**Figure 2**

Using a lining applicator, or other non-sharp instrument, gently push the top of the 'O' Ring downwards and tuck it under and into the internal seal groove **(Figure 3)**.

Once the black 'O' Ring (a) is partially inserted as above, use the instrument to gently ease the opposite side of the 'O' Ring upwards and into the groove **(Figure 4)**. Finally ensure that the 'O' Ring is lying straight and evenly all the way round in

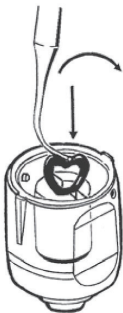
**Figure 3**

the groove. Lubricate with a small amount of petroleum jelly to aid easy insertion of the Pro-Tip® nozzles.

### **Red 'O' Ring**

Place the red 'O' Ring into position and seat it into the groove with a non-sharp instrument.

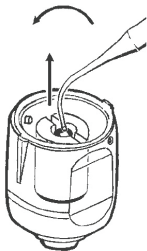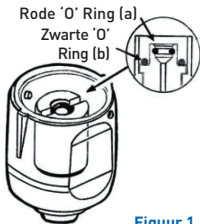
**Figure 4**

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**Young Innovations, Inc.**