## COMPOSITION

Chlorure d'aluminium hexahydraté 25 % m/V, oxyquinol, excipient hydroalcoolique.

### PROPRIÉTÉS

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### (EN)

**COMPOSITION**
Hexahydrated aluminium chloride 25 % w/V, oxyquinol, hydroalcoholic excipient.

#### PROPERTIES

Aluminium chloride has an astringent effect. It induces local haemostasis and the contraction of the upper strata of the free gingiva by precipitating tissular and serum proteins.

#### INDICATION

- gingival exudation and gingival haemorrhages,
- gingival retraction by *in situ* impregnation of a cord.

#### CONTRAINDICATIONS

Possible allergy to aluminium.
Any history of hypersensitivity or photosensitivity to quinoleic ingredients.

#### SIDE EFFECTS

Allergic reaction, burning sensation.

#### INSTRUCTIONS FOR USE

- **Gingival exudation and gingival haemorrhage**
First insert a (non-medicated) gingival retraction cord into the sulcus. Carefully impregnate the cord *in situ* with a few drops of solution, using a pipette. Leave the cord in place for 3 to 5 minutes. Remove it, making sure that no fragment remains into the gingival sulcus. Wash thoroughly.

- **Gingival retraction**
First insert a (non-medicated) gingival retraction cord into the sulcus. Carefully impregnate the cord *in situ* with a few drops of solution, using a pipette. Leave the cord in place for 3 to 5 minutes, the retirer en s'assurant qu'il ne reste aucun fragment dans le sillon gingivo-dentaire et rincer abondamment.

#### WARNINGS AND PRECAUTIONS FOR USE

- Instil only the number of drops required to obtain the desired result.
- Reduce to a minimum the contact time of the solution with the mucosa.
- As soon as local haemostasis is obtained or the impression performed, thoroughly wash the application site so that there is no potentially irritating aluminium salt left.
- Not to be used for over-impregnation of a medicated cord.
- Do not swallow.
Warning
Flammable liquid and vapour.
Cause skin irritation.
Cause serious eye irritation.
Refer to safety data sheet.

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