
en Dental Adhesive Resin Cement

[READ CAREFULLY BEFORE USE]

PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS
1. Do not use this product on patients with known allergies to this material, acetone and/or methacrylate monomers.
2. Operators with known allergies to acetone, methacrylate monomer must not use this product.
3. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
4. Use protective medical (dental) gloves and protective glasses to avoid sensitization to this product.

APPLICATION
Adhesive cementation of Alumina or Zirconia restorations, porcelain or indirect composite resin laminate veneers, crowns, bridges, inlay, onlay and metal reinforced indirect restorations.
Recommended for cementation of various types of posts and core materials.
DIRECTIONS FOR USE
I Pretreatment of cavity or abutment tooth
1. Remove temporary seal and cement completely and clean cavity or abutment tooth using the conventional method.
2. Pulp protection: In case where cavity is close to pulp, pulp capping with calcium hydroxide is indicated.

Recommended pressure and primer for each restorations
Restorations Pressure Primer
Metal about 0.3-0.5 MPa (about 3-5 kgf/cm²) M.L. Primer SHOFU Universal Primer
Porcelain, Indirect Composite resin about 0.1-0.2 MPa (about 1-2 kgf/cm²) Porcelain Primer
Alumina or Zirconia ceramics about 0.2-0.3 MPa (about 2-3 kgf/cm²) AZ Primer SHOFU Universal Primer

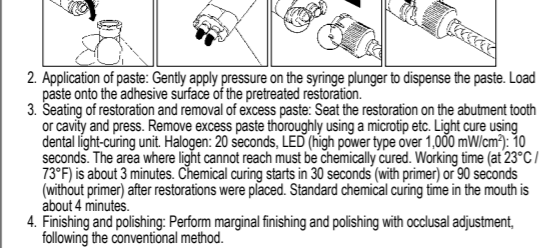NOTE ON USE
Store at refrigerator (1-10°C/34-50°F). Avoid high temperatures and high humidity.
Allow the materials to reach room temperature before use.
Do not use eugenol-containing material and Hydrogen peroxide since they might retard the curing process and weaken the bond strength.

COMPOSITION
Paste A: UDMA, TEGDMA, Fluoroaluminosilicat-Glas, Initiator
Paste B: UDMA, TEGDMA, Carboxylic acid monomer, 4-AET, 2-HEMA, Fluoroaluminosilicat-glass, Initiator
Primer A: Water, Acetone, Initiator
Primer B: 2-HEMA, 4-AET, Carboxylic acid monomer, Acetone
CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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