Hersteller/Manufacturer:  
DKL CHAIRS GmbH  
An der Ziegelei 3  
D-37124 Rosdorf  
(+49) 0551-50060  
info@dkl.de  
dkl.de

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## **User Manual**

Röntplex Strahlenschutz - Patient/Dental  
Röntplex X-Ray Protection - Patient/Dental

Deutsch 2-7  
English 8-13

Form-Nr./no. GDT/EN-RÖNTOPLEX  
Rev. 01 / 20.05.2021  
Änderungen vorbehalten.  
Subject to change without notice.

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Dear Customer,

Thank you for choosing a RÖNTOPLEX product. Please read this document carefully and follow all of the instructions, warnings and notices.

Please contact us if you have any questions or comments.

The manual is part of the product and therefore should be kept in the immediate vicinity, so that important information and safety instructions for use can be looked up at all times.

Do not give the product to a third party without a valid manual.

#### Disclaimers

The contents of the manual are for information purposes only. The products described are subject to change without notice in the course of continuous development. DKL only assumes warranty for this manual and the products within the scope of product liability.

#### Changes

Röntoplex products are subject to continuous further development. DKL reserves the right to modify the instructions, packaging and products without prior notice.

#### Translation

The master document is in German. In case of translation, precedence remains with the german version.

EN

#### Contents

Symbols	3
1.0 Identification of the Product	4
2.0 CE marking	4
3.0 Conditions of Transport & Storage	4
4.0 Manufacturer and Contact Information	4
5.0 Intended Purpose	4
6.0 Intended Use	5
7.0 Cleaning & Disinfecting	6
8.0 Product Safety	6
9.0 Warranty	7
10.0 Shipment / Return Shipment	7
11.0 Waste Disposal	7

## Symbols in the User Manual

WARNING!  
(risk of injury)

CAUTION!  
(to prevent  
damage occurring)

General explanations,  
without risk to  
persons or objects

## Symbols on the Product

With this symbol we declare compliance with the essential safety and performance requirements according to Annex I of Regulation (EU) 2017/745.

This product is a Medical Device.

Article Number

Serial Number

Manufacturer

Date of Manufacture

UDI - Product Identification

Observe the operating instructions!

The product must not be stored in direct sunlight!

The product must not be washed in the washing machine.

The product must not be tumble-dried.

The product must not be dry cleaned.

The product must not be ironed.

## Symbols on the Packaging

The product must be stored in a dry place.

Temperature limits to which the product can be safely exposed.

The product must not be stored in direct sunlight!

## 1.0 Identification of the Product

The production label contains the following information:

DPSE, DPSK, DSE, DSK, DSDE, DSSD for Model number,

description, lead equivalent, X-ray tube voltages according to IEC 61331-3:2014 / DIN EN 61331-3, CE marking, date of manufacture, corresponding standards and UDI-marking.

## 2.0 CE marking

With this symbol DKL declares that the products comply with the provisions in the applicable EU directives.

DKL declares that the products are in conformity with IEC 61331-3:2014, DIN EN 61331-3 and the Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. DKL is DIN EN ISO 13485:2016 certified.

## 3.0 Conditions of Transport & Storage

Limitation of the temperature range to -30°C to +50°C

Protect from heat

The product must be stored in a dry place.

USE DKL Apron holder for storing patient aprons. Store all other products on a flat surface. Radiation protection should not be folded or allowed to hang over edges, and must be stored away from sources of heat.

Ensure that protective apron is kept, stored and used only within the standard temperature and humidity range for the home and work environment.

## 4.0 Manufacturer and Contact Information

DKL CHAIRS GmbH  
An der Ziegelei 3  
37124 Rosdorf  
Germany  
+49 (0)551 50060  
info@dk.de  
www.dkl.de

## 5.0 Intended Purpose

These DKL products are used as patient protective devices in dental x-ray diagnostics for X-rays or panorama X-rays.

## 6.0 Intended Use

Dental radiation protective aprons are intended to protect the patient's chest, upper trunk, and thyroid from scattered radiation during dental radiological examinations of the teeth and the jaw.

**DPSE** Use patient protection **DPSE ( Adults )** and **DPSK ( children )** when taking panorama X-rays of standing patients. If Necessary, supplement the model with the thyroid protection DSDE.

**DPSK**

Put the apron on the patient so that the front of the body is covered from the neck to at least below the gonads as well as the entire sternum and shoulders and the protection is not in the imaging area. Close the hook and burr fastener in the front.

**DSE** Use patient protection **DSE ( Adults )** and **DSK ( Children )** when taking X-rays of sitting or supine patients. If necessary, supplement the model with the thyroid protection DSDE.

**DSK**

Put the apron on the patient so that the front of the body is covered from the neck to at least below the gonads as well as the entire sternum and shoulders and the protection is not in the imaging area. Close the hook and burr fastener in the shoulder area of the patient.

**DSDE** Use the patient thyroid protection to extend the protection of the dental radiation protective aprons.

Place the thyroid sternum shield on the patient so that it covers the front of the neck including the thyroid and overlaps the apron. Close the thyroid shield in the neck.

**DSSD** Use patient Protective X-ray Shield for Dental use when taking X-rays of sitting or supine patients.

Place the X-ray shield to the patient so that the front of the neck is protected.

### Insufficient Protective Area

An overlap of thyroid shield and radiation protection clothing is necessary to ensure continuous radiation protection. Non-continuous coverage with radiation protective clothing increases patient exposure to radiation.

The radiation protective materials achieve the following absorption values for the indicated X-ray tube voltages according to IEC 61331-3:2014 / DIN EN 61331-3:

Lead Equivalent	50kV	60kV	70kV	80kV	100kV	120kV	150kV
0,5 Pb mm	>99%	>99%	99%	98%	96%	94%	92%

## Sizes and Dimensions

Size	Symbol	Length	Width
Adults	DA1	80 cm	60 cm
Children	DC1	60 cm	45 cm

## 7.0 Cleaning & Disinfecting

Lacking hygiene can lead to infections of patients, operators/users, or third parties. Therefore clean and disinfect according to the applicable statutory provisions.

### 7.1 The following markings apply to the radiation protective products

The product must not be washed in the washing machine.

The product must not be tumble-dried.

The product must not be dry cleaned.

The product must not be ironed.

Do not sterilize the product.

### 7.2 Cleaning

Visible soiling must be removed immediately. Wipe the products with a damp cloth.

In order to avoid damage to the product, no aggressive, abrasive or corrosive agents should be used

### 7.3 Disinfecting

Disinfect the product after each use. We recommend wipe disinfection.

Do not use alcohol-based disinfectants, pure alcohol (e.g. ethanol and isopropanol), or oxidising as well as active chlorine or salt-containing disinfectants as they attack the surface of the outer material more strongly than disinfectants based on other chemicals. Avoid applying too much pressure while disinfecting. To maintain the properties of our radiation protective clothing, we recommend the use of the DKL disinfectant proPad. Those are Ready-to use wipes for disinfecting alcohol-sensitive surfaces.

## 8.0 Product Safety

Prior to each use of the product, check for visible damage. Do not use a damaged product!

To ensure the radiation protection still provides complete protection, it should be checked regularly, at least once every 3 years, tears, clearly visible kinks, protrusions, and uneven areas are all indications of a defect. We recommend that the radiation protective clothing is inspected in the style of the standard DIN 6857-2:2016. This involves a fluoroscopic examination with X-rays at regular intervals, in addition to regular visual and tactile tests.

A repair service is available through our factory in Rosdorf. We will gladly provide you with information about this. We do not accept any warranty for repairs not performed by DKL.

## 9.0 Warranty

DKL guarantees the safety and reliability of the product solely on the following basis:

- The product is used solely for its intended purpose and its operation complies with the information provided in these instructions for use.
- No unauthorised changes are made to the product.
- The product is regularly inspected and serviced at the specified times.

The workmanship and materials used have a 12-month manufacturer's warranty.

## 10.0 Shipment / Return Shipment

Röntoplex radiation protection must not be subjected to mechanical stress during shipment. It is not allowed to fold, or stack radiation protection. It is recommended that the products are loosely rolled and packed properly supported in a strong shipping box of a suitable size.

It is prohibited by law to send potentially contaminated goods. Any returns must therefore be properly cleaned and decontaminated.

## 11.0 Waste Disposal

Observe your local and national laws, guidelines, standards and regulations for disposal. You are welcome to contact us with any questions.