signum® liquid

DE

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GB Instructions for use

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GB

	KULZER <small>MITSUI CHEMICALS GROUP</small>
	Manufacturer: Kulzer GmbH Leipziger Straße 2 63450 Hanau (Germany)
	Importado e Distribuído por Kulzer South América Ltda. CNPJ 48.708.010/0001-02 Rua Cenzo Sbrighi, 27 – cj. 42 São Paulo – SP – CEP 05036-010 sac@kulzer-dental.com Resp. Técnica: Dra. Regiane Marton – CRO 70.705 Nº ANVISA: vide embalagem

FR

DE

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GB

Product description
Liquid for modelling and for reinitiating the dispersion layer

Range of application
– simplifying modelling procedures
– restoring the dispersion layer of light-curing veneering materials.

Composition
Dimethacrylate, multifunctional meth- acrylic acid ester, silane, photo-initiators

Handling
To simplify modelling procedures and to adapt veneering materials more easily, Signum liquid can be used to moisten the modelling instrument or brush. During the polymerisation of light-curing veneer- ing materials, a dispersion layer develops on the surface that forms the chemical bond between the individual layers. Do not touch or remove this layer. To restore a damaged dispersion layer, roughen the surface with a diamond and apply a thin layer of Signum liquid with a brush, wet- ting a larger area than needed. Allow Signum liquid time to react. If Signum liq- uid is not used, individual layers can dis- connect and flake off.

Note
Do not dilute pastes with Signum liquid. This will affect the mechanical and opti- cal properties of the veneering material in a negative way. Using too much Signum liquid will cause bubbles to form and may lead to a slippery surface which will make the application of the pastes more diffi- cult. If Signum liquid is applied in the mouth, a rubber dam must be used.

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Storage advice
Do not store material above 25°C (77°F). Avoid direct sunlight. Do not use the material after the expiry date. Always store containers tightly closed. Store out of the reach of children.

Hazard/Safety information
Causes skin irritation. Causes serious eye irritation. May cause an allergic skin reaction. Avoid breathing dust/fume/gas/ mist/vapours/spray. Wear protective gloves/protective clothing/eye protec- tion/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Take off contaminated clothing and wash before reuse. If skin irritation or rash occurs: Get medical advice/attention.

Safety data sheets and more information are available at our website www.kulzer-dental.com

® = registered trademark of Kulzer GmbH

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® = marchio registrato della Kulzer GmbH