# signum<sup>®</sup>

## metal bond

**KULZER**  
MITSUI CHEMICALS GROUP

<b>(DE)</b> Gebrauchsanweisung .....	3	<b>(HU)</b> Használati utasítás .....	69
<b>(GB)</b> Instructions for use .....	9	<b>(LT)</b> Naudojimo instrukcija .....	75
<b>(FR)</b> Mode d'emploi .....	15	<b>(PL)</b> Instrukcja obsługi .....	81
<b>(ES)</b> Instrucciones de uso .....	21	<b>(HR)</b> Upute za uporabu .....	87
<b>(IT)</b> Istruzioni per l'uso .....	27	<b>(RO)</b> Instrucțiuni de utilizare .....	93
<b>(PT)</b> Instruções de uso .....	33	<b>(TR)</b> Kullanma talimatı .....	99
<b>(NL)</b> Gebruiksaanwijzing .....	39	<b>(SK)</b> Návod na použitie .....	105
<b>(SE)</b> Bruksanvisning .....	45	<b>(BG)</b> Инструкции за употреба ..	111
<b>(FI)</b> Käyttöohjeet .....	51	<b>(SI)</b> Navodila za uporabo .....	117
<b>(GR)</b> Οδηγίες χρήσης .....	57	<b>(RS)</b> Uputstva za upotrebu .....	123
<b>(CZ)</b> Návod k použití .....	63		

Flüssig

Liquid

Liquide

Líquido

Liquido

Líquido

Vloeibaar

Flytande

Nestemäinen

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Kapalná

Folyékony

Skysta(s)

Płynny

Tekuć

Lichid

Sıvı şekilde

Kvapalné

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Teokoć

Tečno

**MD**

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Medical device

Dispositif médical

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Medisch hulpmiddel

Medicinteknisk produkt

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Stand: 2021-10

**Medical Device - for use by dental health care professionals only.**

**Intended purpose**

Bonding agent for crown, bridge and partial framework.

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

**Composition**

Signum metal bond I: Acetone; MDP; Acetic acid

Signum metal bond II: Methylmethacrylate; UDMA; Mixture of acrylpolymer in methylmethacrylate; titanium dioxide; silica; Diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide

## **Indications**

- Bonding agent between framework surfaces made of metal alloys (precious metal, non-precious metal or titanium) and (meth-) acrylate-based veneering materials in conjunction with macromechanical retention.
- Bonding agent between partial framework surfaces made of metal alloys (precious metal, non-precious metal or titanium) and (meth-) acrylate-based veneering materials or denture base materials (hot or auto-polymerisates).
- Bonding agent for extraoral repair of metal-supported composite veneers.
- Bonding agent for the extraoral repair of porcelain bonded restorations.

## **Contraindications**

In case of known or suspected allergy to components of the product, its use is contraindicated. Do not use this product in case of known or suspected allergies against (meth)acrylate compounds.

## **Potential side effects**

Hypersensitivities to the product or its components cannot be excluded in individual cases.

## Product application

Veneering of crown, bridge and partial framework

- Sandblast the surface to be veneered ( $\text{Al}_2\text{O}_3$  110–125  $\mu\text{m}$ , 3 bar). Jet angle approx. 45 °, distance between jet nozzle and object surface 2–4 cm, use disposable blasting material. Then clean with clean tap water (no steam jet) and dry carefully with clean compressed air (oil-free). Avoid surface contamination.
- Signum metal bond I is now placed drop by drop, per veneer unit, in the serving bowl and applied to the dry, dust- and grease-free surface with a disposable brush. Allow to air dry.
- Shake Signum metal bond II several times in order to achieve an optimal distribution of the color pigments on the surface during application.
- Then Signum metal bond II is filled into the serving bowl and applied with another disposable brush - cure in a polymerisation device (90 s in Heraflash, HiLite power or HiLite power 3D).
- In order to achieve an optimal adhesive bond, it is necessary to apply the required opaque without contamination of the intermediate layer (dispersion layer) through contact or a longer break and then to apply the components of the veneering system or the denture base materials (hot or self-curing polymers) from Kulzer.
- Close Signum metal bond I and II immediately after use. The application of Signum metal bond II must not be opaque. The color coverage is achieved by applying Signum opaque F.

### Repair / restoration of a veneered metal framework

- Carefully grind down the damaged area of the veneer down to the framework.
- Sandblast the framework surface as described above.
- Clean with water and compressed air, then dry carefully.
- The exposed metal surface is moistened with Signum metal bond I using a disposable brush. Allow to air dry. Signum metal bond II is then applied thinly with another disposable brush and cured with one of the named polymerization devices. Care must be taken to avoid contamination of the intermediate layers (dispersion layer) with impurities.
- Fracture surfaces of silicate ceramic veneers are conditioned with Signum ceramic bond in accordance with the instructions for use.
- Fracture surfaces of composite veneers are conditioned with Signum liquid in accordance with the instructions for use.
- The damaged or chipped area can then be repaired with light-curing veneering materials from Kulzer.
- Close Signum metal bond I and II immediately after use.

### **General information**

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

### **Warnings and safety instructions**

Highly flammable liquid and vapour. Wear protective gloves/protective clothing/eye protection/face protection. Avoid contact with skin. May cause an allergic skin reaction. Causes skin irritation. Skin contact may cause sensitization. If on skin (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower. Avoid contact with eyes. Causes serious eye irritation. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Avoid breathing dust/fume/gas/mist/vapours/spray. May cause respiratory irritation. May cause drowsiness or dizziness. For SSCP please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact [sscp-service@kulzer-dental.com](mailto:sscp-service@kulzer-dental.com)

### **Storage conditions**

Store in a well-ventilated place. Do not store above 25°C (77°F). Avoid direct sunlight. When product has been removed, reseal carefully. Do not use after expiry date.

### **Disposal information**

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

## **Complaint notice**

Please quote batch number **LOT** and article number **REF** in all correspondence about the product. According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Safety data sheets and more information are available at our website [www.kulzer.com](http://www.kulzer.com)

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® = registered trademark of Kulzer GmbH
Kulzer GmbH
Kulzer, LLC