Sani-Tip®

Conversion Kits

Sani-Tip® Kits de conversion / Sani-Tip® Kit di conversione / Sani-Tip® Umrüstkits / Sani-Tip® Kits de conversión

Instructions For Use

Mode d'emploi / Istruzioni per l'uso / Gebrauchsanleitung / Instrucciones de uso

Indications for Use

Used to transmit air or water to the operatory field during operative procedures.

Contraindications

This Conversion Kit is not indicated for use with any other air/water syringes.

Warnings

- Sani-Tip® Conversion Kits are designed to function exclusively with Sani-Tip disposable syringe tips. To minimize physical harm to patient, or equipment damage and/or malfunction, use only adapter components and clear Sani-Tip plastic syringe tips manufactured by Dentsply Sirona for the Sani-Tip® System. Follow the enclosed instructions.
- Sani-Tip® disposable syringe tips are for single only. Dispose after use. The risk of reuse of product for another patient could result in cross-contamination.
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- To ensure adequate protection from cross contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti-retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used

Precautions

- Although this conversion kit may be used with all Sani-Tip disposable syringe tips, it is recommended for use with the Standard (76mm) Sani-Tip disposable syringe tips.
- To help reduce cross contamination, use a new Sani-Shield disposable air water syringe sleeve for every patient.

Adverse Reactions

None known.

Residual Risks

- Tip is not properly installed into adapter.
- Cross-contamination can lead to patient infection or illness, use appropriate PPE when handling and dispose tip after each patient use.

Adapter Installation and User Instructions

For the parts diagram, installation and user instructions by conversion kit, visit www.dentsplysirona.com/IFU and search "Sani-Tip Adapter Installation and User Instructions with Troubleshooting".

INFECTION CONTROL PROCEDURES

Warnings

Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

Limitations of Processing

Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

Instructions

Point of use:

Remove excess soil with disposable lint-free cloth.

Containment and Transportation:

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

Preparation for decontamination:

Remove excess debris from the components using a disposable lint-free cloth. Discard cloth after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter O-ring.

Mechanical Cleaning: Ultrasonic Cleaners

- Follow manufacturer instructions for instrument cleaning solution, observing concentration rates and contact times.
- Prepare an enzymatic instrument cleaning solution per manufacturer's instructions paying attention to concentration rates. A pH neutral instrument cleaning solution such as Dentsply Sirona Resurge® Instrument Cleaning Liquid is recommended.
- Adhere to the ultrasonic cleaning bath manufacturer's recommended liquid level.
- Place the adapter, thumb nut, plastic cone and O-ring into the ultrasonic bath and cover for the indicated contact time; the instruments must be sufficiently covered.
- Remove the instruments from the ultrasonic cleaning bath. Rinse adapter, thumb nut, plastic cone and O-ring under tepid (potable) running water for 30 seconds to remove cleaning agent.
- Inspect the adapter, thumb nut, plastic cone and O-ring for visible debris. If debris remains, manually clean using a nylon brush wetted with the cleaning solution to remove it. Rinse the

adapter, thumb nut, plastic cone and O-ring under tepid (potable) running water for 30 seconds to remove cleaning agent.

- Dry with disposable lint-free cloth.

Cleaning: Manual

- Clean all external surfaces of the Sani-Tip conversion kit, including the air water/water syringe handpiece with a mild detergent and water solution. Dry with a disposable lint-free cloth.
- Prepare an enzymatic instrument cleaning solution per manufacturer's instructions paying attention to concentration rates. A pH neutral instrument cleaning solution such as Dentsply Sirona Resurge® Instrument Cleaning Liquid is recommended.
- Disassemble the conversion kit and remove plastic cone and adapter O-ring.
- Fully immerse the adapter, thumb nut, plastic cone and O-ring in the instrument cleaning solution for the recommended contact time.
- Rinse the components thoroughly under tepid (potable) running water for 30 seconds
- Inspect components for visible debris, if debris remains, repeat the cleaning procedure using a nylon brush wetted with the cleaning solution to remove it. Rinse components under tepid (potable) running water for 30 seconds to remove cleaning agent.
- Dry with a disposable lint-free cloth.

Disinfection:

- Prepare disinfection solution per manufacturer instructions.
- Fully immerse the plastic cone and O-ring in the instrument disinfection solution for the recommended concentration rates and contact time.

At the end of the contact interval, scrub briefly with a nylon brush dipped in the disinfection solution.

- Rinse the components thoroughly under tepid (potable) running water for 30 seconds.
- Dry with a disposable lint-free cloth.

Packaging:

Place thumb nut and adapter in a paper/plastic steam sterilization pouch. Components may be loaded into dedicated instruments tray.

Sterilization:

Note: The plastic cone is not steam autoclave sterilizable.

- Use a Dynamic Air Removal sterilizer (Pre-vacuum Steam Sterilizer/Cassette Pressure Pulse Steam Sterilizer). Place bagged adapter and thumb nut into the steam autoclave. Place pouch per manufacturer's instructions. After warm-up is completed, operate at a sterilizing temperature of 134°C/273°F for 3 minutes or 137°C/279°F for 3 minutes.
- **Alternate method:** Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature of 134°C/273°F for 3 minutes or 137°C/279°F for 3 minutes. Components should be used immediately.

Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

Maintenance, inspection and testing:

- Visually inspect to ensure all contamination is removed.

- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

Storage:

To maintain sterility, instruments should remain bagged until ready to use.

Manufacturer Contact

In the United States, contact Dentsply Sirona Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local Dentsply Sirona Representative.

Trouble Shooting Problem	Solution
<p>Water leaking, wet air, and/or loose tips, indicates either:</p> <p>a.) Center hole of Sani-Tip disposable tip is not fully seated over water spindle inside new adapter.</p> <p>b.) Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.</p> <p>c.) Internal water spindle is damaged.</p>	<ol style="list-style-type: none"> Loosen thumb nut, when interchanging Sani-Tip disposable tips. Insert new tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut. Make sure correct Sani-Tip Conversion Kit is properly installed. Confirm presence of plastic cone and all O-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by Dentsply Sirona for the Sani-Tip® System. If internal water spindle inside Sani-Tip adapter is damaged, replace adapter.
<p>Tip does not fit through thumb nut.</p>	<ol style="list-style-type: none"> Loosen thumb nut a full turn to expand opening of plastic cone.
<p>Little or no water</p>	<ol style="list-style-type: none"> Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.
<p>Too much/little air or water pressure.</p>	<ol style="list-style-type: none"> Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

For troubleshooting specific to conversion kits, visit www.dentsplysirona.com/IFU and search "Sani-Tip Adapter Installation and User Instructions with Troubleshooting".

ALWAYS TEST PRIOR TO USE:

Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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Dentsply DeTrey GmbH