carl
martin

FINEST DENTAL INSTRUMENTS

Secure
LOCK

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INSTRUCTIONS FOR USE

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3a.

- 3.-3a. Put and screw the injection needle into the ampoule. (Attention: at first, insert the ampoule into the holder and then screw onto the injection cannula.)

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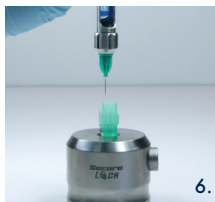
5.-5a. For the active use of the syringe, hold the safety stand and pull out the syringe of the needle cap - the syringe is now ready for use.

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6. After the use, the syringe will be put into the needle cap again and is consequently secured.

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7.-7a For the removal of the syringe together with the needle cap out of the safety stand, press the release button on the side by what the holding mechanism inside will come off.

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8.-8d. The injection needle is now unscrewed from the syringe head, inserted in the protective cap and then safely removed.

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9. The disposal of the contaminated needle is effected by the legally required guidelines.

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10. Within the context of a thermal processing (thermal disinfectant), we recommend to place the product head first (openings downwards) in the cleaning device. By this the rinsed disinfecting solution can run out of the openings again.

The safety stand SECURE LOCK is suitable for the autoclave (max. 134°C)

QSA313
www.carlmartin.de

NEUENKAMPER STR. 80 - 86 · D-42657 SOLINGEN · GERMANY

TEL. +49(0)212/81 00 44 · FAX +49(0)212/8 73 45

E-MAIL: INFO@CARLMARTIN.DE · INTERNET: WWW.CARLMARTIN.DE