FR

**MODE D'EMPLOI**

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EN

**INSTRUCTIONS FOR USE**

- Break the sterility guarantee seal just before use by turning the two parts of the protective sleeve in opposite directions.
- Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
- Remove the plastic cap just before the injection is administered. It can be used again to remove the needle from the syringe to avoid risk of needle stick injury.

- Discard the used needle into a puncture-resistant container in accordance with all applicable medical waste regulations.

**WARNINGS**

- Never recap needles by hand. Use only accepted dental recapping procedures. Recapping needles by hand or recapping bent or broken needles can cause unintended needle sticks.
- Do not bend, break, or otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury. Do not use short needles (< 30 mm) when the expected depth in soft tissue approaches the length of the needle.
- Avoid extreme pressure and excessive needle movement during injection as this may cause needles to break, which may result in serious injury to you and/or your patient.
- Be especially vigilant when repositioning a needle in a patient who appears to be apprehensive.
- For Septoject XL: avoid either too rapid of an injection and/or excessive pressure as that may lead to increased patient pain during injection

**CAUTIONS**

- Store in original packaging until ready to use.
- Read the Instructions For Use supplied with your injection device.
- If seal is broken, the needle is NOT STERILE and MUST NOT BE USED.
- For single use only: This device is for use in a single patient. Any reuse exposes other patients and the user to critical risks, including infection and trauma since the needle is no longer sterile and is more likely to break."
- If multiple injections are required, replace the needle for each cartridge in order to minimize the risk of needle point damage.
- 1 patient / 1 cartridge / 1 needle
- Advice for first time users of Septoject XL needles: Because XL needles have a larger lumen (bore) than other needles, prior to injection, soft tissue should be stabilized to reduce the risk of needle deflection.

**PRESENTATION**

100 two-point disposable non-resterilisable dental needles

*For professional dental use only*

AU

**Sponsor:** Specialities Septodont Pty Ltd.  
14, Cliffbrook Crescent, Emu Plains, NSW 2750 - Australia

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