# Shaded PMMA Disc Temporary Crown and Bridge Provisorische Kronen und Brücken Couronne et bridge temporaire Temporanea per corone e ponti Temporal de corona y puente Tijdelijke kroon en brug

Instructions for Use  
Gebrauchsanweisung  
Mode d'emploi  
Istruzioni per l'uso  
Instrucciones de uso  
Gebruiksaanwijzing

THE DENTAL  
SOLUTIONS  
COMPANY™

 Dentsply  
Sirona

## Indications for Use

Dentsply Sirona Shaded PMMA Discs are indicated for manufacturing temporary crowns and bridges by a CAD/CAM system.

en

## Contraindications

- May not be used for patients with hypersensitivity to methacrylate or any of the other ingredients.

## Warnings

- Possible cross-reactions or interactions of this material with other material(s) already present in the oral environment must be taken into consideration by the dentist.

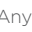
## Precautions

- Keep product dust away from eyes.
- After use, wash your hands.
- Do not swallow the product.
- Do not inhale dust particles during grinding.
- Use local vacuum suction and suitable mouth/face protection during manual machining at the workplace.

## Adverse Reactions

- If properly processed and used adverse effects, are highly unlikely. However, reactions of the immune system (such as allergies) to the substances contained in the material or localized paraesthesia (such as taste disturbances or irritation of the oral mucosa) cannot be completely ruled out.
- Residual dust during milling operation may cause irritation.

## Symbols on product labels

	Manufacturer
	Batch Code
	Reorder Number
	Manufacture Date
	Expiration Date
	Consult Instructions for Use
	Caution
	Keep Away from Sunlight
	Medical Device
	U.S. Prescription Only
	CE European Conformity Marking

Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

## Technical Specifications

Meets applicable requirements of ISO 10477: 2018[E] Dentistry - Polymer-based Crown and Veneering Materials.

## Requirements for Milling Temporary Crowns & Bridges

- > Minimum thickness: 1mm occlusion and 0.6mm cervical
- > Minimum connection diameters: Anterior: 3.6mm; Posterior: 4.4mm
- > The provisional can be fixed with all cements for temporary fixation.

## Step-by-Step Instructions for Milling Temporary Crowns & Bridges

### Procedure for Fabrication

- Place the disc on the machine following the equipment manufacturer's instructions.
- After milling, remove the disc from the equipment and remove the sprues.
- Finish and polish using conventional techniques.

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